

Product of:



MR SYSTEMS
MONITORED REHAB SYSTEMS

MR Cube manual 4.5.0.7



MR CUBE



MRS-E1301

Table of contents

1. User manual MR Cube	3
2. Add Bluetooth connection	7
2.1 Bluetooth installation Windows XP	7
2.2 Bluetooth installation Windows Vista	12
2.3 Bluetooth installation Windows 7	15
3. MR Cube installation	17
3.1 Introduction	17
3.2 Minimal requirements:	17
3.3 Permissions	17
3.4 Autorun	20
3.5 Server Installation	21
3.5.1 Install Firebird	22
3.5.2 Install .NET Framework	27
3.5.1 Install XNA Framework	31
3.6 Client Installation	36
3.7 Uninstall	40
4. Registration	43
5. Setting the COM port	46
6. Patient	46
5.1 Indication	46
5.2 Extended history	47
7. Quick Start menu	49
8. Rehab	50
8.1 Rehab	50
8.2 Mixed Rehab	53
8.3 Rehab History	54
9. Exercises	56
9.1 Functional exercises	56
9.2 Functional exercises: the scores	60
9.3 Mixed Exercises	60
9.4 Exercise history	61
10. Challenging Programs	62
10.1 Dual Tasks	62
10.1.1 Mathematics	62
10.1.2 Recognition	65
10.1.3 Memory	67
10.2 Sports	70
10.2.1 Race game	70
10.2.2 Ski game	73
10.2.3 Football game	76
11. Testing with the MR Cube	79
11.1 Coordination Test	79
11.2 Interpretation Coordination Test	83
11.3 Power Test	85
11.4 Proprioceptive Test	88
11.5 Interpretation Proprioceptive Test	93
11.6 Response time test	95
11.7 Interpretation response time test	99
12. Test history	99
13. MR Cube Settings	100
13.1 Show serial	100
13.2 Settings	101
13.3 Administrator Settings	102
13.4 Back-up and restore	104
13.5 Institute Management	104
14. Shortcuts	106

1. User manual MR Cube


The MR Cube system includes:

- MR Cube
- Battery charger including manual of battery charge
- Velcro tape with D-ring
- Karabiner
- Magnet



The MR Cube

The MR Cube exists of different parts and accessories. In the following paragraphs the working of those parts and accessories is explained.

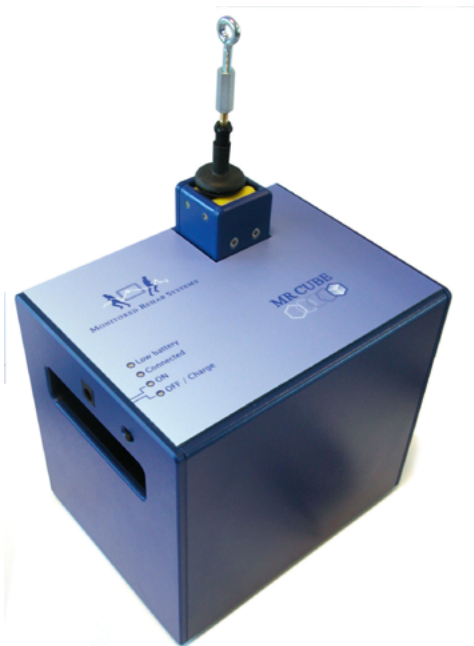

The MR Cube cable

The MR Cube cable is the cable which makes the MR Cube measure the range of motion and the user's position during training. The cable is guided into the MR Cube by the yellow rollers. At the end of the cable an eye is attached which enables the user to use the MR Cube in many different ways. To pull out the cable a minimum Force of about 5 Newton (500 gram) is needed.



IMPORTANT:

DO NOT LET THE CABLE SNAP BACK. DO NOT TRAVEL OVERRANGE (max 2.3 meter). THIS CAN CAUSE DAMAGE TO THE MR CUBE.

The ON/OFF slide switch

At the same place where the LED'S are attached on top of the MR Cube, the ON/OFF slide switch is attached on the side. This button is a slide switch. The On and OFF position are indicated with the lines on top of the MR Cube.



Battery charger connector

Besides the ON/OFF slide switch, the connector for the battery charger is attached.

On top of the MRCube 4 LED's are attached. These LED's are:

Low battery LED

This LED will start burning if the supply voltage from the battery becomes low and the charger is not connected. After this led starts burning, the MR Cube can still be used for about one hour without the charger being connected.

Connected LED

This LED will start burning if a connection is established between the MR Cube and the computer which runs the MR Cube Software.

ON LED

This LED will start burning if the MR Cube is turned on. It makes no difference if the MR Cube is working with or without the charger connected.

OFF / Charge LED

This LED will start burning if the MR Cube is turned OFF and the charger is connected. If the MR Cube is turned on and the charger is connected, the OFF / Charge LED will not burn.

The Battery Charger

The battery charger can be connected to the battery charger connector and can be used as a charger as well as a voltage supply.

If the MR Cube is OFF and the battery charger is connected, the battery is charged and the OFF / Charge LED will burn.

If the MR Cube is ON and the battery charger is connected, the charger is used as a voltage supply and as a charger. In this case, only the ON LED will burn.

On the battery charger itself, another led is attached. The colors of this led will change depending on the status of the charge cycle.

Yellow: No battery connected/Idle state

Orange: Charging

Green: Ready

Orange/Green: Error (reconnect the charger)

Warning for MR Cube users in USA: Do not use MR Cube if battery charger is connected.



The Karabiner

The karabiner can be used to attach the cable from the MR Cube to the D-ring of the Velcro tape or to other equipment.



The Velcro tape with D-ring

The Velcro tape can be used to attach the MR Cube to a dumbbell, barbell or other equipment. Make sure the Velcro tape cannot slide off the equipment while being used. Snapping back the cable into the MR Cube can cause damage to the MR Cube.



The magnets

The magnet with moveable hook can be attached on top of a weight stack. The eye at the end of the MR Cube cable can be attached to the hook.

The two other magnets can be used to attach the MR Cube cable to other equipment.



IMPORTANT:

MAKE SURE THAT THE MR CUBE CABLE IS ATTACHED PROPERLY. LETTING THE CABLE SNAP BACK, CAN CAUSE DAMAGE TO THE MR CUBE.

2. Add Bluetooth connection

This Chapter will explain how to add a connection between the MR Cube and a computer:

To connect the MR Cube to a computer, the computer needs to be provided with Bluetooth. This can be a default function on your computer, or you can add this function by installing a Bluetooth USB converter. The following instructions are about installing the MR Cube on a computer which is provided with Bluetooth. The pictures in these instructions may vary from the screens you will see during installation. This is because of the different Microsoft operating systems and the different Bluetooth configuration programs.

If the MR Cube is connected to another than the default computer, the Bluetooth pairing should be done again. The MR Cube should be switched off before re-use. We do not recommend to use one MR Cube on more computers according to the above extra actions.


2.1 Bluetooth installation Windows XP

To start the installation, open My Bluetooth Places by double clicking the Bluetooth icon on your desktop, taskbar or in the Control Panel.



Make sure the MR Cube is turned on and that the serial number of the MR Cube is known. The serial number can be found at the bottom of the MR Cube.

In My Bluetooth Places, click "Add a Bluetooth Device".

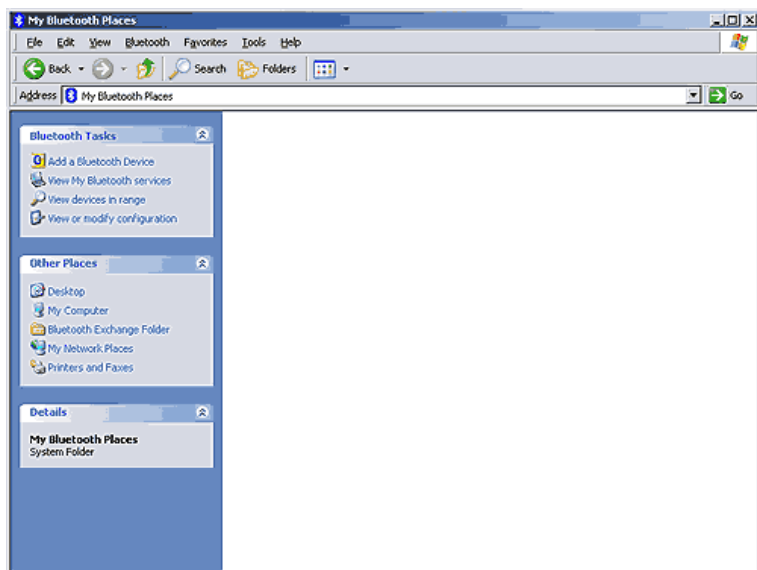


The Bluetooth Setup Wizard will start:




Press <Next> to continue.

The wizard will show all available devices. If your MR Cube is found by the wizard, it will be shown on the Bluetooth Device Selection screen. The name of the device must be MR Cube + "serial number". In this case MRCube014. If your computer cannot find the MR Cube, make sure the MR Cube is turned on and try again.


If your MR Cube is found, double click it or select it and press <Next> to continue.
If your MR Cube is still not found, please contact your dealer.

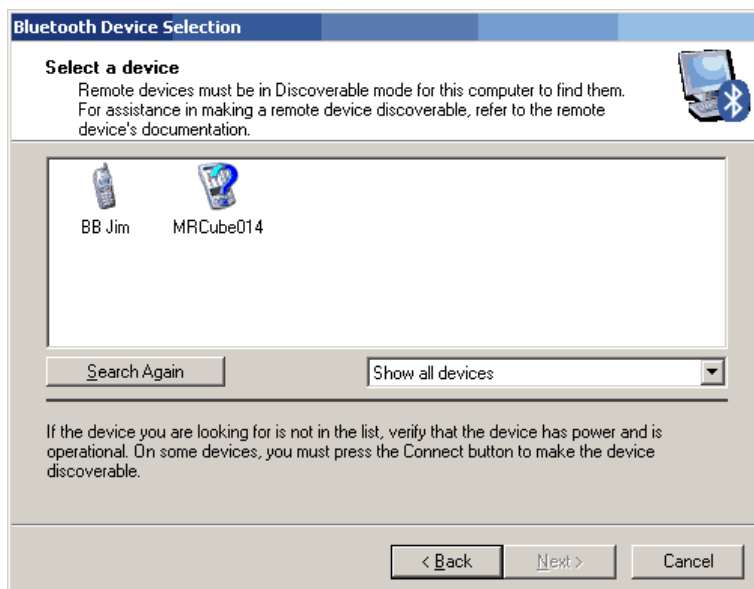


The Bluetooth Security Setup appears. In the field "Bluetooth security code:" field the security code must be typed. For the MR Cube, this code is 2031. After typing the code, press <Pair Now>.

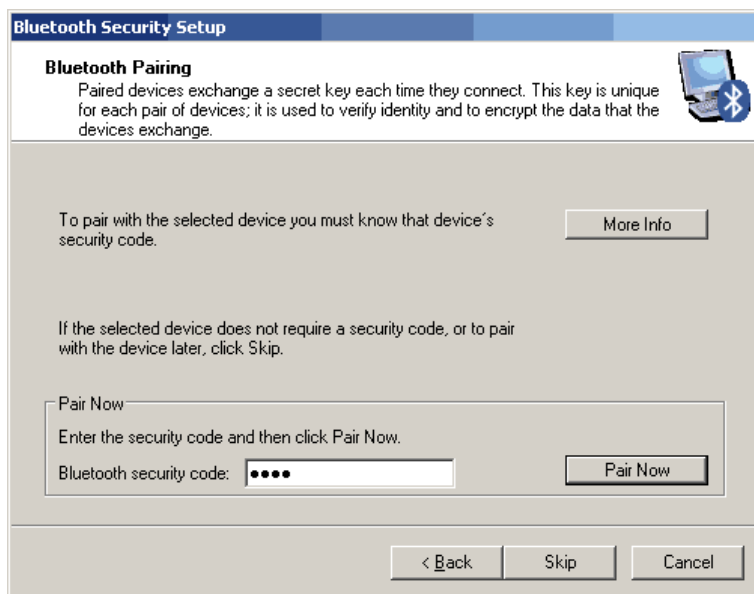

The "Bluetooth Service Selection" screen appears. This screen shows the available services. Select "SerialPort" and press <Next> to continue.

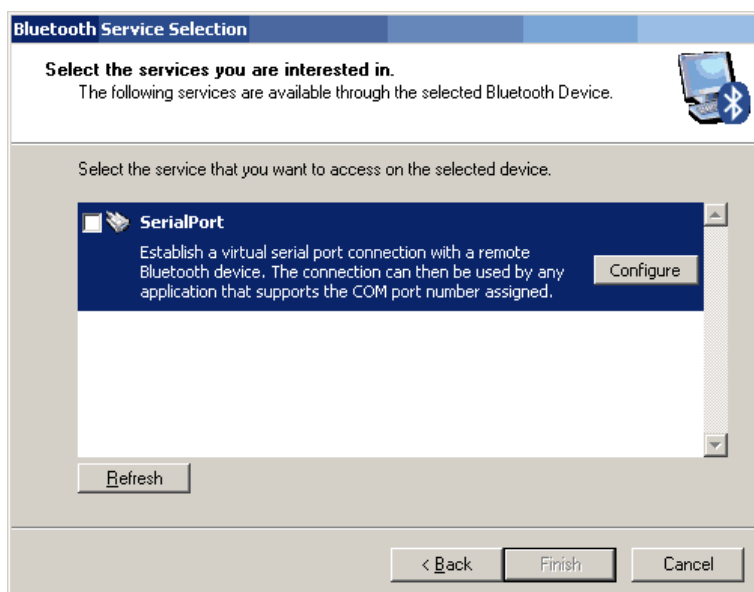


The following message may appear. Press <OK>.

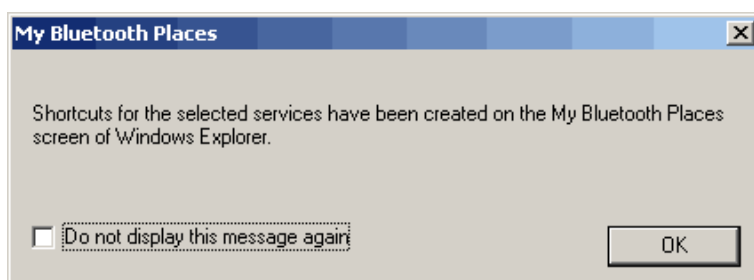


The "Bluetooth Setup Wizard Completion Page" appears.




Press <Finish> to continue. The screen closes and the MR Cube is installed. Open "My Bluetooth Places". The MR Cube is shown and connected. Right click the MR Cube and select <Disconnect>.

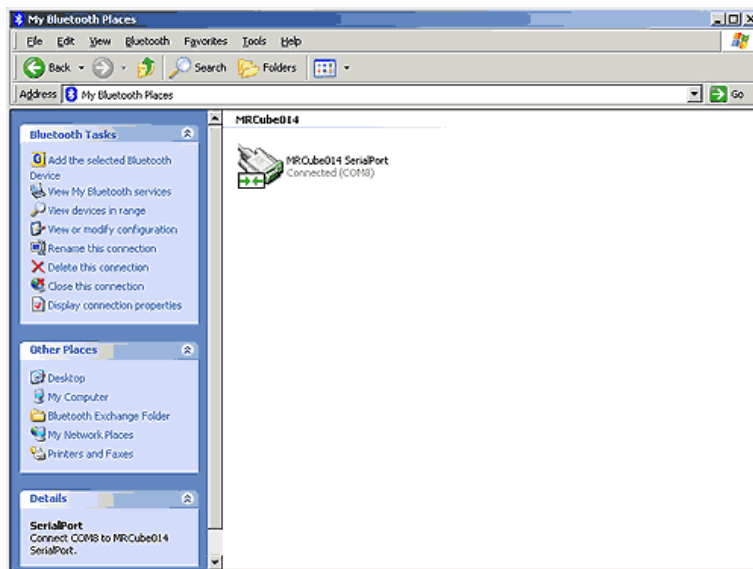

Right click again and select "Properties".

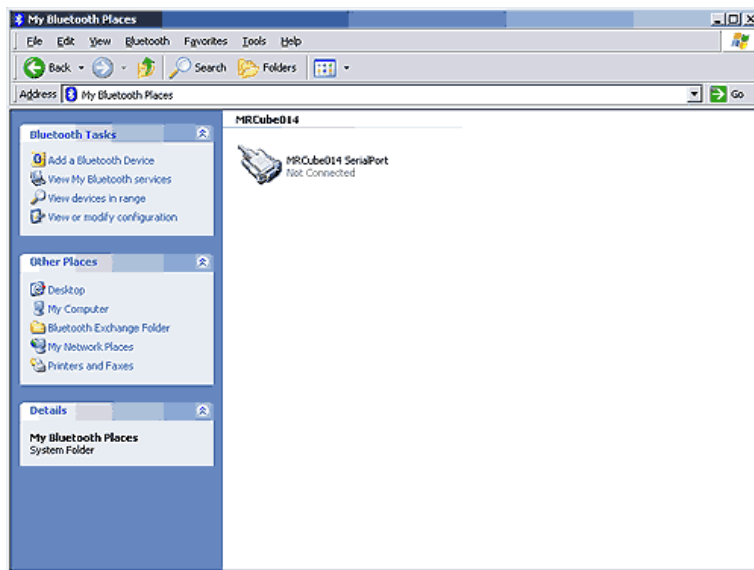

"Bluetooth Properties" appears and the COM Port is shown. This COM Port number must be used in the MRS software. Press <OK> to continue.

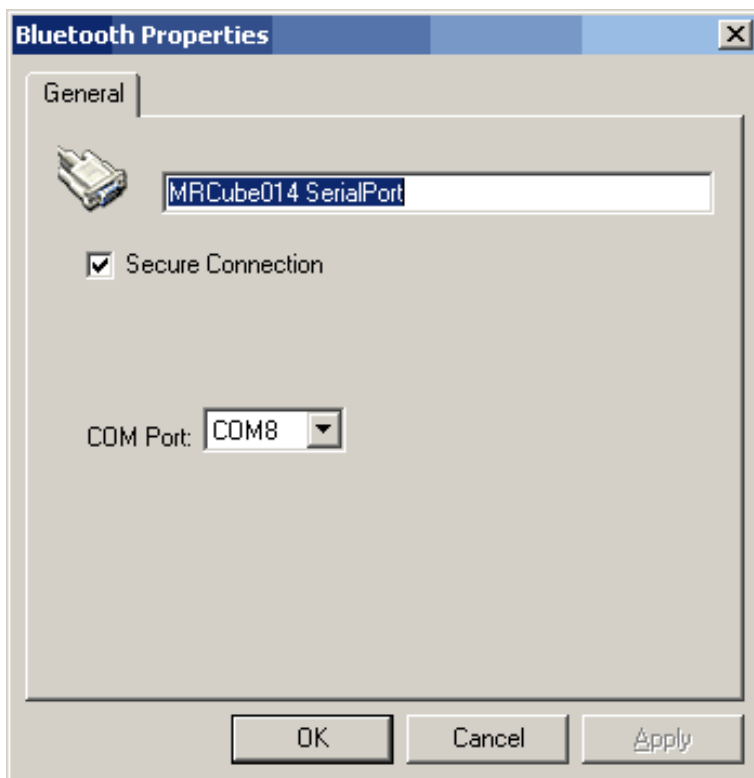


2.2 Bluetooth installation Windows Vista

How to add a connection between the MR Cube and a computer:


To connect the MR Cube to a computer, the computer needs to be provided with Bluetooth. This can be a standard function on your computer, or you can add this function by installing a Bluetooth USB converter. The following instructions are about installing the MR Cube on a computer which is provided with Bluetooth. The pictures in these instructions may vary from the screens you will see during installation. This is because the different Microsoft operating systems and the different Bluetooth configuration programs.

To start the installation, open My Bluetooth Places by double clicking the Bluetooth icon on your desktop, taskbar or in Control Panel.

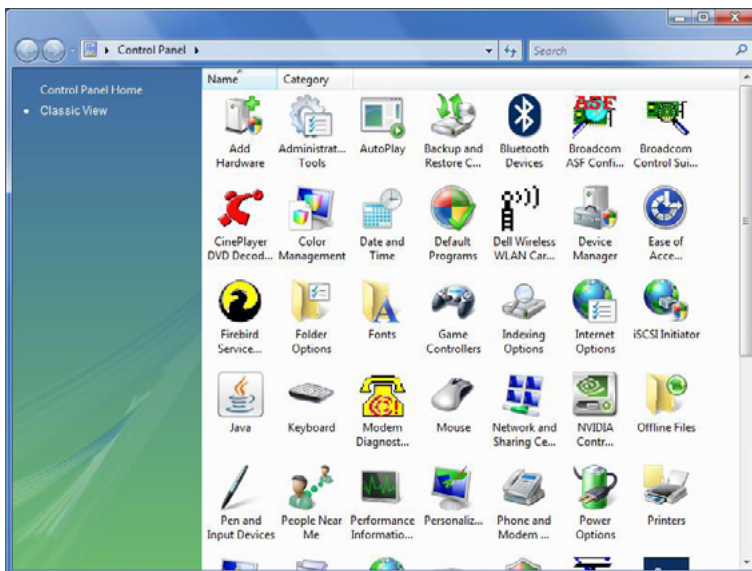

Make sure the MR Cube is turned on and that the serial number of the MR Cube is known. The serial number can be found underneath the MR Cube.

In Bluetooth Devices, click <Add...>

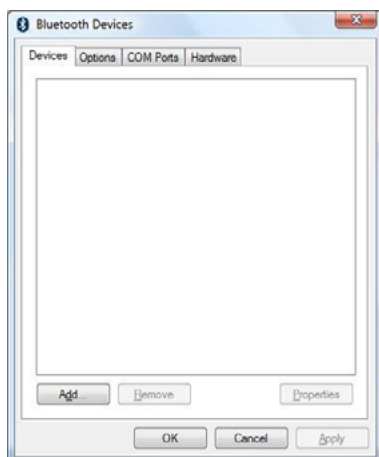


The Add Bluetooth Device Wizard will start.

Make sure the MR Cube is turned on and select "My device is set up and ready to be found".


Press <Next> to continue.



The wizard will show all available devices. If your MR Cube is found by the wizard, it will be shown on the Bluetooth Device Selection screen. The name of the device must be MRCube + "serial number". In this case MRCube014. If your computer cannot find the MR Cube, make sure the MR Cube is turned on and search again. If your MR Cube is found, double click on it or select it and press <Next>.

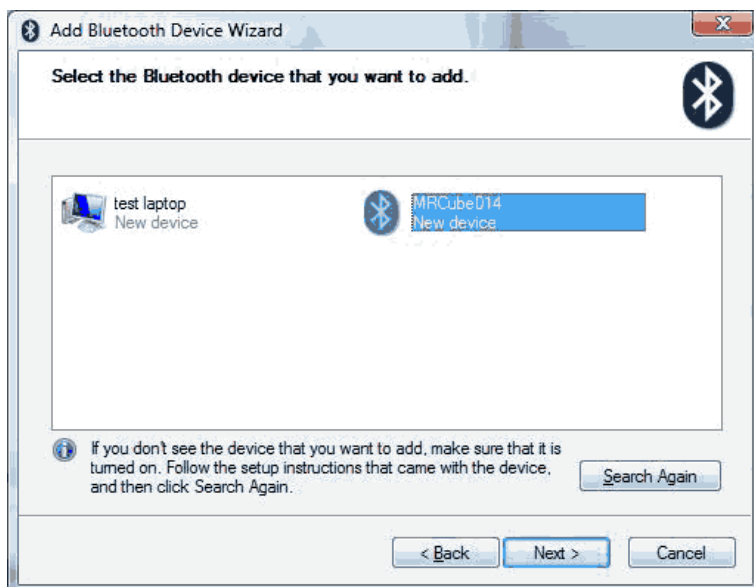


Select "Use the passkey found in the documentation:" For the MR Cube, this code is 2031. After typing the code, press <Next> to continue.

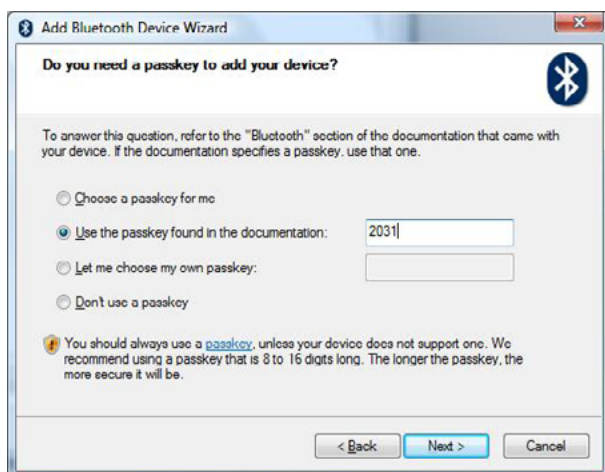

The assigned COM ports are shown in the following screen. The outgoing COM port is the COM port to use in the Monitored Rehab Systems Software. In this case COM 4. Press <Finish> to continue.



The installed MR Cube is shown in the Bluetooth Devices. Press <OK> to continue.

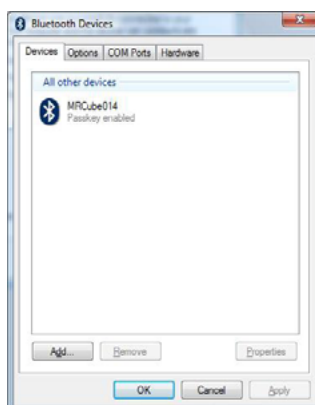


2.3 Bluetooth installation Windows 7


How to add a connection between the MR Cube and a computer:

To connect the MR Cube to a computer, the computer needs to be provided with Bluetooth. This can be a default function on your computer, or you can add this function by installing a Bluetooth USB converter. The following instructions are about installing the MR Cube on a computer which is provided with Bluetooth. The pictures in these instructions may vary from the screens you will see during installation. This is because the different Microsoft operating systems and the different Bluetooth configuration programs.


To start the installation, open My Bluetooth Places by double clicking the Bluetooth icon on your desktop, taskbar or in Control Panel.

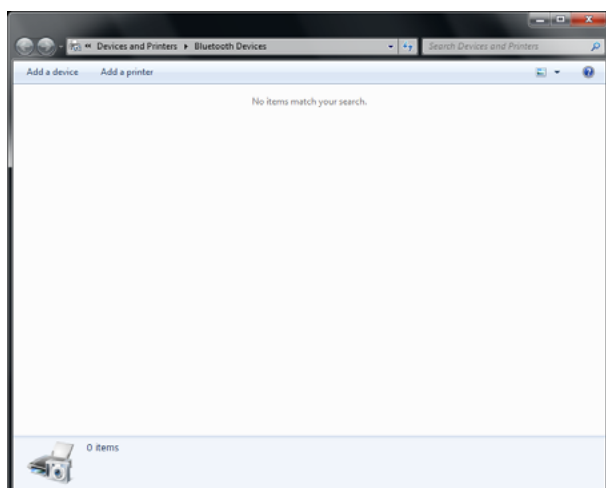

Make sure the MR Cube is turned on and that the serial number of the MR Cube is known. The serial number can be found at the bottom of the MR Cube. In Bluetooth Devices, click <Add a device>.

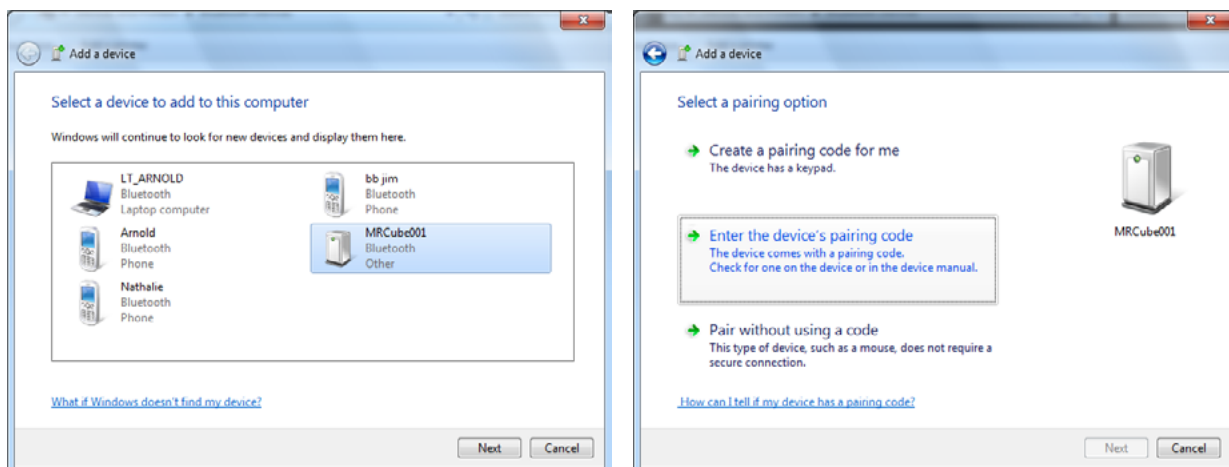


Make sure the MR Cube is turned on. Press <Next> to continue. Select "Enter the device's pairing code".

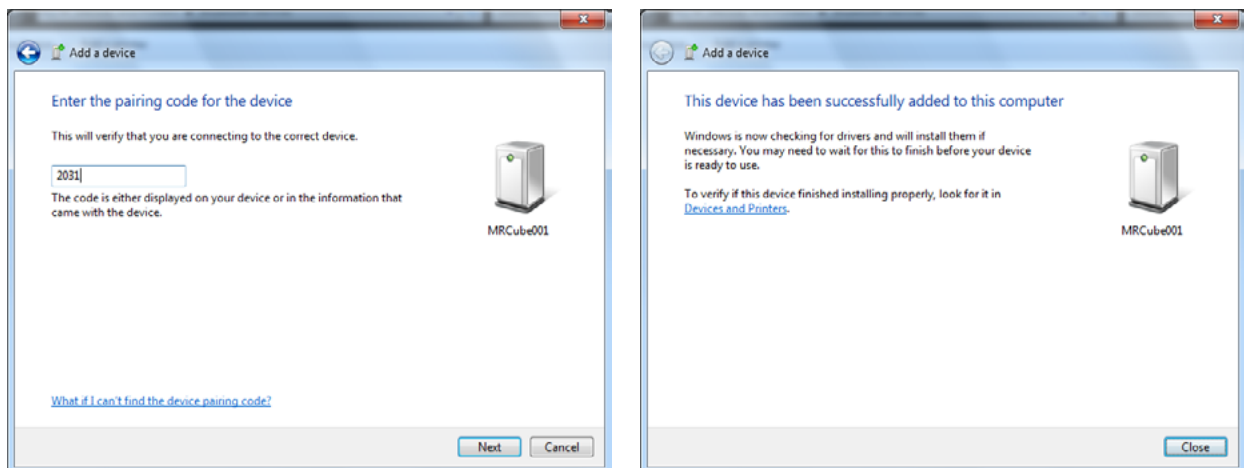

For the MR Cube, this code is 2031. After typing the code, press <Next> to continue.

To verify the correct COM port, open the Control Panel. Choose Hardware and Sound - Devices and printers. Double click the MRCube01 device.

The assigned COM ports are shown in the following screen. The outgoing COM port is the COM port to use in the Monitored Rehab Systems Software. In this case COM 4.

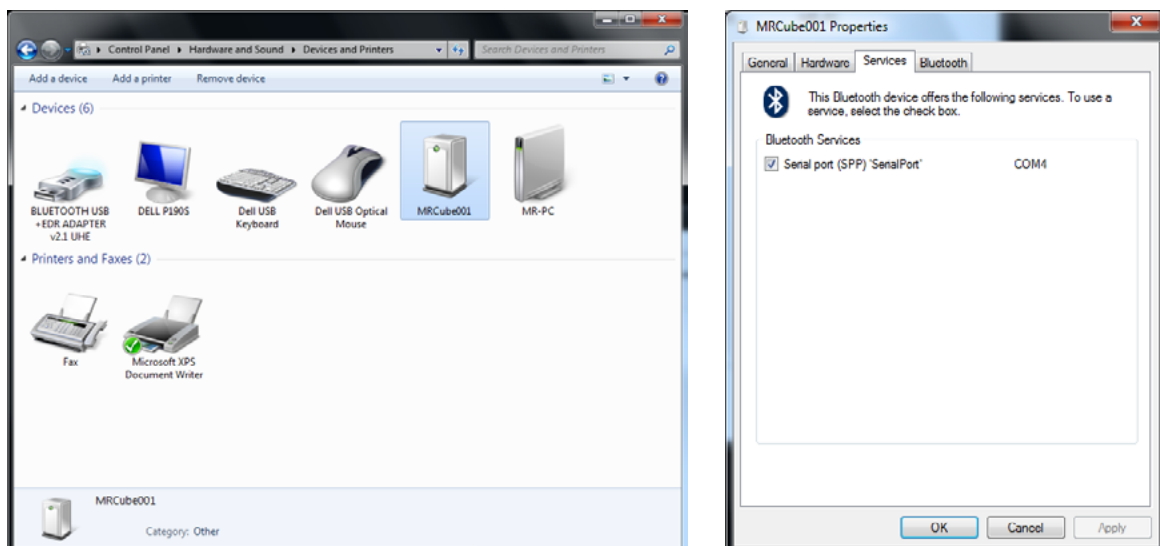

3. MR Cube installation

3.1 Introduction

In this chapter the installation of Monitored Rehab Systems software is described. The server and the client installation are clearly explained step by step. Keep this installation manual next to you during the installation.

A number of terms, which are used in the manual, are explained here:

Server: The computer in the network where the database is stored, it is the most important computer. The client computers are connecting to this database.

Client: A workstation. Clients make connection with the database, which is located on the server.

Firebird Server: A program that enables the communication between MRS software and the database.

IP address: A kind of address from the computer; every computer in the network has his own IP address. (For example 192.168.0.1)

Database path: a routing to the location where the database is stored on the hard disk. The default path is generally: "C:\Program Files\MRS Software\Database\MRS.gdb".

3.2 Minimal requirements:

Processor: Pentium IV or compatible AMD processor

Operating system: Windows XP/Windows Vista/Windows 7

Memory: 2048 Mb

Hard disk: 2 Gb available

Screen resolution: 1024 x 768

Graphic adaptor: ATI or Nvidia DirectX 10 comp. Vertex Shader 3.0 comp. with at least 256 MB VRAM

Extra for MR Cube: Bluetooth (onboard or USB)

3.3 Permissions

You need administrator rights to install the MRS software.

Users of the software needs permissions (full control) on the following folders:

Windows XP:

- C:\Program Files\MRS Software
- C:\Program Files\Firebird
- C:\Documents and Settings\All users\Application data\MRS
- C:\Documents and Settings\[USER]\Application data\MRS

Windows Vista/Windows 7:


- C:\Program Files\MRS Software*
- C:\Program Files\Firebird*
- C:\ProgramData\MRS
- C:\Users\[USER]\AppData\Roaming\MRS

* If you are using a 64 bit version of windows the "Program Files" folder is named: "Program Files (x86)".

Windows Vista/Windows 7: User Account Control (UAC) must be turned off.



How to add permissions

To add permissions, browse to the "MRS Software" folder:

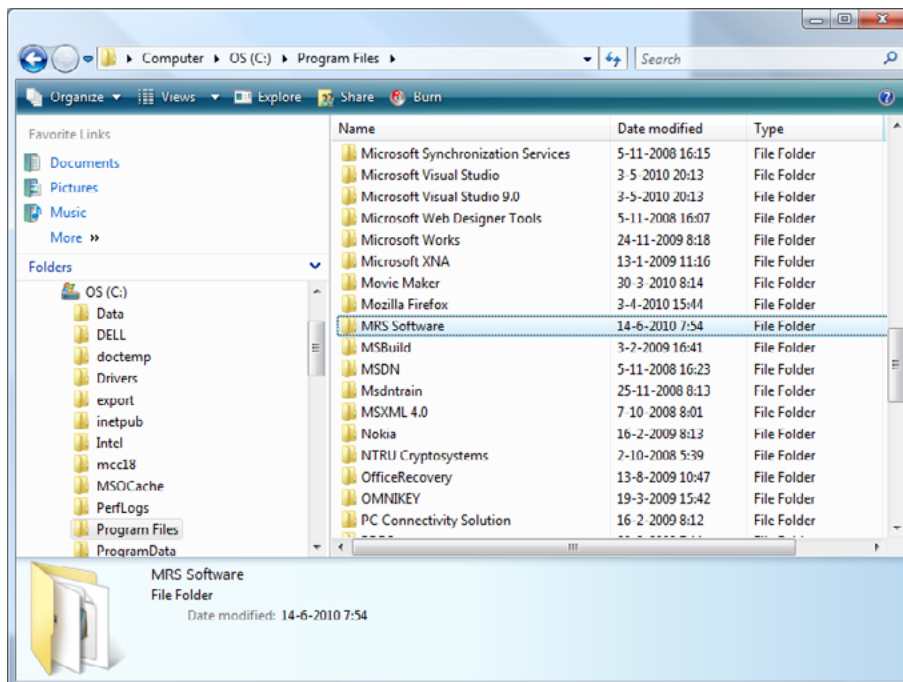


Right click the "MRS Software" folder and choose properties.


Select the tab "Security".

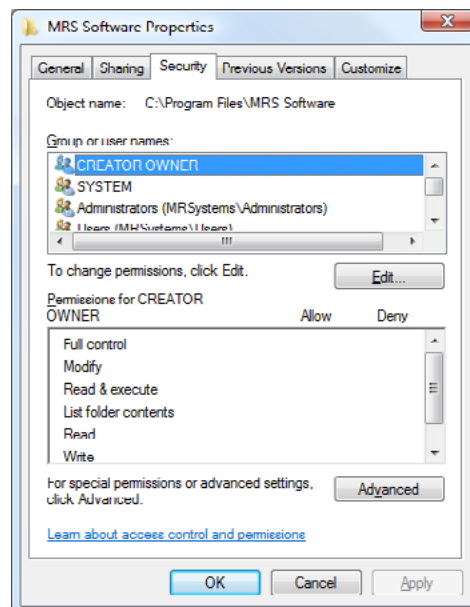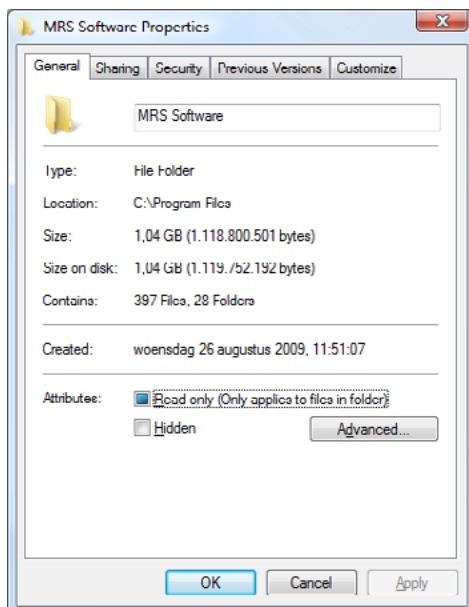

Click <edit>.

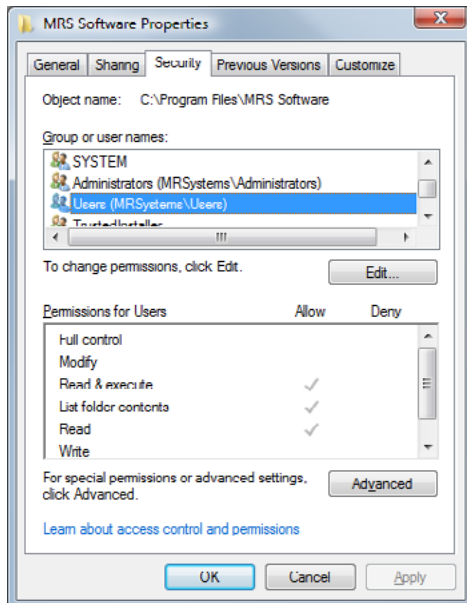


Select "Users"

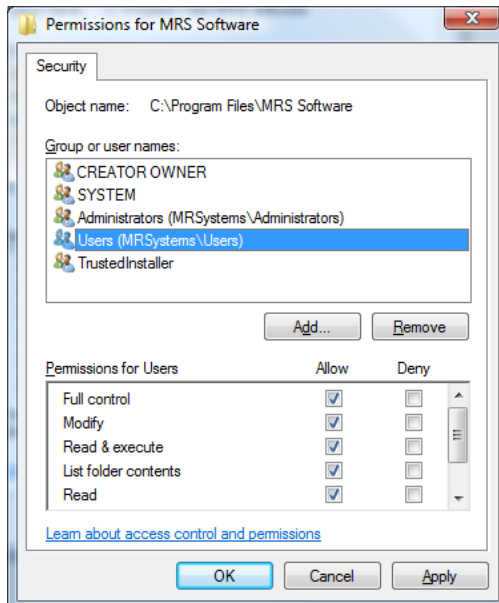


Select Full control and click <Apply>. All users have all permissions on the "MRS Software folder".

How to turn off user account control (UAC)


To turn off user account control, browse to Control Panel and select "User Accounts".

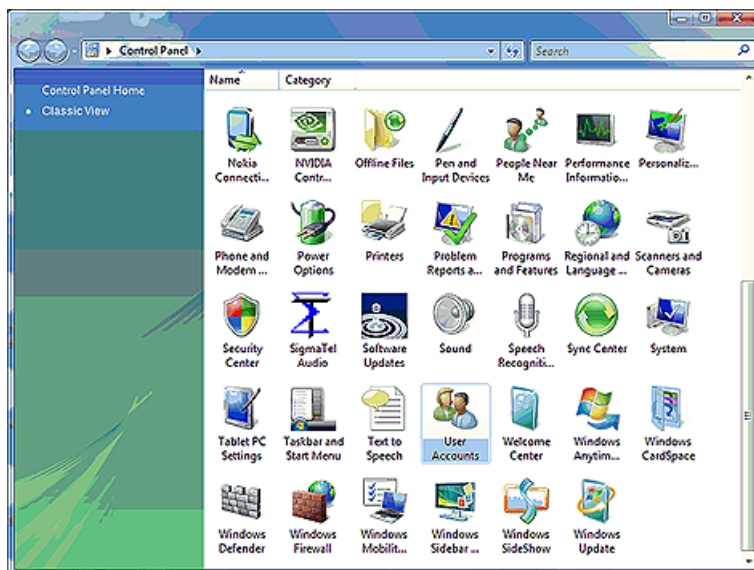

Click "Turn User Account Control on or off".

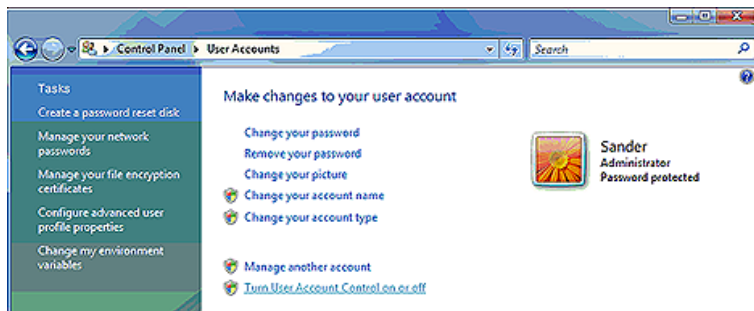


If Windows asks permission, choose <Yes>. The following screen will appear. The checkbox needs to be unchecked. If it is already unchecked, leave it this way. Otherwise, uncheck the checkbox and press <OK>. The computer will ask to restart. Choose <Yes>.

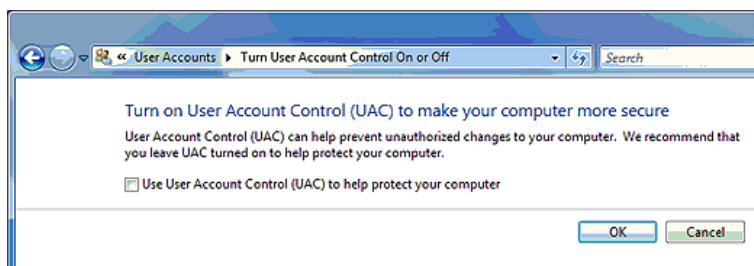

3.4 Autorun

Put the installation CD in the CD-Rom player of the computer. If "Autorun" is switched on, the following screen appears:

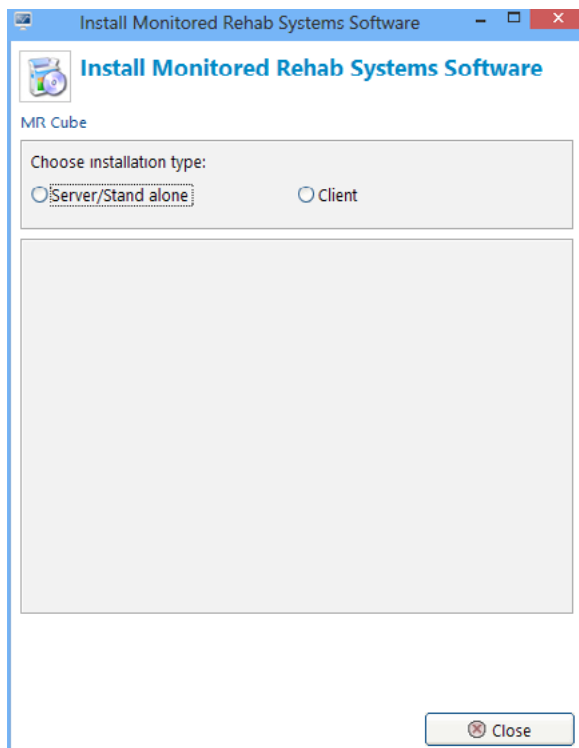

If the screen doesn't appear, double-click on the icon "my computer" on your desktop. Select the CD-Rom player (generally the D-disk) and click on your right mouse button. Choose <Open> in the menu that appears. Now double-click on the icon "installMRS.exe".

The following software is required:

- Firebird 2.5 (only on server)
- .NET Framework 4.0
- XNA Framework 4.0

If they are not installed, press the buttons to install the software.


In the menu can be chosen between the server and the client. Always start with installing the server. Click therefore check <Server/Stand alone>.

Attention!

Always install the server first, with a new installation or update. After this, install the clients.

3.5 Server Installation


After choosing to install the server, this screen appears:

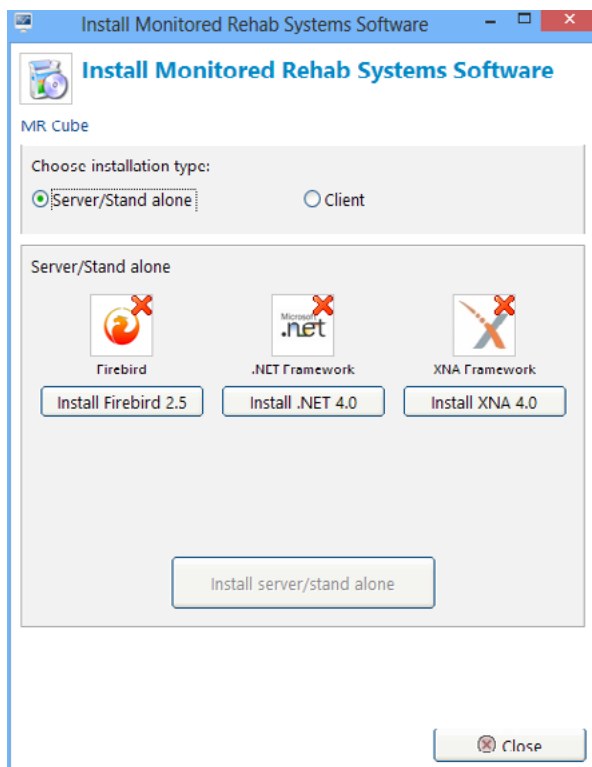


The red crosses indicate that the necessary depending software is not yet installed on the server. Before continuing the installation of the MR Cube, these programs have to be installed. Install from left to right.

3.5.1 Install Firebird


After selecting <Install Firebird 2.5> this screen appears:

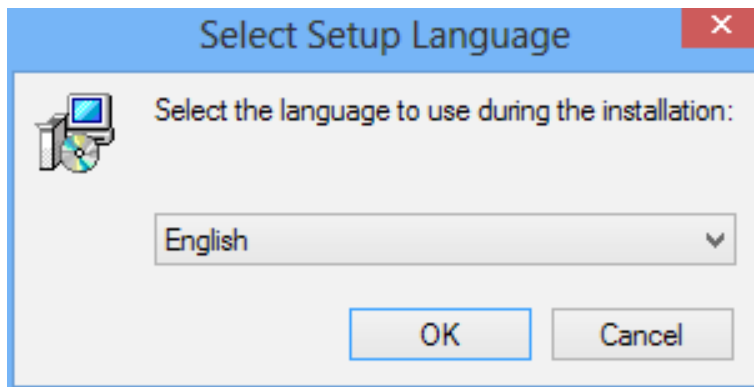


Choose the desired language and press <OK>.

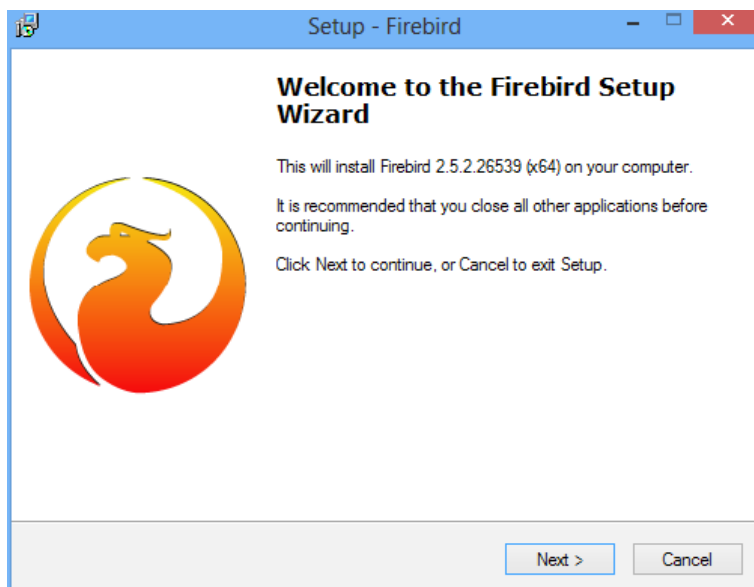


Firebird advises to close all other applications before installing.
Close the running applications if you like.
press the button <Next>.

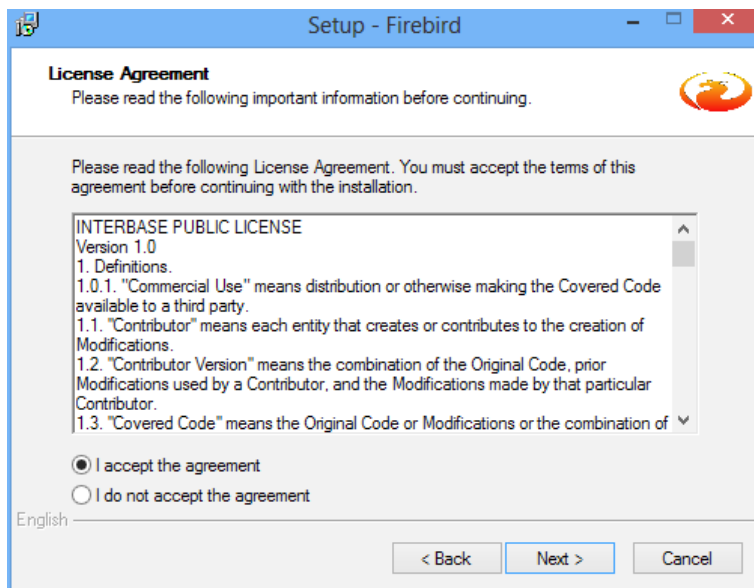


Check "I accept the agreement" and choose <Next>.

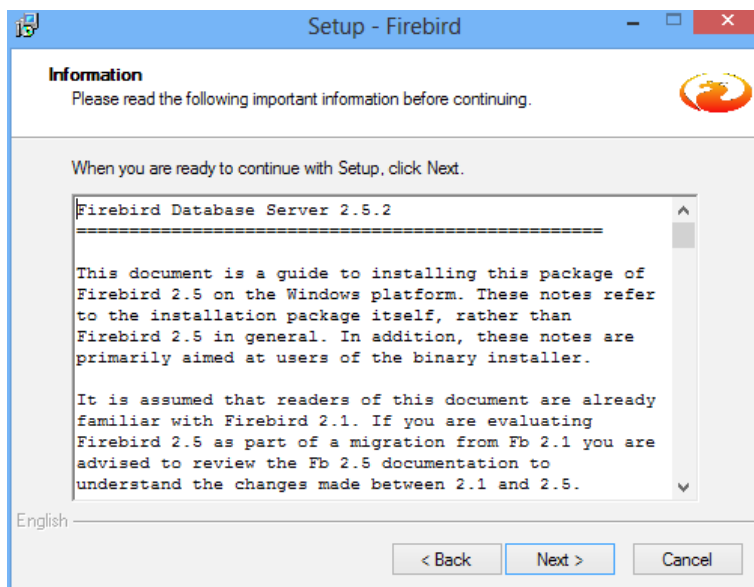


Choose the path where Firebird will be installed. The default path is C:\Program Files\Firebird\Firebird_2_5. Select <Next>.

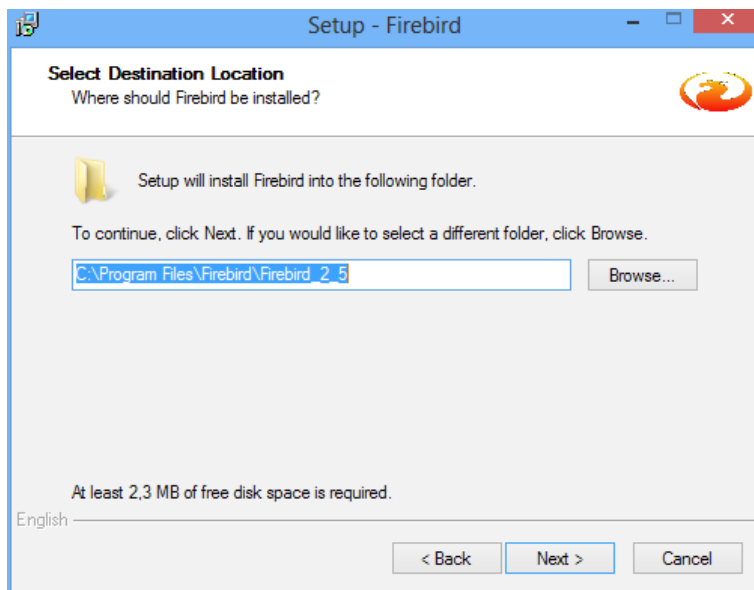


If the set-up asks which components should be installed, choose "Super Server binary". Select <Next>.

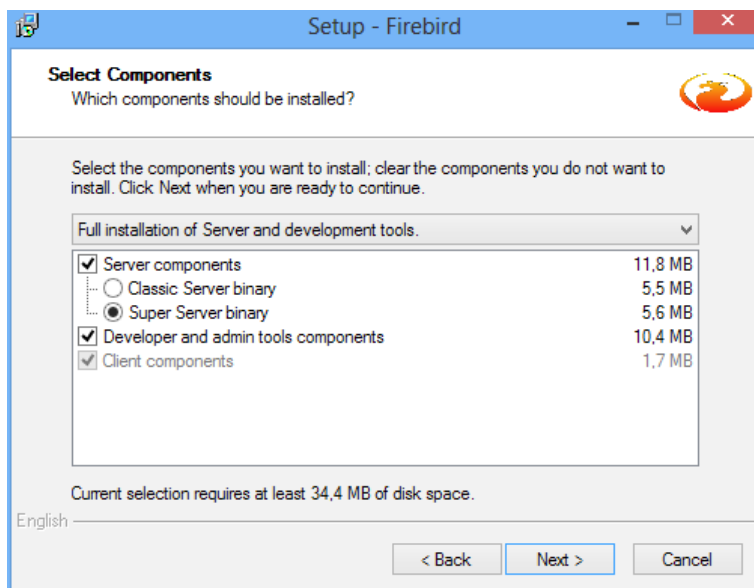


Firebird would also like to know if shortcuts are required. This is optional.

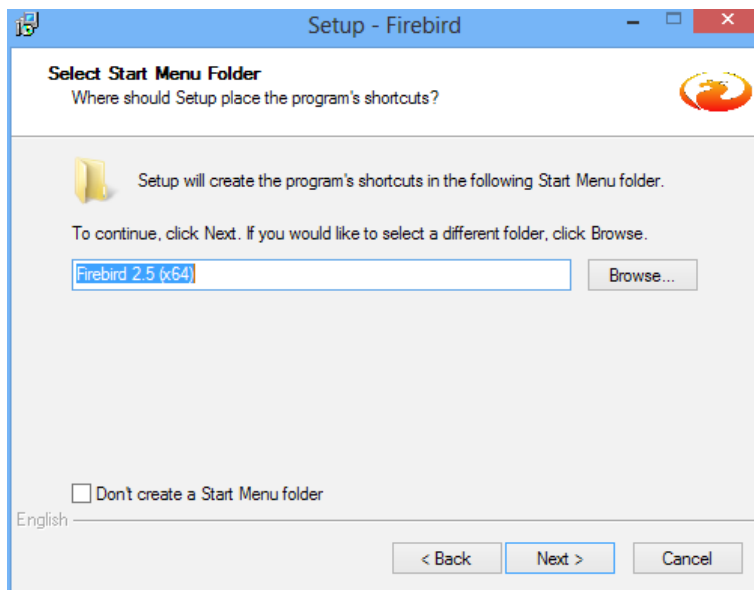


Additional tasks can also be chosen. It is important to set Firebird AS A SERVICE. Also check the settings like in the example:

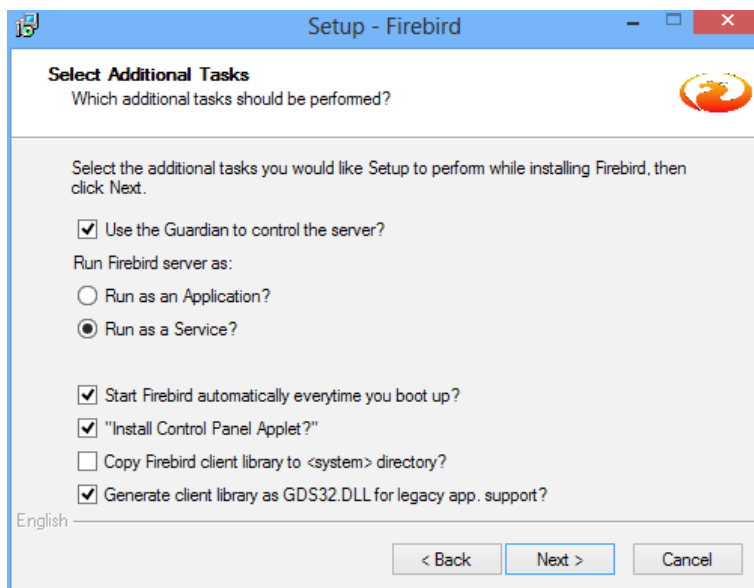

Now a summary will appear. Choose <Install> to start installing Firebird.

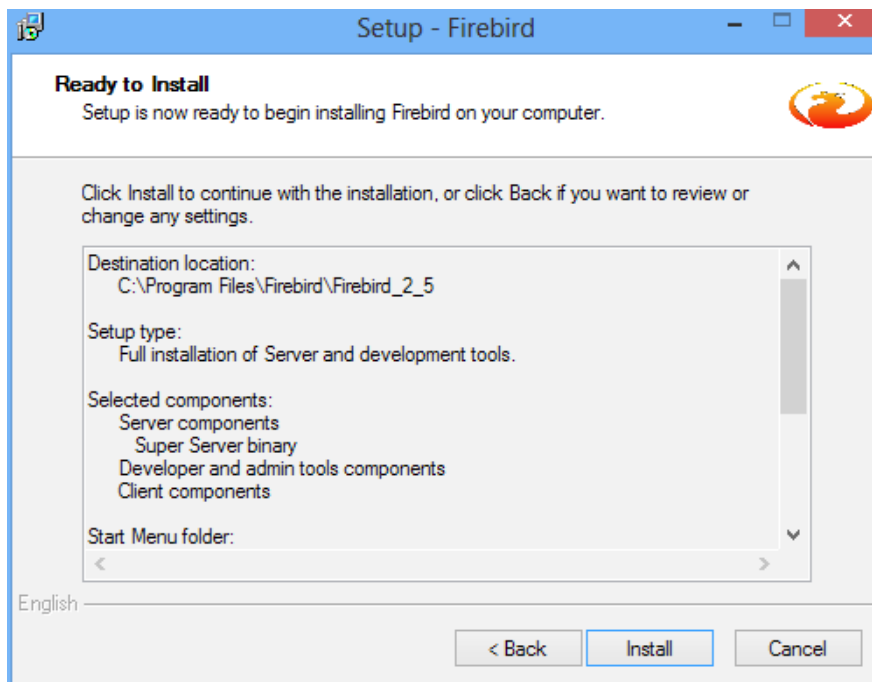


During the installation a progress bar is shown.

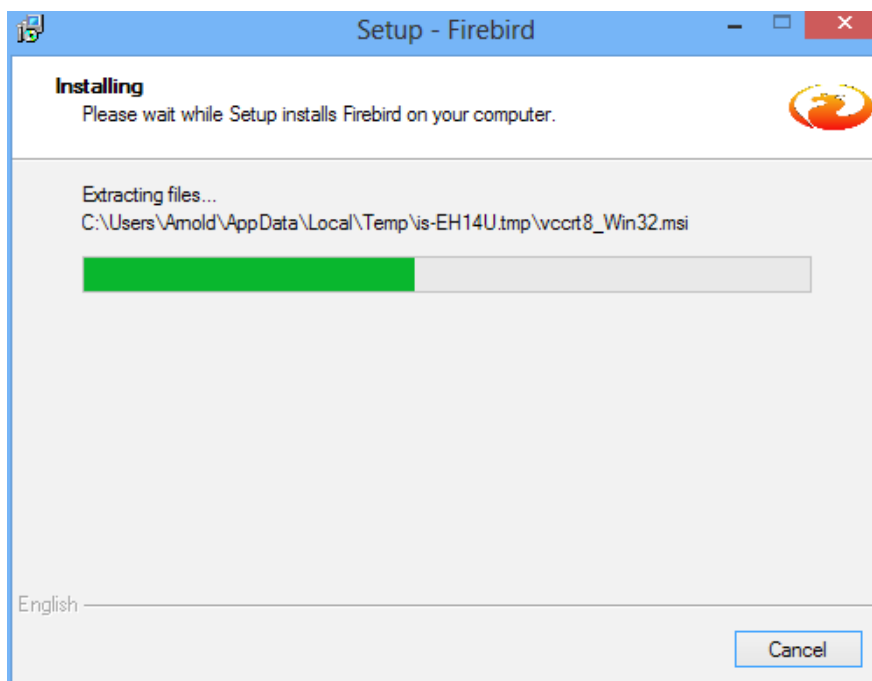

When the installation is finished, information about the application is shown. After reading the button <Next> can be pressed.




The installation is now almost ready. With the button <Finish> will the installation be completed and can the installation of the MR Cube be continued.

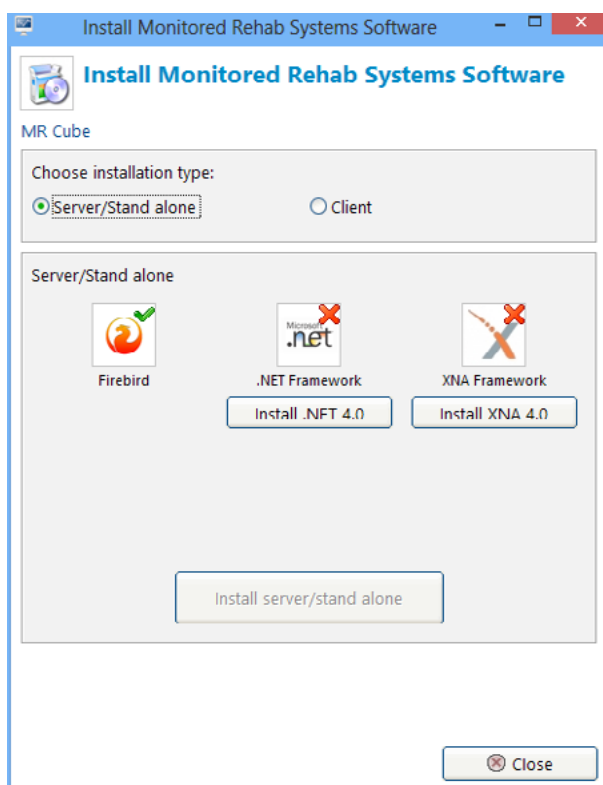


3.5.2 Install .NET Framework

When .NET Framework 4 is not yet installed, press <Install .NET 4.0>


Select the checkbox to confirm the legal pack was read and accepted.

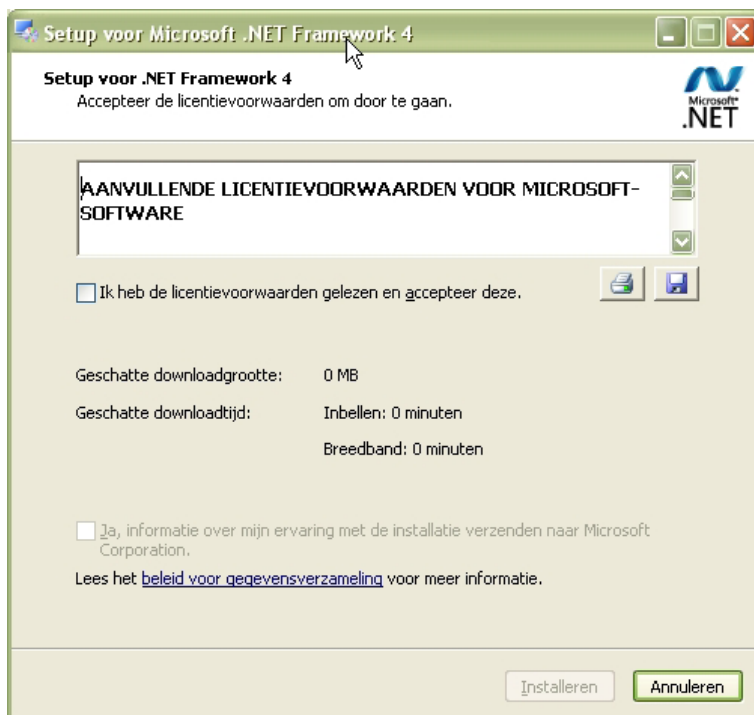


When accepted a new checkmark appears. Checking this box is optional.

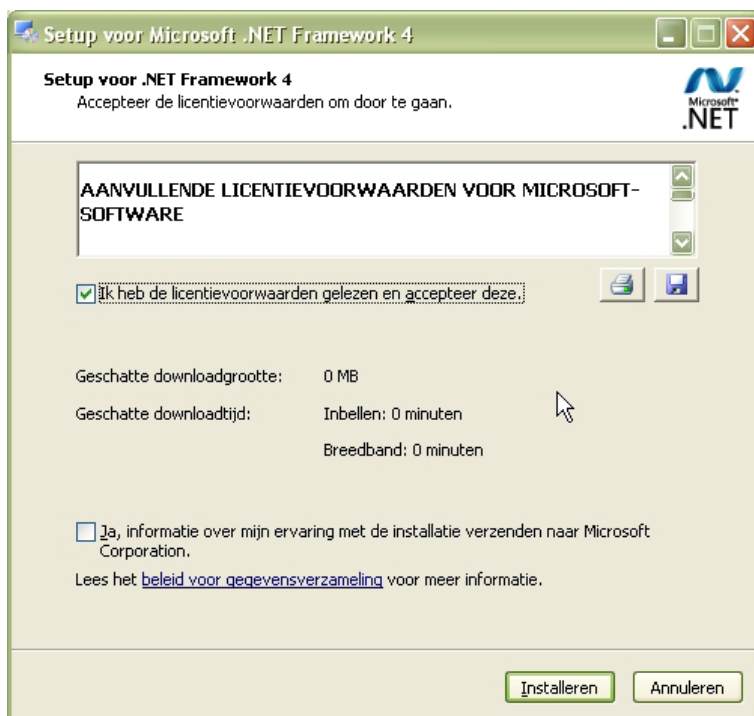

Now the installation will start. A progress bar is shown.

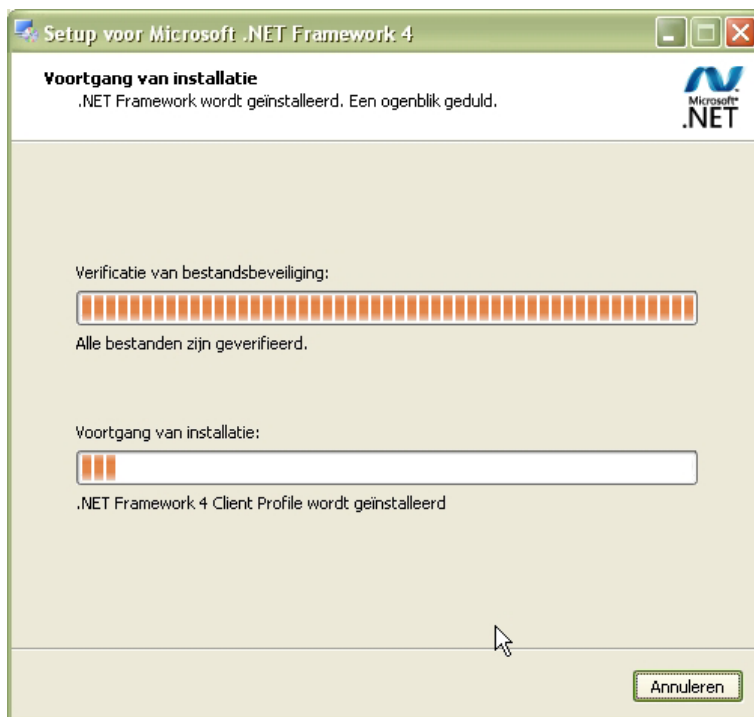


If the installation is ready, this screen appears:

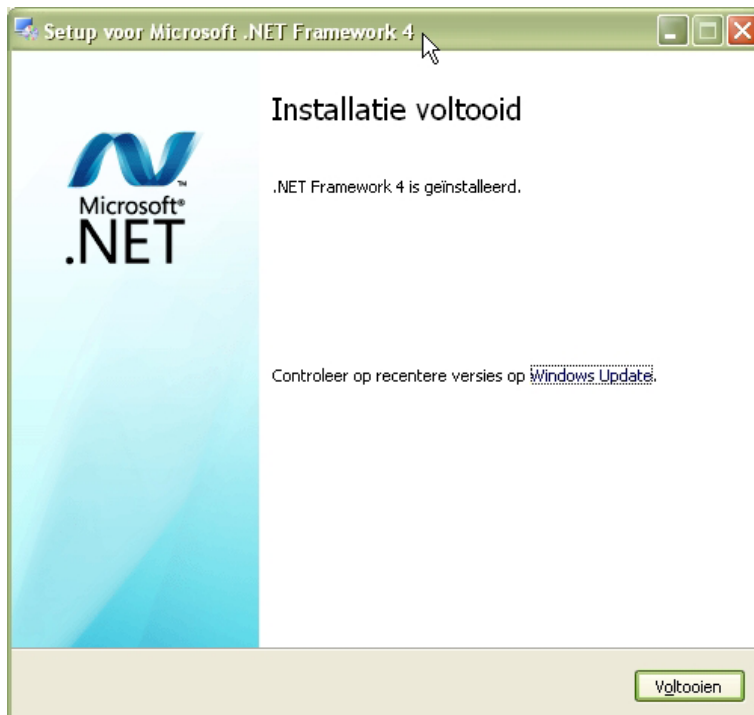

Press <Finish> / <Voltooien> to finish the installation.


If the MR Cube is being installed on a Server 2008 computer; the .NET Framework 3.5 components also have to be enabled.
Go to Start - Control Panel - Software/ Programs.
Choose "Add/Remove Windows Components"

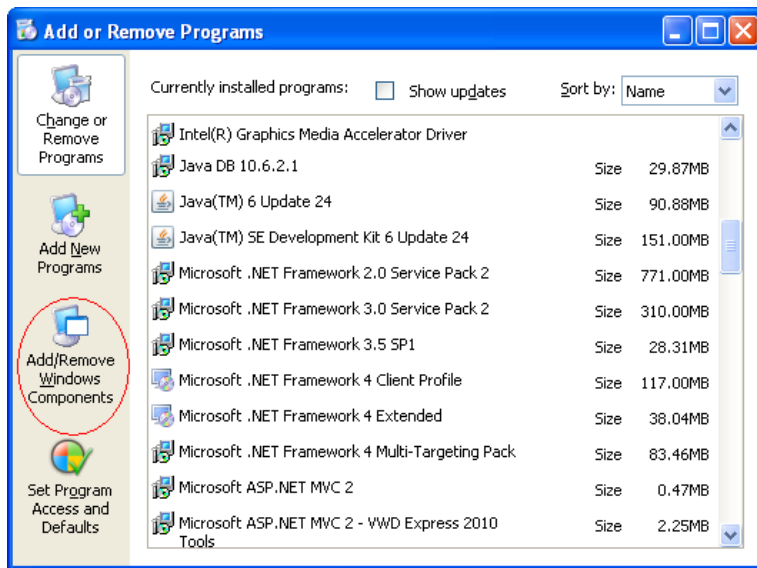

Select the .NET Framework 3.5 components and press <OK>




After installing everything correctly, this screen may appear:

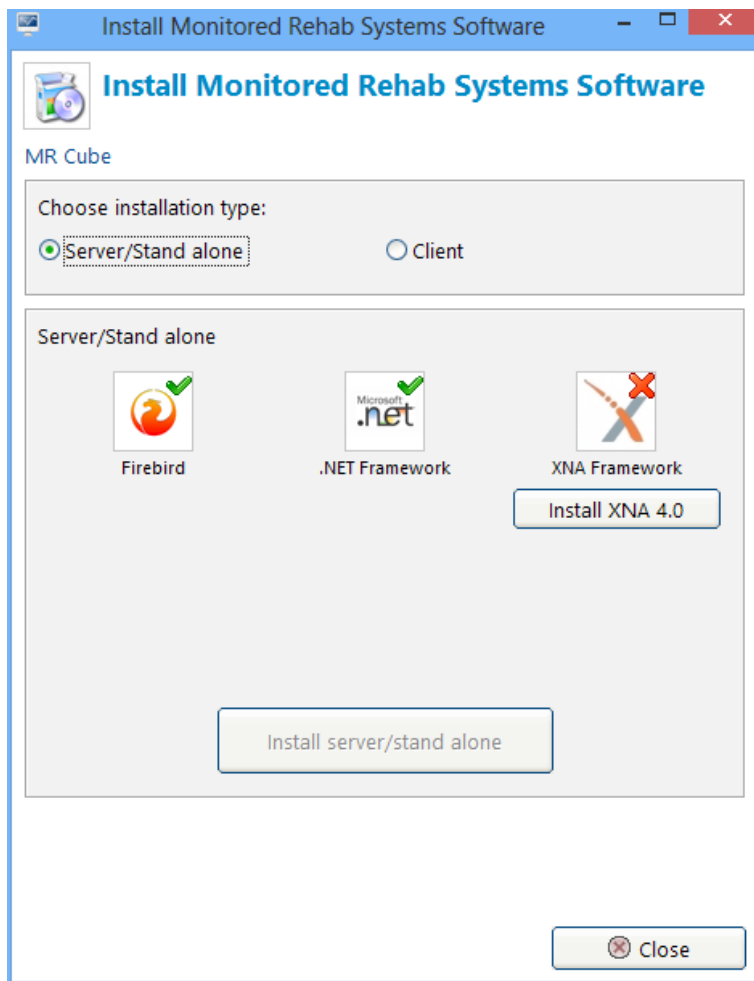


3.5.3 Install XNA Framework

If the button Install XNA version 4.0 appears, this means you don't have this software or correct version and you have to click this button.


The following screen will appear:

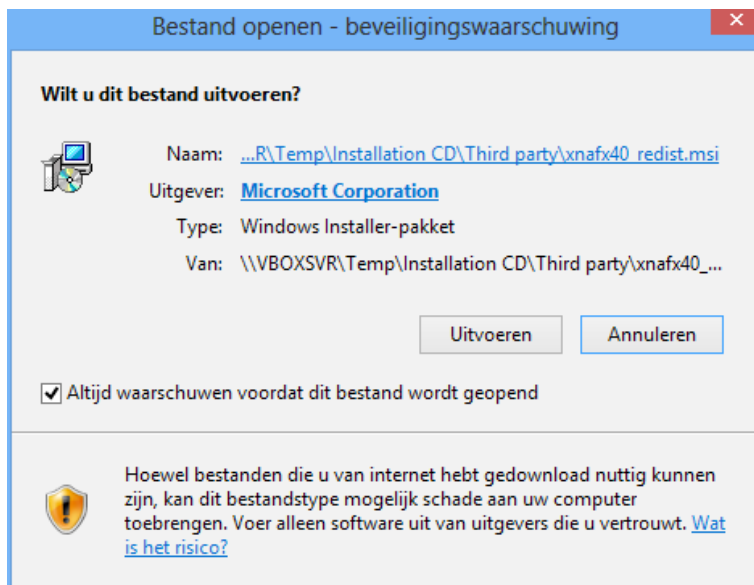


Choose <Run>/ <Uitvoeren>.

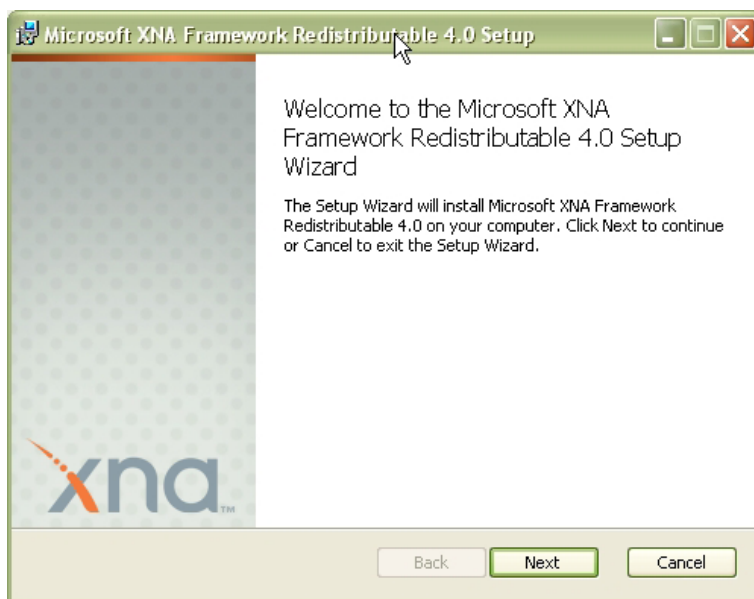


Press <Next> to continue.

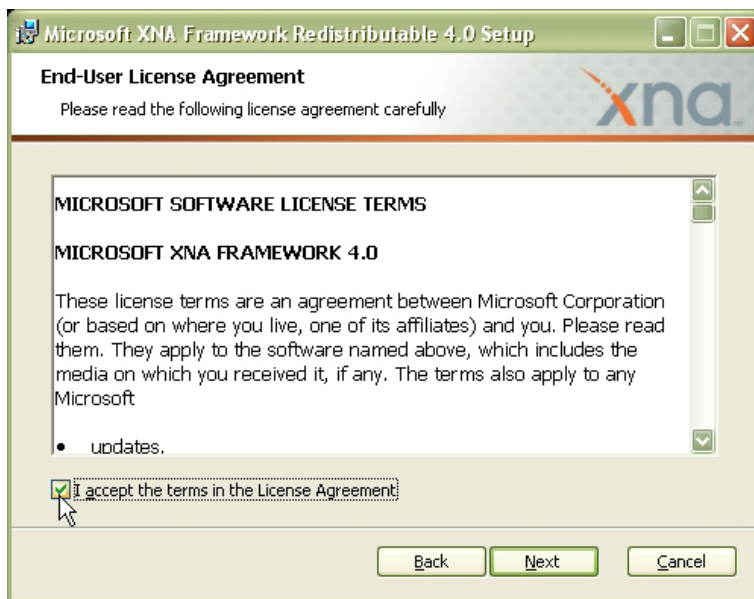


Check "I accept the terms in the License Agreement".
Press <Next> to continue.

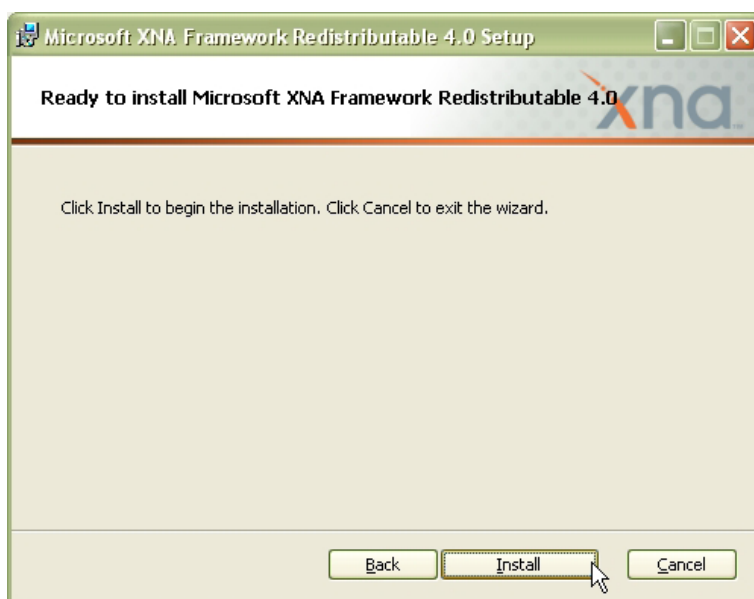

Press <Install>. This installation takes several minutes.

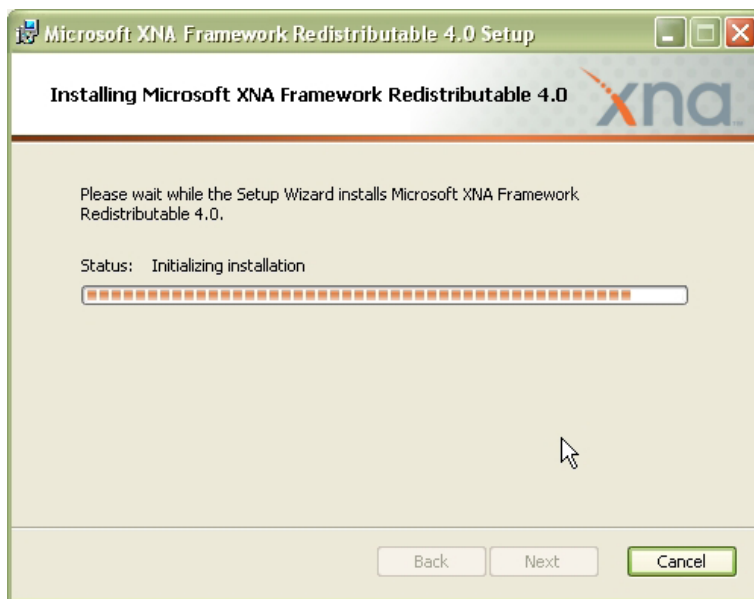


The following screen will appear when the installation has been successful:

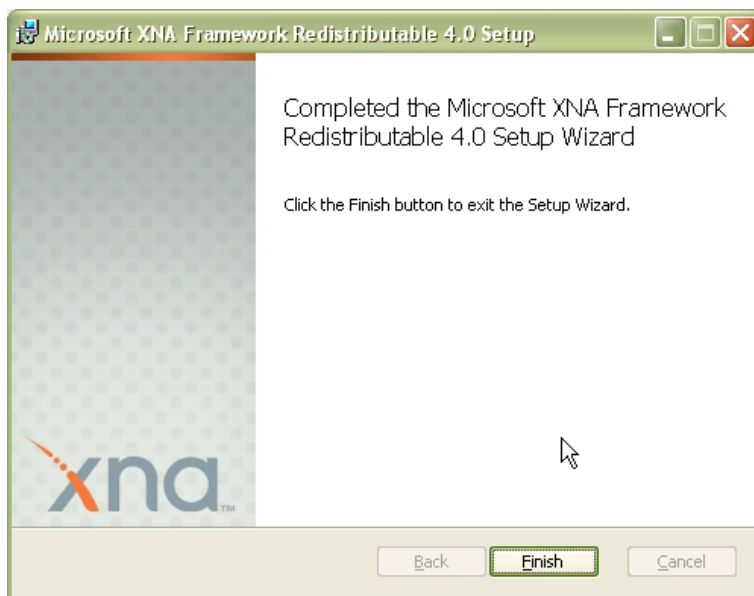

Press <Finish> to continue.


The installation screen now should be like this:

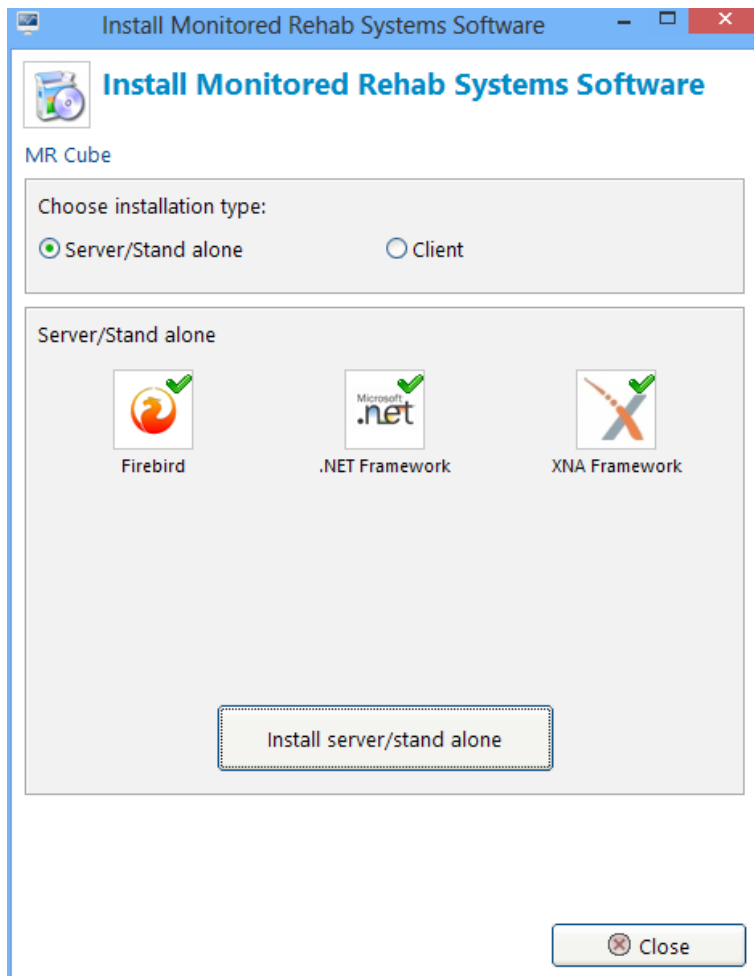

Press <Install server/ stand alone>.

3.6 Client Installation

If "Client" is chosen in the menu screen, Firebird is not required, just .NET Framework and XNA Framework.

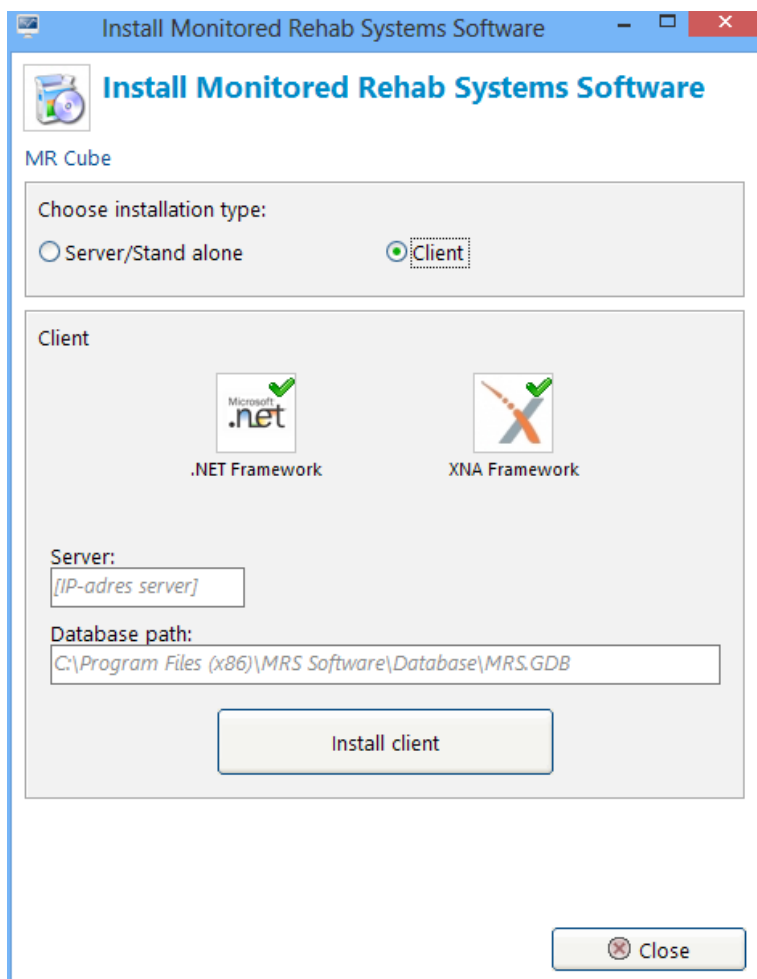

When the green checkmark shows, the right version is already installed on the computer. If there is a red cross, follow the installation steps from chapter 3.5.2 and 3.5.3.

All green?

Insert the IP address of the server:


Give the path to the database (viewed from the server)

Press <Install client>.


Choose the desired language.



Press <Next> or <Volgende>.

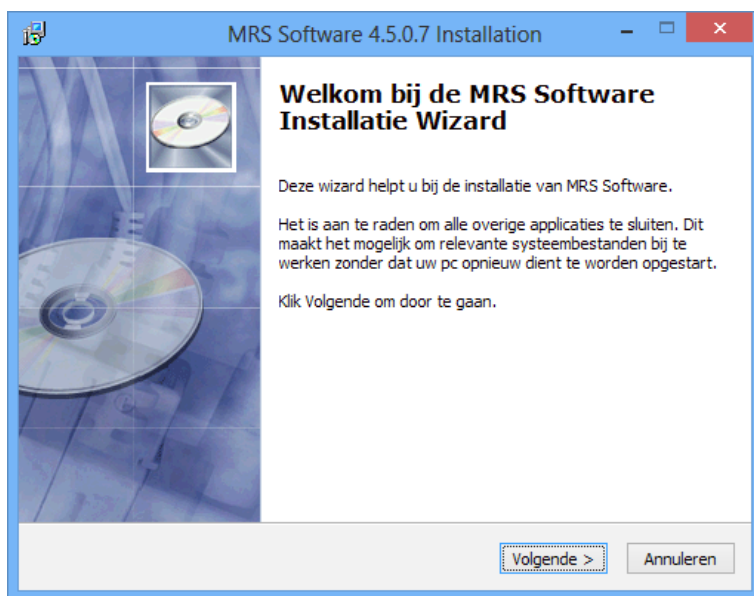

Choose the destination folder: Default is C:\Program Files (x86)\MRS Software.

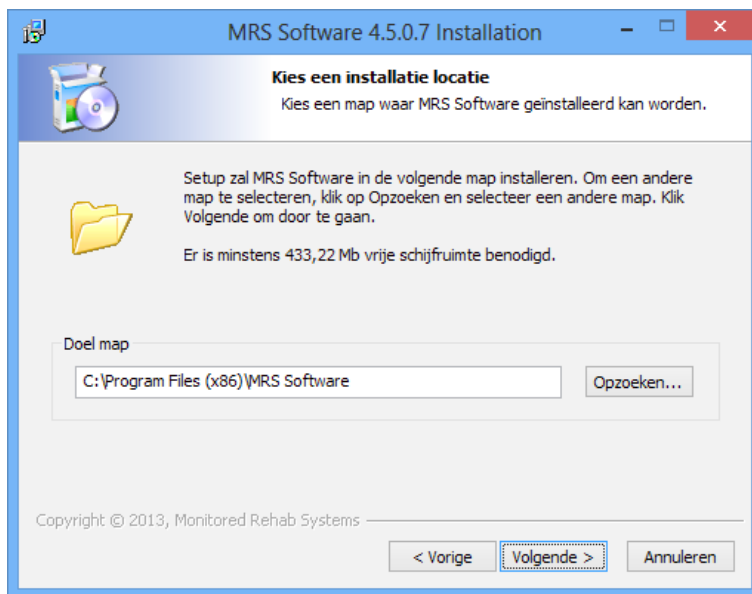


The software can have an icon on the desktop. Check the square to create this desktop icon.




A summary screen will appear:




During the installation a progress bar is shown.

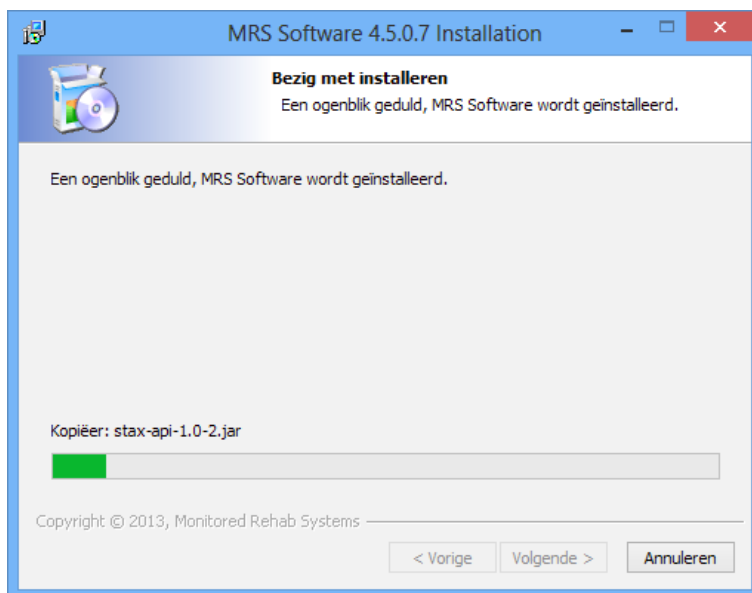

When the installation is almost finished, our website can be opened by selecting "Visit product website". Press <Finish> / <Einde> to finish the installation.

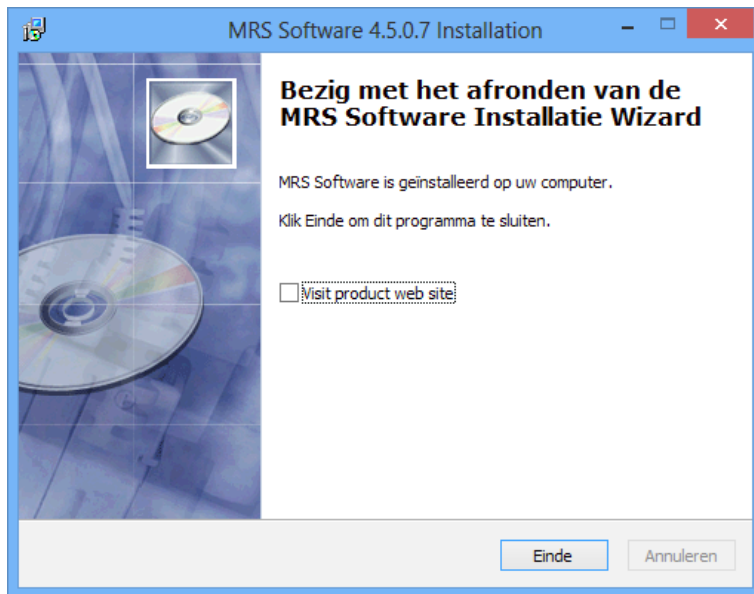

When totally finished, this screen appears:

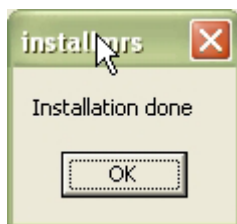


3.7 Uninstall


To safely remove the MRS software and avoid problems, follow the steps in this chapter:

Uninstall on the server computer:

Open the Control Panel via the <Start> button.
Locate the icon "Firebird Service Manager" and double click it.

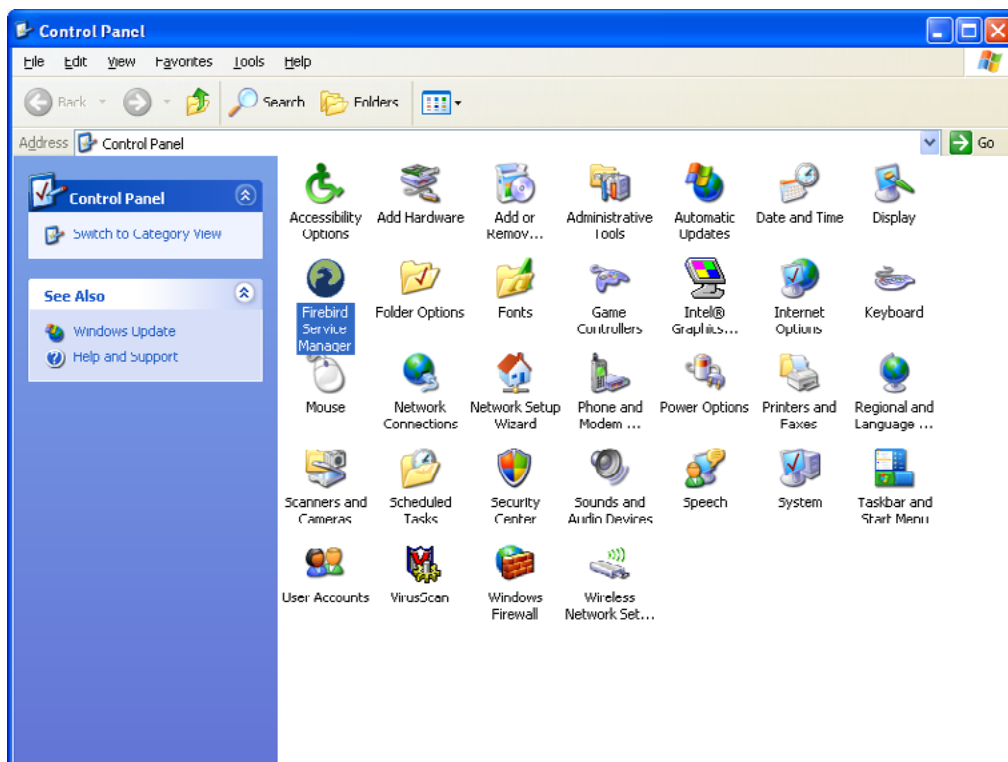

Firebird will open the following screen:

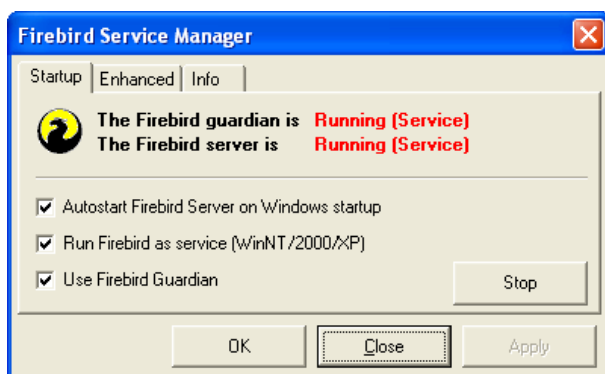


Press the <Stop> button.




The red text is changing into "Stopped". This means that Firebird is not active at this moment.
Press <Close>.

Uninstall on both server/ stand alone and clients:

Open the Control Panel via the <Start> button.

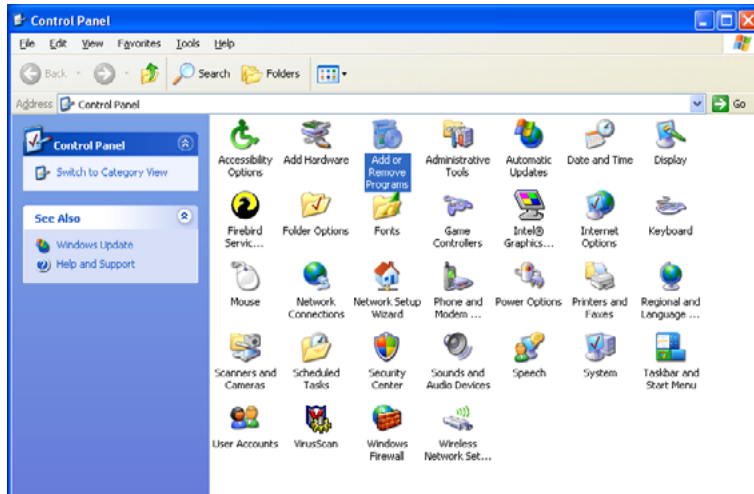

Locate the icon "Add or Remove Programs" in the control panel.
Double click on the icon.

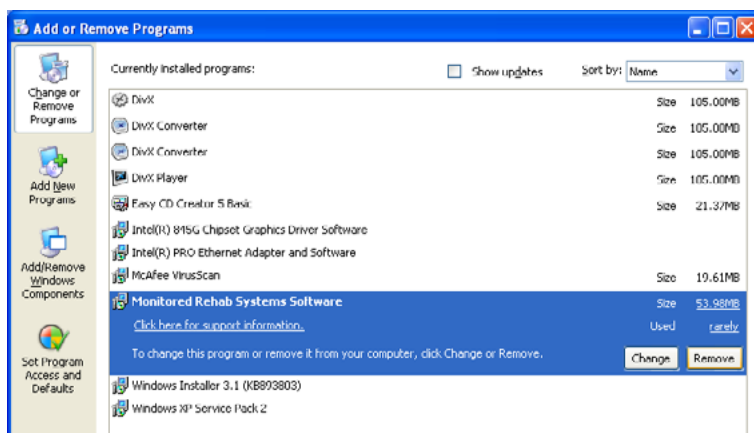

Select "Monitored Rehab Systems Software" and press the <Remove> button.
The software is now definitely removed.

4. Registration

If you start the MRS Software, you have to log in.

You can log in with:


Username: mradmin

Password: mradmin

We advise to add your own user(s) with their own competences (we have user, supervisor and administrator)


Before use, the MR Cube software needs to be registered. This registration is valid for one year and will be renewed each calendar year. After installation, the software is in demo mode which is valid for 30 days. During startup, the software shows how many days you have left to register.

At the first start, the following message will appear:

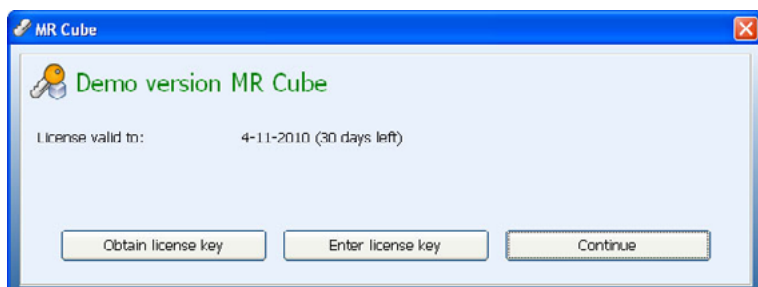

Select <Continue> if you don't want to register at this time.

Select <Obtain License Key> if you do want to register. Enter your details:

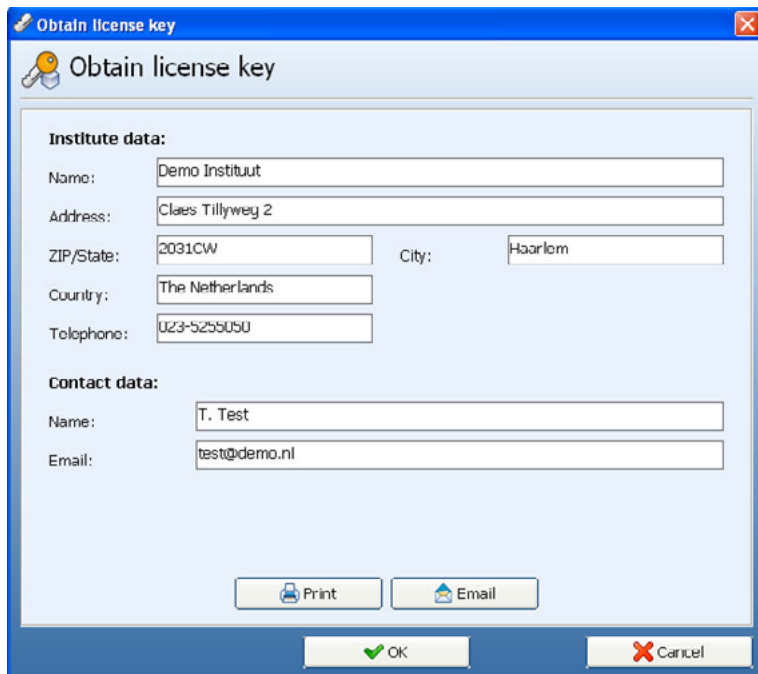

Select <Print> to print the registration form.

Select <Email> to e-mail the registration form, your local e-mail program must be opened for this action (webbased e-mail will not work).

Obtain license key


Send or fax this document to:
 Monitored Rehab Systems
 PO box 4959
 2003 EZ Haarlem
 +31 (0)23 750 54 44

Name:	Demo Instituut
Address:	Claes Tillyweg 2
ZIP/State:	2031CW
City:	Haarlem
Country:	The Netherlands
Telephone:	023-5255050

Name:	T. Test
Email:	test@demo.nl

Module:	MR Cube
Registration key:	11-21472


Print Preview. You can fax this to +31 (0) 23 750 5445 or save as PDF and e-mail as an attachment to info@mrsystems.nl

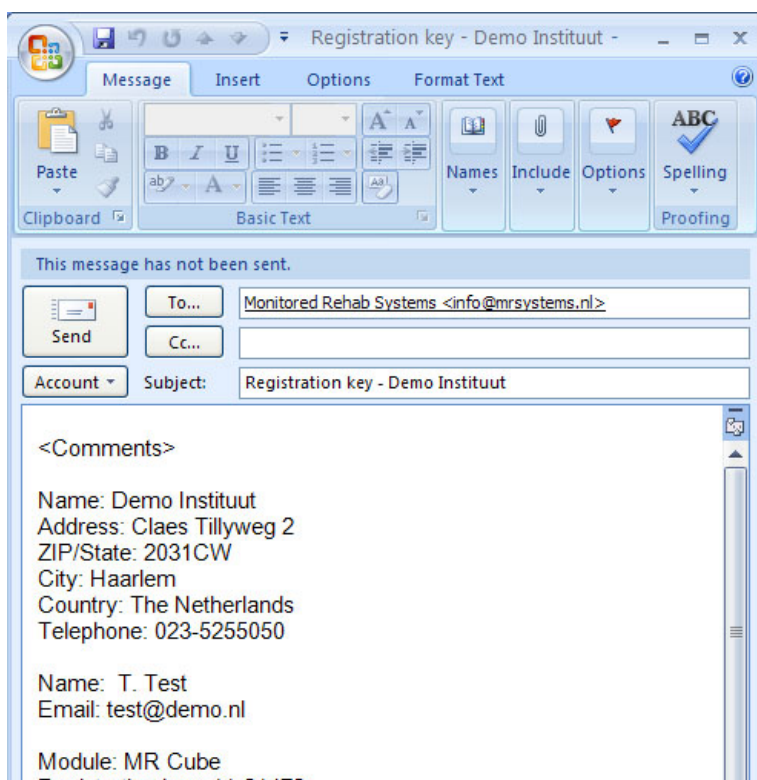


Example of an automatically generated e-mail.

After registration you will receive an e-mail from Monitored Rehab Systems (within 3 days), with a license key.


To enter the received key, start the software and choose <Enter license key>.

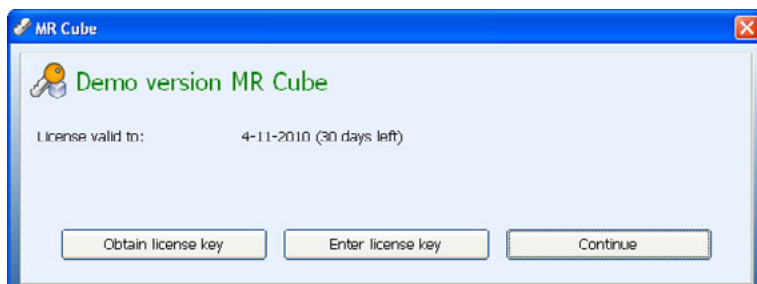

The code provided by Monitored Rehab Systems can be copied/pasted in the text field under "License Key" (do not type, it gives error messages because the difference between l, l, L, l O, 0 o are not clear enough).

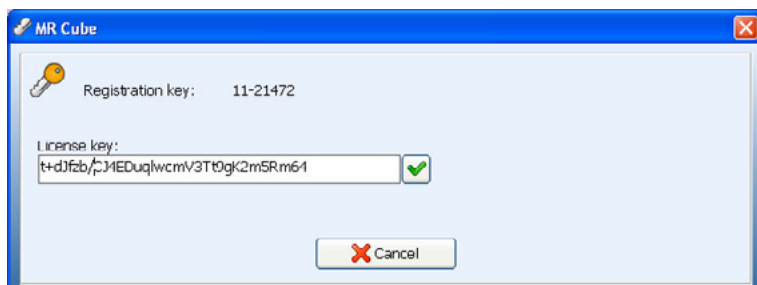

Then, select the green checkmark. Your software is now registered.

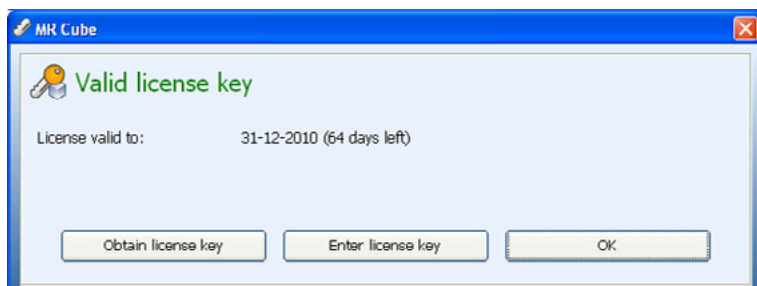


At the end of each calendar year you will receive (depending on your agreement with your local dealer) an invoice for next year's license. After payment of the invoice you will automatically receive a new license key for the following year.

If you do not register, a message "License expired" will appear frequently.

5 Setting the COM port

After pairing the COM port with the MR Cube (chapter 2), the COM port has to be set in the MRS Software. After installing and registering the software, go to tab Settings - Administrator settings - MR Cube- COM port. Save the settings. Now you can continue with chapter 6.

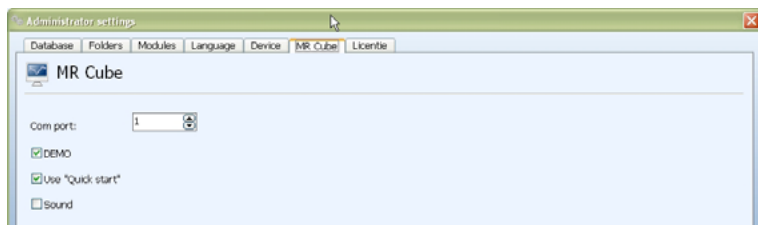


6. Patient

To insert a new patient select the option "New patient" in the menu.




Fill in the fields of the form with the patients data. The information in these fields will be linked with all performance results of the executed test and training programs. The yellow fields are required.

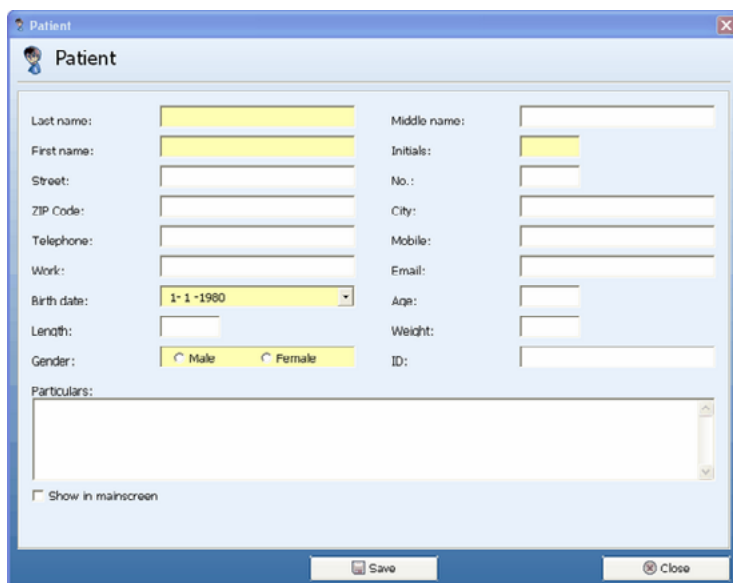

6.1 Indication

In the indication screen you can enter additional patient data like the diagnosis, involved side and more. This information is shown in the header on the printouts.

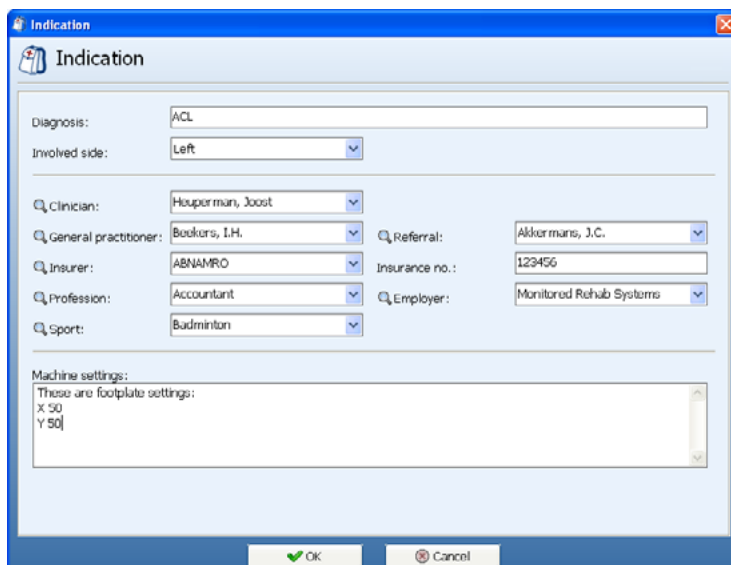

Indication

Diagnosis:


Involved side:

Machine settings:
These are footplate settings:
X 50
Y 50

OK Cancel

6.2 Extended history

Choose "Patient" - "Extended History". Here you get an complete overview of all test-, rehab and training programs with belonging scores from the selected patient. Every overview can be printed out. To view the results of one specific test double-click on a test.

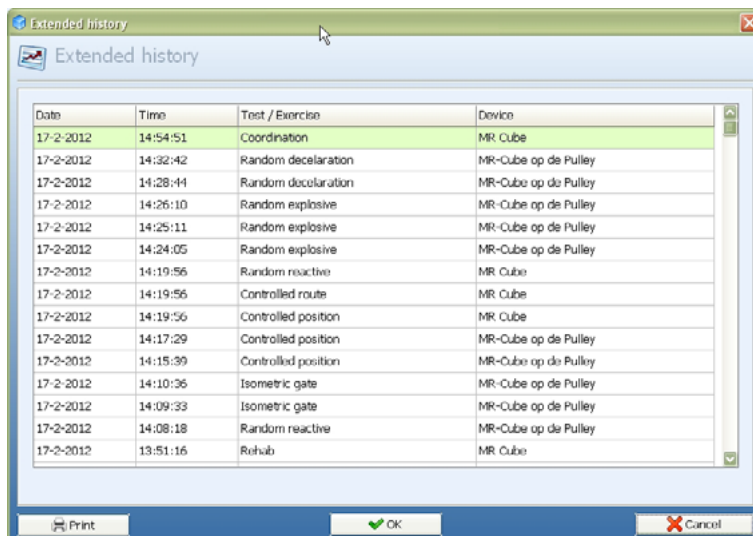

Extended history


Date	Time	Test / Exercise	Device
17-2-2012	14:54:51	Coordination	MR Cube
17-2-2012	14:52:42	Random deceleration	MR-Cube op de Pulley
17-2-2012	14:28:44	Random deceleration	MR-Cube op de Pulley
17-2-2012	14:26:10	Random explosive	MR-Cube op de Pulley
17-2-2012	14:25:11	Random explosive	MR-Cube op de Pulley
17-2-2012	14:24:05	Random explosive	MR-Cube op de Pulley
17-2-2012	14:19:56	Random reactive	MR Cube
17-2-2012	14:19:56	Controlled route	MR Cube
17-2-2012	14:19:56	Controlled position	MR Cube
17-2-2012	14:17:29	Controlled position	MR-Cube op de Pulley
17-2-2012	14:15:39	Controlled position	MR-Cube op de Pulley
17-2-2012	14:10:36	Isometric gate	MR-Cube op de Pulley
17-2-2012	14:09:33	Isometric gate	MR-Cube op de Pulley
17-2-2012	14:08:18	Random reactive	MR-Cube op de Pulley
17-2-2012	13:51:16	Rehab	MR Cube

Print OK Cancel

Report Preview			
File Page Zoom			
Page 1 of 6 Zoom 100.0 %			
Extended history			
Name: Test AaMonitored		Clinician: A.P. Ergotherapeut (e)	
Birthdate: 29-5-1976		Referral: Linderhofius	
Print date: 10-6-2010		Involved side: Both	
Diagnosis: asdfasdf			
Date	Time	Test	Device
10-6-2010	11:41:09	IsometricGate	DEMO
10-6-2010	11:39:53	SkiGame	DEMO
8-6-2010	9:19:17	SkiGame	DEMO
8-6-2010	9:17:46	SkiGame	DEMO
8-6-2010	9:15:15	SkiGame	DEMO
8-6-2010	9:13:11	SkiGame	DEMO
8-6-2010	8:44:05	Power test	DEMO
8-6-2010	7:45:57	Power test	DEMO
8-6-2010	7:32:12	Power test	DEMO
7-6-2010	11:45:25	RandomReactive	Unknown device
4-6-2010	11:19:26	Memory	DEMO
4-6-2010	11:19:20	RecognitionGame	DEMO
4-6-2010	11:19:13	Memory	DEMO

7. Quick Start menu

When the Quick Start in the Settings menu is checked, the following screen will appear in the tab of the MR Cube.

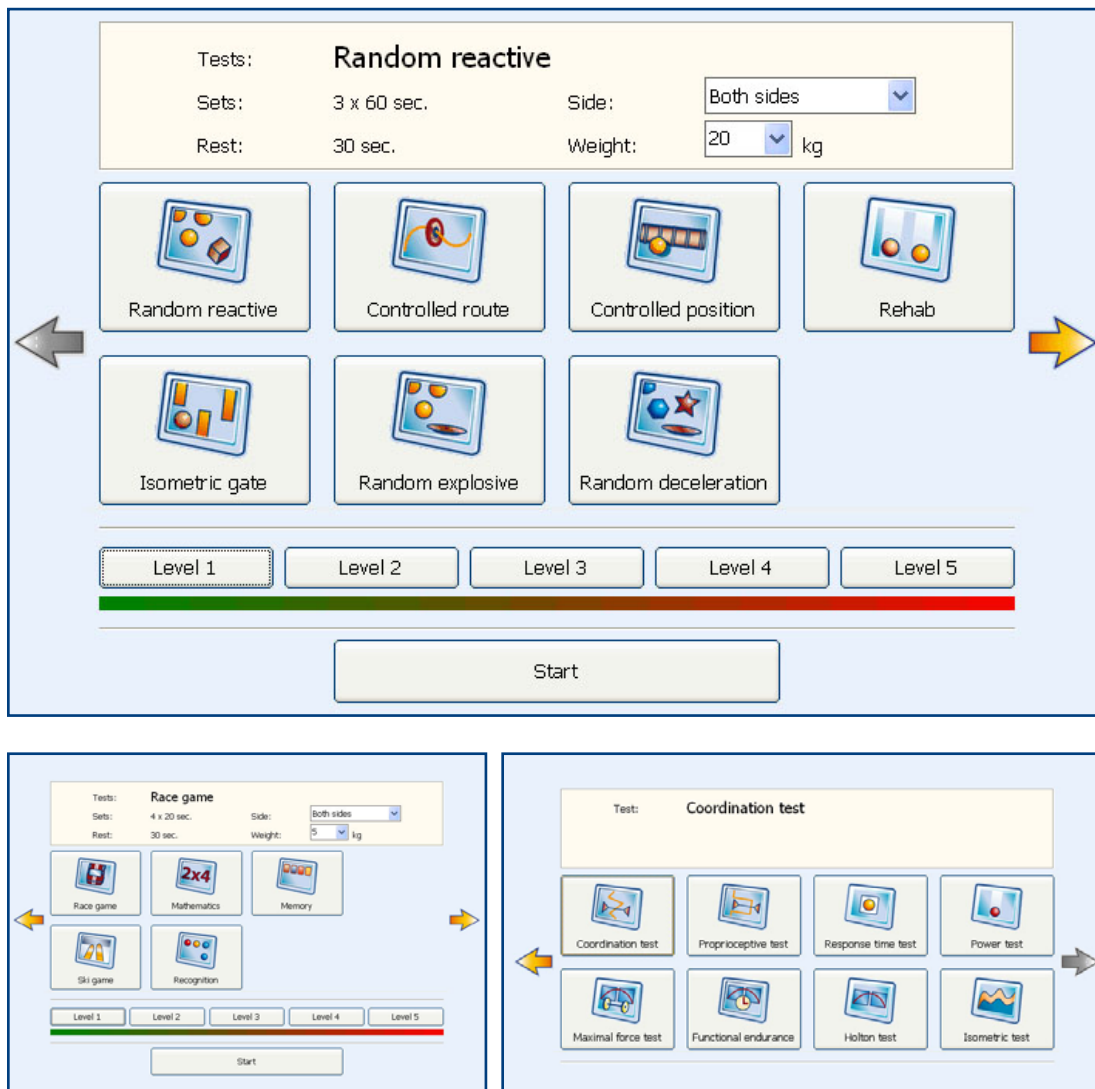

- 1) Choose exercise/challenging program/test by clicking on the yellow arrow and select button;
- 2) Choose side;
- 3) Choose weight;
- 4) Choose level;
- 5) Press start.


The exercise will continue starting the same way as in the advanced mode. Explanation about these exercises is described in the following chapters.

8. Rehab F5

In the main menu "MR Cube" the first option for training is Rehab (F5).



8.1 Rehab

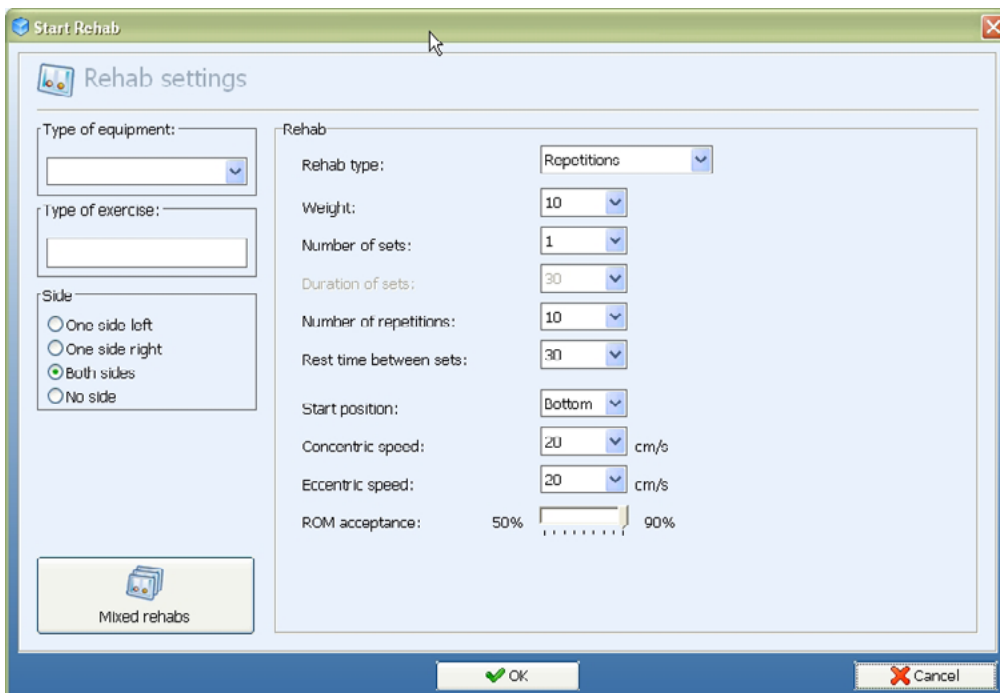

Start Rehab

Rehab settings

Type of equipment:

Type of exercise:

Side:

- ☐ One side left
- ☐ One side right
- ☒ Both sides
- ☐ No side

Mixed rehabs

Rehab

Rehab type:

Weight: 10

Number of sets: 1

Duration of sets: 30

Number of repetitions: 10

Rest time between sets: 30

Start position: Bottom

Concentric speed: 20 cm/s

Eccentric speed: 20 cm/s

ROM acceptance: 50% 90%

OK Cancel


This training program ends after a number of repetitions or after a pre-set period of time.

How to set up a rehab training step by step


- 1) Select "type of equipment" where the computer is connected to
- 2) Describe the type of exercise in "type of exercise"
- 3) Select One side left/right, Both sides or No side (no use of legs) training
- 4) Select rehab type (time or number of reps)
- 5) Select the exercise weight
- 6) Select number of exercise sets
- 7) Select the number of repetitions
- 8) Select the rest time between sets
- 9) Select start position. Bottom starts with concentric movement, top starts with eccentric movement
- 10) Select the concentric speed
- 11) Select the eccentric speed
- 12) Choose ROM acceptance. 50% means that 50% of the ROM should be done before this repetition is count as a repetition

- 13) This training program ends after a number of repetitions or after a pre-set period of time
- 14) To start the training press <OK> and press any key.


First set the range of motion by making 1 repetition:

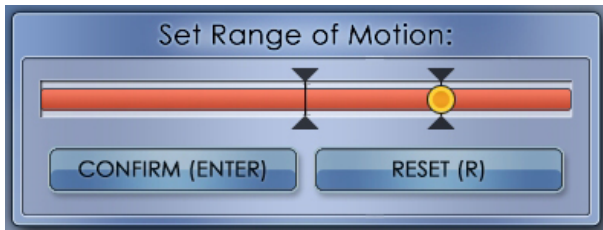

After pressing <Enter> the exercise will start.

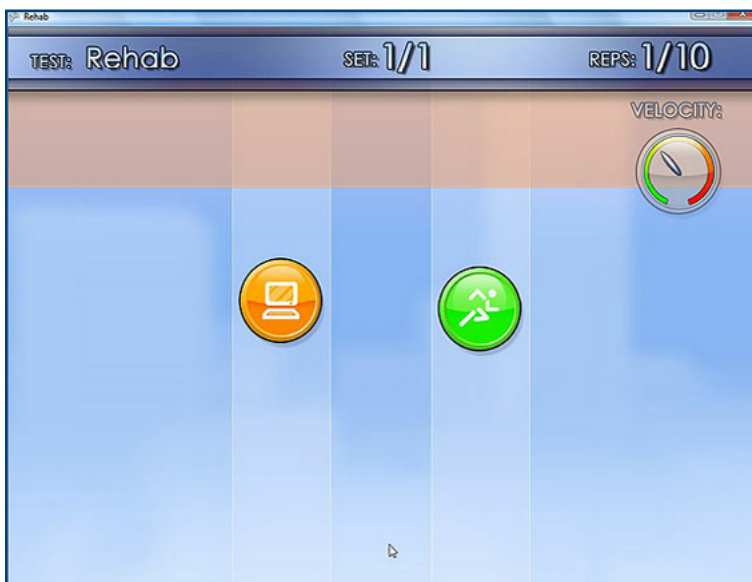

Displayed is the amount of repetitions (10) and the amount of sets (1).
 The "Preferred" speed (orange) is set in the software for both concentric and eccentric.
 The "Actual" speed (green) is the actual speed of the client/patient.
 Task of the patient is to get the green ball (right side) moving with the same speed as the orange ball (left side).
 If the training is finished the following screen will appear:

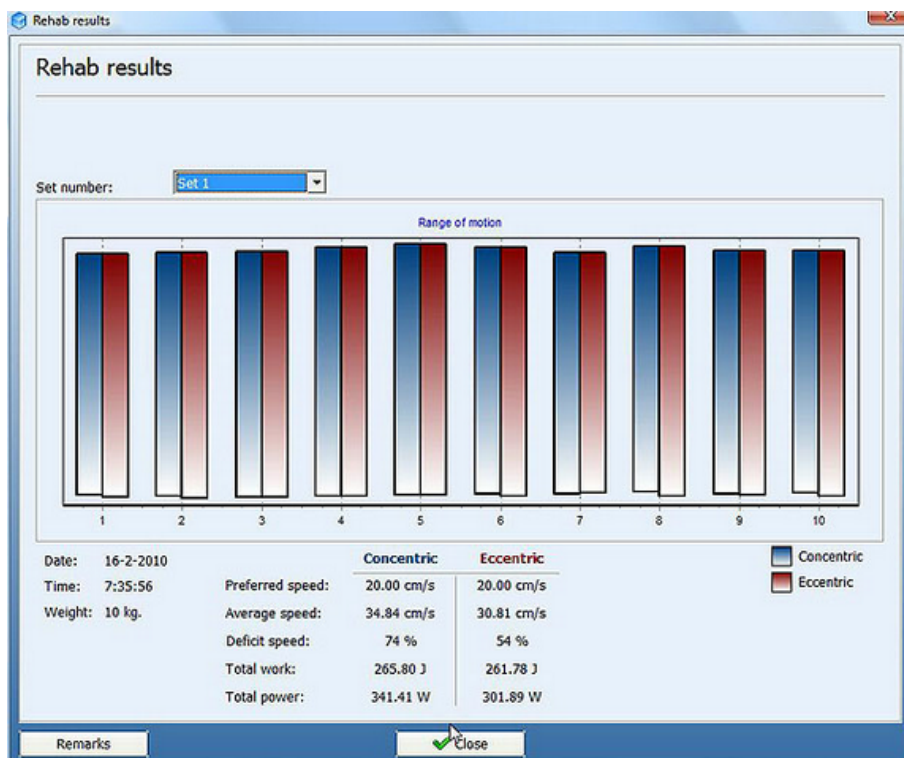

The graphics show the range of motion. Blue is the concentric part of the repetition, red is the eccentric part.

"Preferred speed" is the speed that has been set by the settings.

"Average speed" is the average speed of the exercise.

"Deficit speed" is the difference between preferred and average speed.

"-" : is too slow

"+" : is too fast

"0%" : is perfect

"Total work" is the total work performed by the patient during the exercise in Joules.

"Total power" is the total work performed by the patient during the exercise in Watt.

To add remarks, press the button <Remarks> and enter your text.



Remark

Remark about this training.

...

OK Cancel

8.2 Mixed Rehab

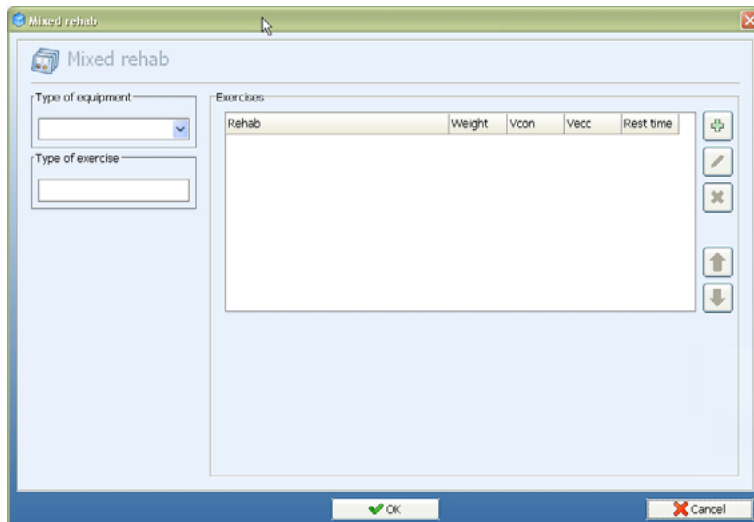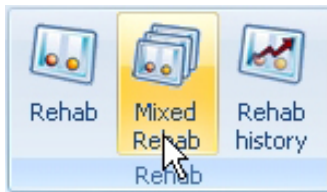


With Mixed rehab it is possible to plan several different exercises, with different settings, for example:

3x10 repetitions 10cm/s up and 15cm/s down (10kg)
 2x5 repetitions 20cm/s up and 20 cm/s down (20kg)
 1x30 sec. (Time) 15cm/s up and 20cm/s down (30kg)

You can use these settings:

- Rehab type: repetition or Time
- Weight
- Number of sets
- Duration of sets
- Number of repetitions (not with "Time")
- Rest time between sets
- Start position
- Concentric speed
- Eccentric speed
- ROM acceptance in %
- Side: left, right, both or none

By pressing the + icon the following screen appears:

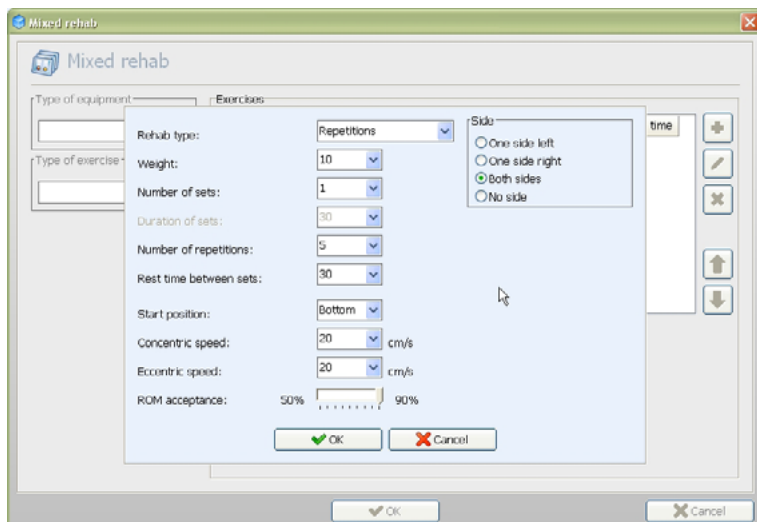

By pressing the pencil, the exercises can be changed.

By pressing <OK> the exercises will start.

Everything else works the same as the Rehab exercises.

8.3 Rehab History



In this menu you can see all sets done in the rehab training. Double click on the rehab to see the results.

Rehab history

Rehab history

Date	Time	Rehab	Device	Device
4-7-2012	10:19:52	Rehab (4 x 5 Repetitions)	Both sides	DEMO
4-7-2012	10:19:51	Rehab (1 x 5 Repetitions)	Both sides	DEMO

Rehab: 4 x 5 repetitions
 Weight: 13
 Number of sets: 4
 Number of repetitions: 5 repetitions
 Side: Both sides
 Concentric speed: 20 cm/s
 Eccentric speed: 20 cm/s

Print

OK

Cancel

To print to paper use the "Print" button.

Rehab history

Name: Joost de Koningscobra
 Birthdate: 21-3-1974
 Print date: 30-6-2010


Clinician: Joost Heuperman
 Referral: Akkermans
 Involved side: Left

Date	Time	Rehab	Device	Device
30-6-2010	15:01:16	Rehab (1 x 10 Repetitions)	Both sides	DEMO
16-6-2010	15:46:02	Rehab (1 x 2 Repetitions)	Both sides	DEMO
17-6-2010	15:25:41	Rehab (1 x 2 Repetitions)	Both sides	DEMO

9. Exercises



9.1 Functional exercises

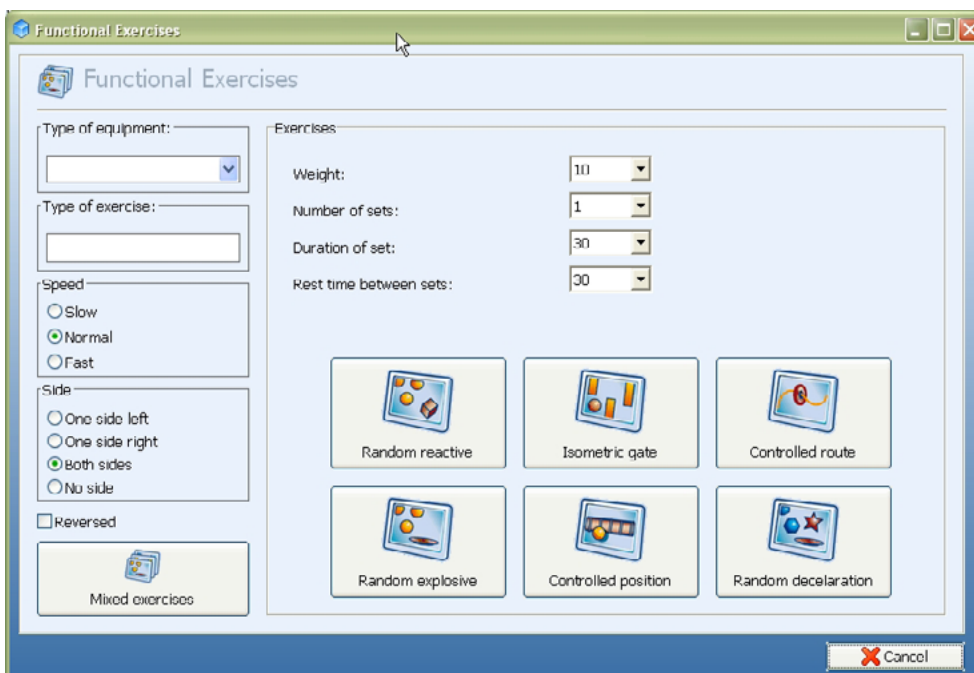

How to set up a Functional Exercise training step by step:

- 1) Select "type of equipment" (optional)
- 2) Choose the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right or both sides leg training
- 5) Select the exercise weight
- 6) Select number of sets
- 7) Select the rest time between sets


First set the range of motion by making 1 repetition:



After pressing <Enter> the exercise will start.


Random Reactive

In this exercise yellow balls will fall down from the screen "at random". The red square can be moved by the patient. Making concentric or eccentric movements will shift the red square to the right (concentric) or to the left (eccentric). The goal is to avoid the yellow balls.

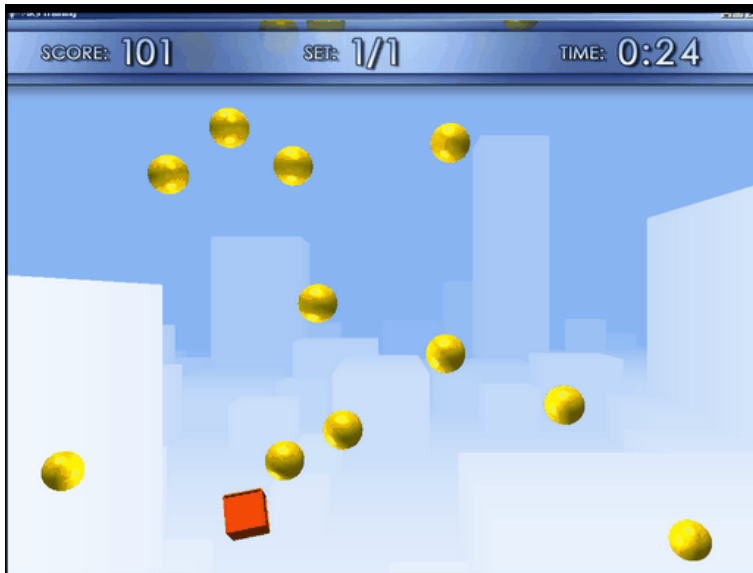

Isometric Gate

The isometric training has concentric, eccentric and isometric moments. The red ball must fit in the isometric (hold) position. The required force is challenging both coordination and proprioceptive skills. The red ball has to find its way by maneuvering it between the gaps in the bars. The spaces between the bars are random. If the exercise is carried out well, the difficulty level will increase. The red ball will move higher, giving the patient less time to spot the gates. As with the other exercises the concentric and eccentric movements will shift the red ball to the right (concentric) or to the left (eccentric).

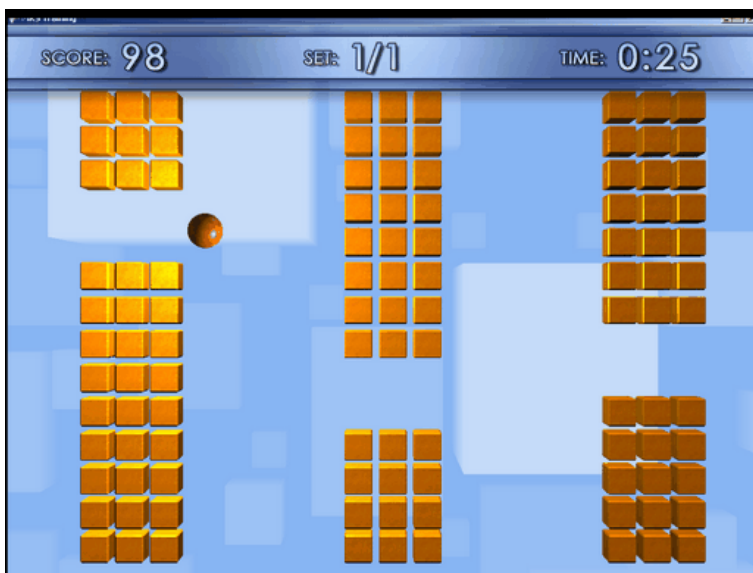

Controlled Route

In this exercise the red circle has to stay on the yellow line (path). The yellow line moves over the screen at random. Concentric/eccentric movements will shift the circle to the right or left. This exercise activates the eccentric/concentric muscle control and invigorates the coordinate abilities.




Random Explosive

In this exercise yellow balls will drop at random slow, normal or fast. These balls have to be caught and bounced back. Fast reaction (explosivity) is necessary to get good results. Concentric/eccentric movements will shift the red plate to the left and to the right.



Controlled position

This exercise copies "simulation". Target is to keep the red ball on the road. The "road" moves at random up and down with unexpected acceleration. Concentric/eccentric movements will shift the red ball to the left and to the right.

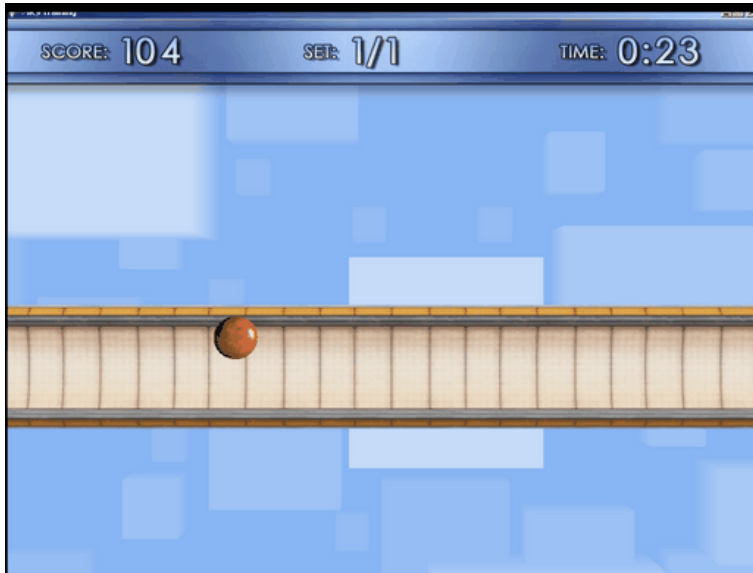

Random Deceleration

In this exercise little green stars with individual changing speeds will drop down the screen at random. The red plate symbolizes the catcher. Concentric/eccentric movements will shift the catcher to the left or to the right. The goal is to catch the green stars before they arrive at the bottom of the screen and avoid the unexpected red stars.



9.2 Functional exercises: the scores

The number of points at the start is based on the duration of the exercise.


The end score is calculated and influenced by:

- performance
- exercise weight
- speed level (slow, medium or high)
- if you perform a perfect exercise for 10 seconds, you get bonus points

The score is kept in the patient file and can be viewed in the exercise history.

9.3 Mixed Exercises

In this window you can create an individual combination of different exercises

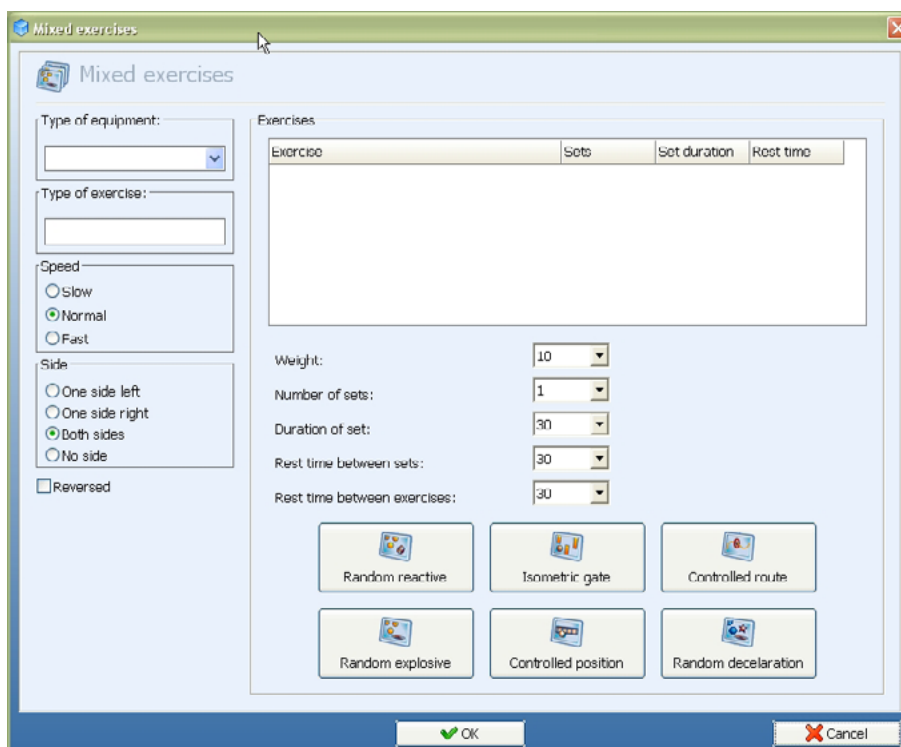


Option 1: how to create an individual mixed exercise training program

- 1) Select "type of equipment" where the MR Cube is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right or both sides leg training
- 5) Select the type of feedback 2D or 3D (preferred) and start position (reversed starts with eccentric movement)
- 6) Select the exercise weight
- 7) Select number of sets
- 8) Select the rest time between sets
- 9) Select the rest time between exercises
- 10) After starting, the range of motion has to be set by doing one repetition:




After pressing <Enter> the exercise will start automatically.
In between exercises, when there is rest time, this screen appears:

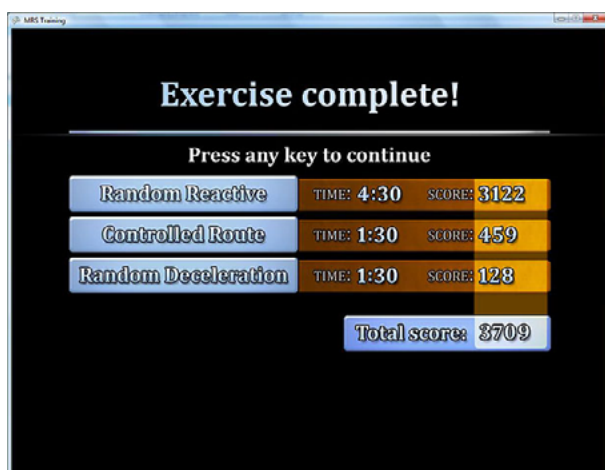


9.4 Exercise history


When a patient is selected, a concise list of all exercise programs with belonging scores will be shown. Scores and dates are shown in graphics and can be (de)selected.

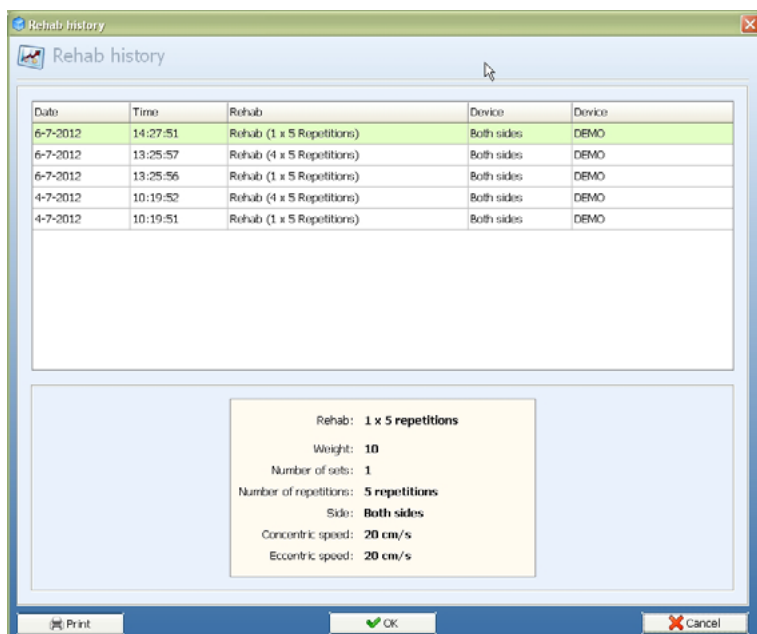


10. Challenging Programs

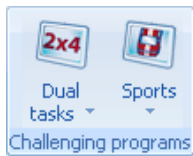

10.1 Dual Tasks

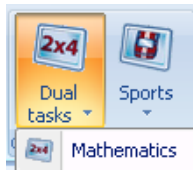

10.1.1 Mathematics

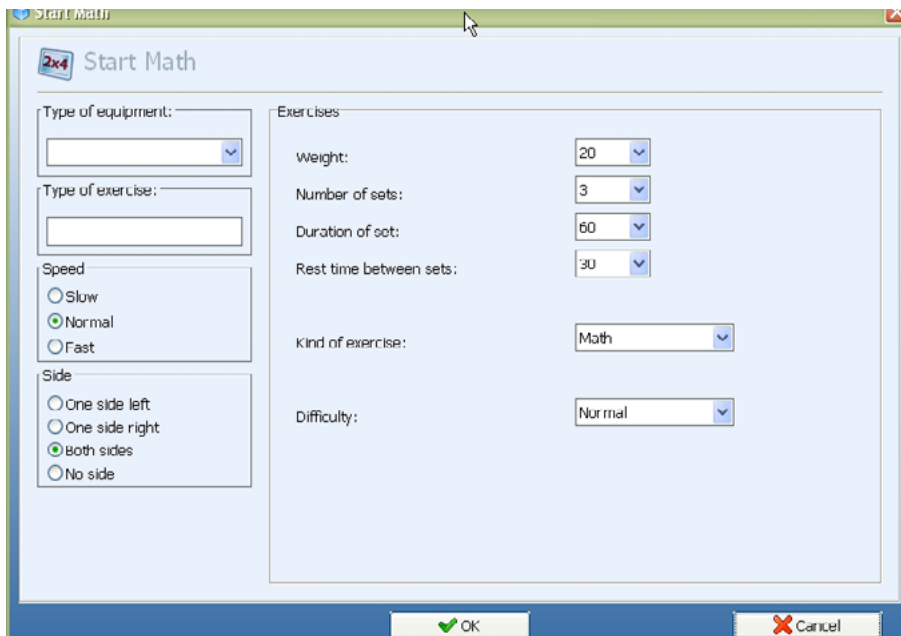

The 'Start Math' dialog box contains the following settings:

- Type of equipment: [Dropdown menu]
- Type of exercise: [Dropdown menu]
- Speed:
 - ☐ Slow
 - ☒ Normal
 - ☐ Fast
- Side:
 - ☐ One side left
 - ☐ One side right
 - ☒ Both sides
 - ☐ No side
- Exercises:
 - Weight: 20 [Dropdown menu]
 - Number of sets: 3 [Dropdown menu]
 - Duration of set: 60 [Dropdown menu]
 - Rest time between sets: 30 [Dropdown menu]
 - Kind of exercise: Math [Dropdown menu]
 - Difficulty: Normal [Dropdown menu]

Buttons: OK, Cancel

How to set up a Math exercise step by step:

- 1) Select "type of equipment" where the equipment is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right, both sides or "no side" training
- 5) Select the exercise weight
- 6) Select number of sets
- 7) Select duration of set
- 8) Select the rest time between sets
- 9) Select your exercise: Math
- 10) Select difficulty

After starting, the range of motion has to be set by doing one repetition:




After pressing <Enter> the exercise will start.

Choose the correct answer by moving the arrow.




If a correct answer is chosen, a green check mark will appear. After this, the next equation will appear.

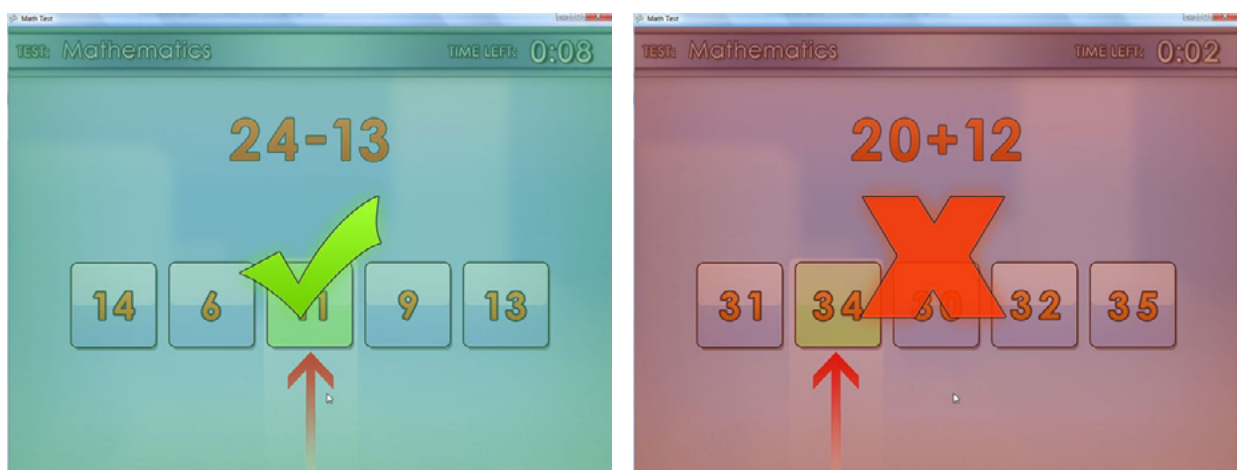


If a wrong answer is selected, a red cross will appear. After this, the next equation will appear.
After finishing the exercise the results will be shown:



The preset settings are blue. The correct answers are green.
Press any key and the following results are shown:

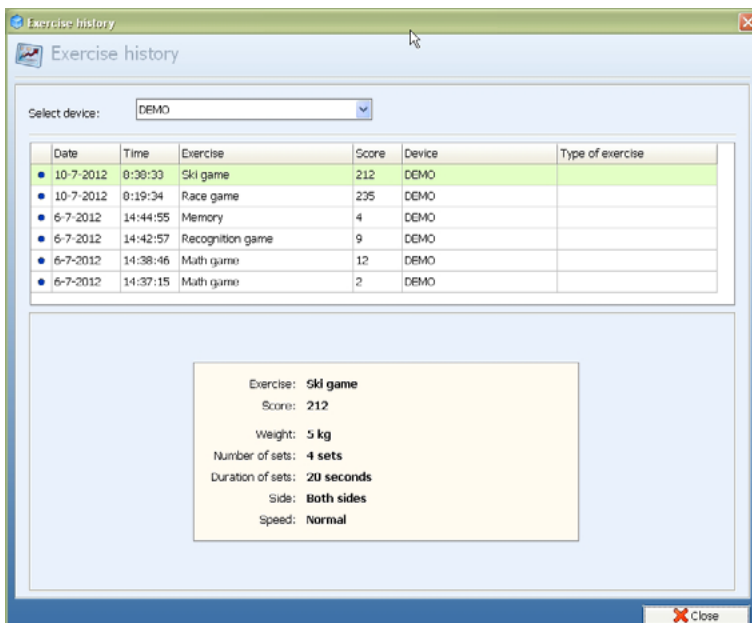

10.1.2 Recognition

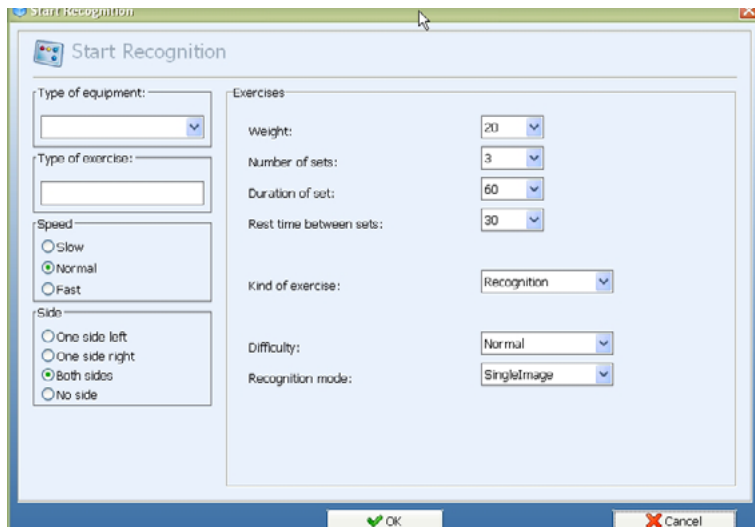

The 'Start Recognition' dialog box contains the following settings:

- Type of equipment: [Dropdown menu]
- Type of exercise: [Dropdown menu]
- Speed:
 - ☐ Slow
 - ☒ Normal
 - ☐ Fast
- Side:
 - ☐ One side left
 - ☐ One side right
 - ☒ Both sides
 - ☐ No side
- Exercises:
 - Weight: 20 [Dropdown menu]
 - Number of sets: 3 [Dropdown menu]
 - Duration of set: 60 [Dropdown menu]
 - Rest time between sets: 30 [Dropdown menu]
 - Kind of exercise: Recognition [Dropdown menu]
 - Difficulty: Normal [Dropdown menu]
 - Recognition mode: SingleImage [Dropdown menu]

Buttons: OK, Cancel

How to set up a Recognition exercise step by step:

- 1) Select "type of equipment" where the equipment is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right, both sides or "no side" training
- 5) Select the exercise weight
- 6) Select number of sets
- 7) Select duration of set
- 8) Select the rest time between sets
- 9) Select your exercise: Recognition
- 10) Select difficulty
- 11) After starting, the range of motion has to be set by doing one repetition:




The 'Set Range of Motion' dialog box features a horizontal slider with a yellow knob. Below the slider are two buttons: 'CONFIRM (ENTER)' and 'RESET (R)'.

After pressing <Enter> the exercise will start.



Choose the correct figure by moving the arrow.


If a correct figure is chosen, a green check mark will appear. After this, the next figure will appear.

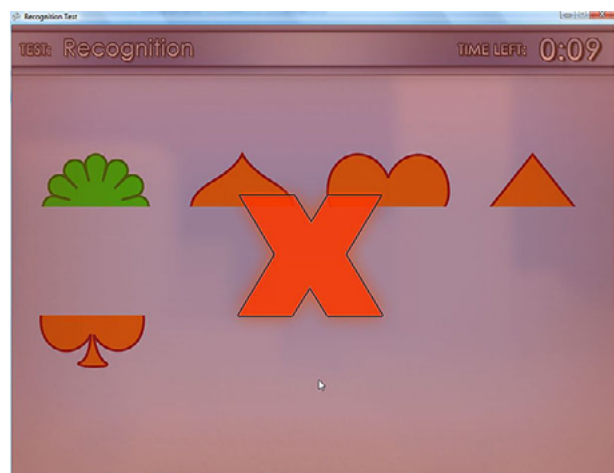

If a wrong answer is selected, a red cross will appear. After this, the next figure will appear.
After finishing the exercise the results will be shown:



The preset settings are blue. The correct matches are green.
Press any key and the following results are shown:

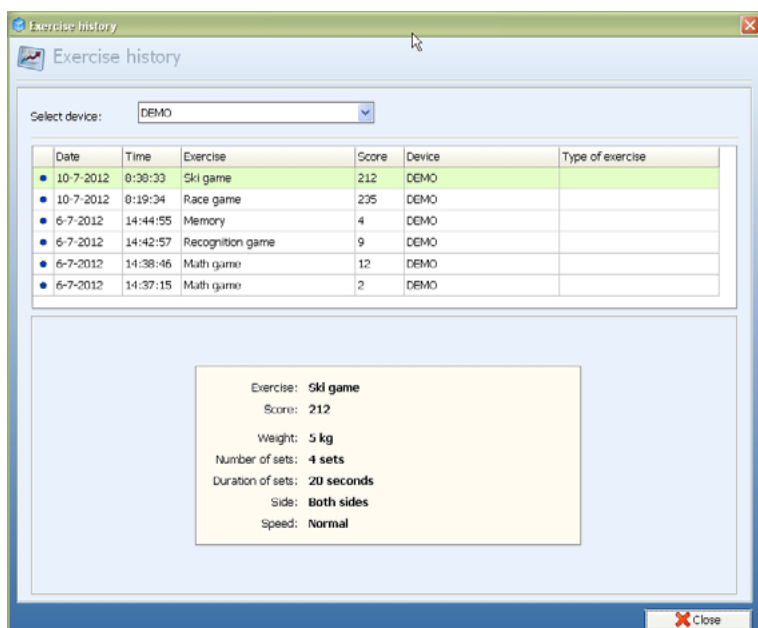

Exercise history


Select device: DEMO

Date	Time	Exercise	Score	Device	Type of exercise
10-7-2012	0:38:33	Ski game	212	DEMO	
10-7-2012	0:19:34	Race game	235	DEMO	
6-7-2012	14:44:55	Memory	4	DEMO	
6-7-2012	14:42:57	Recognition game	9	DEMO	
6-7-2012	14:38:46	Math game	12	DEMO	
6-7-2012	14:37:15	Math game	2	DEMO	

Exercise: **Ski game**
 Score: **212**
 Weight: **5 kg**
 Number of sets: **4 sets**
 Duration of sets: **20 seconds**
 Side: **Both sides**
 Speed: **Normal**

Close

10.1.3 Memory

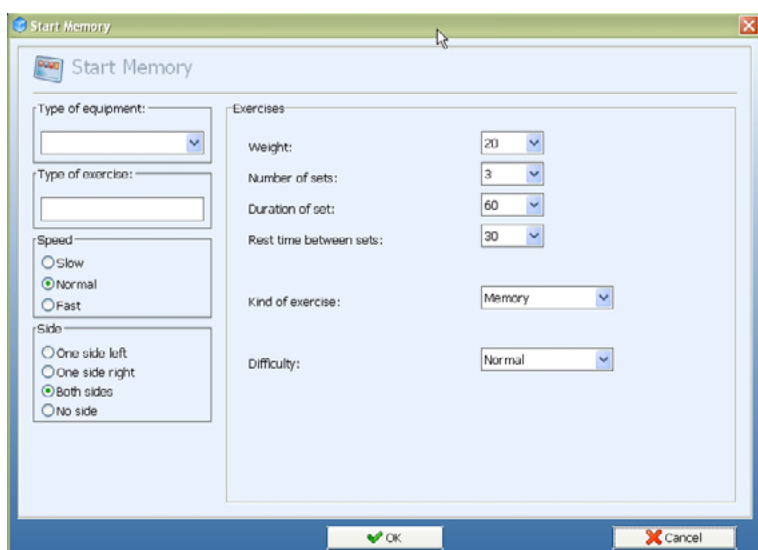

Start Memory

Type of equipment: [dropdown]

Type of exercise: [dropdown]

Speed:
☐ Slow
☒ Normal
☐ Fast

Side:
☐ One side left
☐ One side right
☒ Both sides
☐ No side

Exercises:
 Weight: [20] [dropdown]
 Number of sets: [3] [dropdown]
 Duration of set: [60] [dropdown]
 Rest time between sets: [30] [dropdown]
 Kind of exercise: [Memory] [dropdown]
 Difficulty: [Normal] [dropdown]

OK Cancel

How to set up a Memory exercise step by step:

- 1) Select "type of equipment" where the equipment is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right, both sides or "no side" training
- 5) Select the exercise weight
- 6) Select number of sets
- 7) Select duration of set
- 8) Select the rest time between sets
- 9) Select your exercise: Memory
- 10) Select difficulty
- 11) After starting, the range of motion has to be set by doing one repetition:



After pressing <Enter> the exercise will start.



Cards (colors or numbers) are laid out face up, they turn black after a number of seconds. A new card will appear. Choose the matching card by moving the new card and place it under the correct card. If the two cards match, the player scores one point.

Level settings:	Easy	Normal	Hard
Time (in seconds) to show cards	5	4	3
Retry after wrong answer	3	2	1
Number of questions per set	4	3	3


Time of selection is depending on speed:

Slow	2 sec.
Normal	1 sec.
Fast	0.5 sec.


If a correct match is chosen, a green check mark will appear. After this, the next new card will appear. If a wrong answer is selected, a red cross will appear. After this, the next set of cards will appear.




After finishing the exercise the results will be shown:

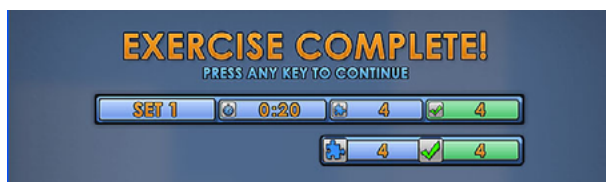


The preset settings are blue. The correct matches are green. Press any key and the following results are shown:

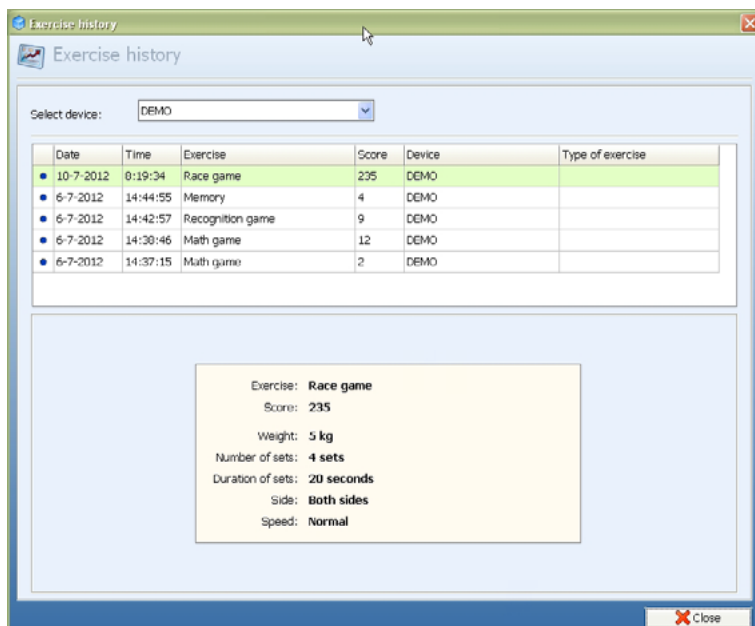

10.2 Sports

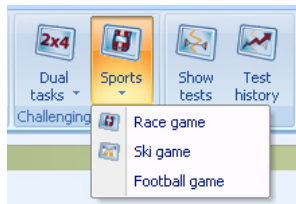

10.2.1 Race Game

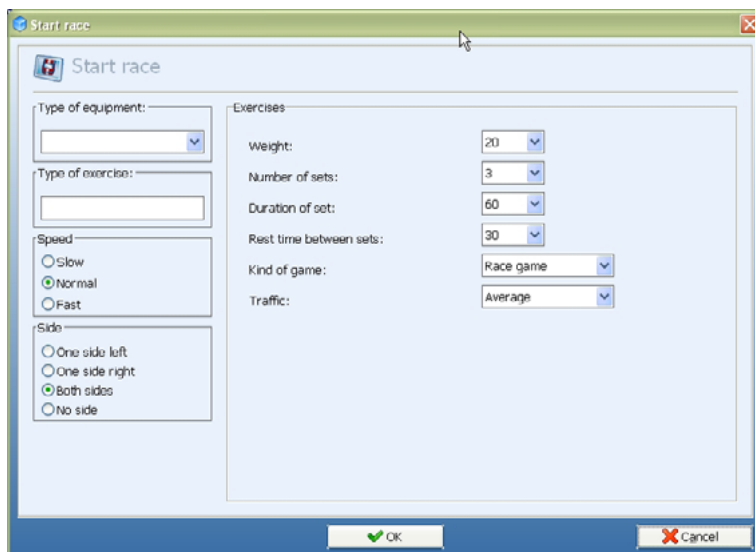

How to set up a race game step by step:

- 1) Select "type of equipment" where the equipment is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right, both sides or no side leg training
- 5) Select the exercise weight
- 6) Select number and duration of sets
- 7) Select the quantity of traffic
- 8) After starting, the range of motion has to be set by doing one repetition:



After pressing <Enter> the exercise will start.

Start driving and avoid other vehicles. Collecting coins by driving through them.
A yellow coin gives you extra time, a red coin gives you a temporary speed boost.
Driving on the emergency lane will diminish the speed.




If you touch another vehicle you will crash and you have to restart.




After finishing the game, the scores will be shown.



Press any key and the following results are shown:

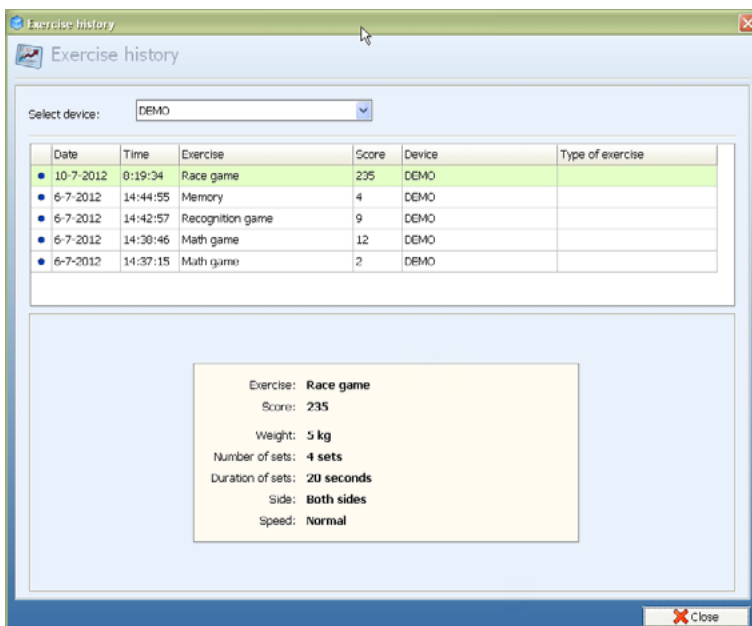

10.2.2 Ski game

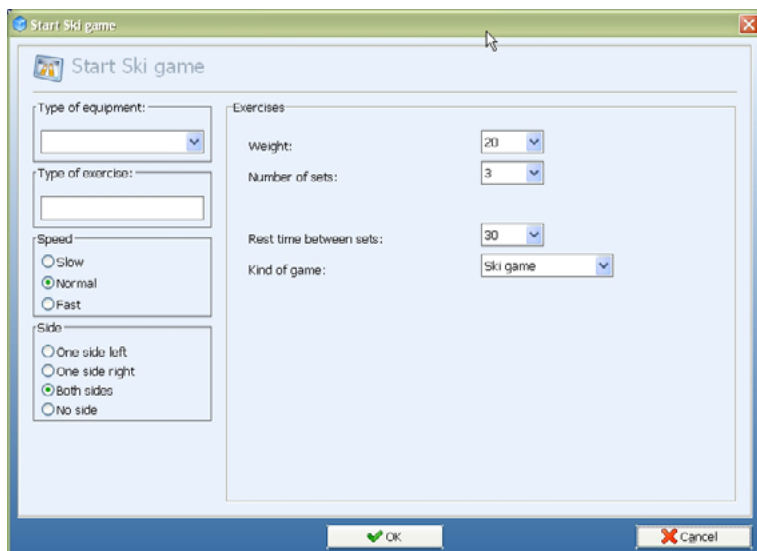

How to set up a ski game step by step:

- 1) Select "type of equipment" where the equipment is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right, both sides or no side leg training
- 5) Select the exercise weight
- 6) Select number of sets
- 7) Select rest time between sets
- 8) After starting, the range of motion has to be set by doing one repetition:



After pressing <Enter> the exercise will start.



Start skiing and try to stay within the two flags but also to ski as fast as possible to the finish.

If you do not ski between the flags, you get 10 seconds added to your time.




After a crash you have to restart skiing.




After finishing the game, the scores will be shown.



Press any key and the following results are shown:

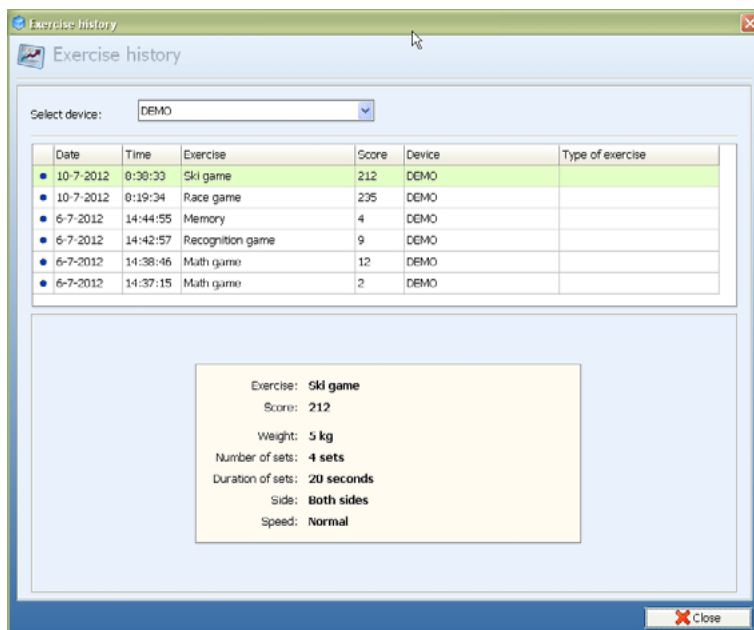

10.2.3 American Football game

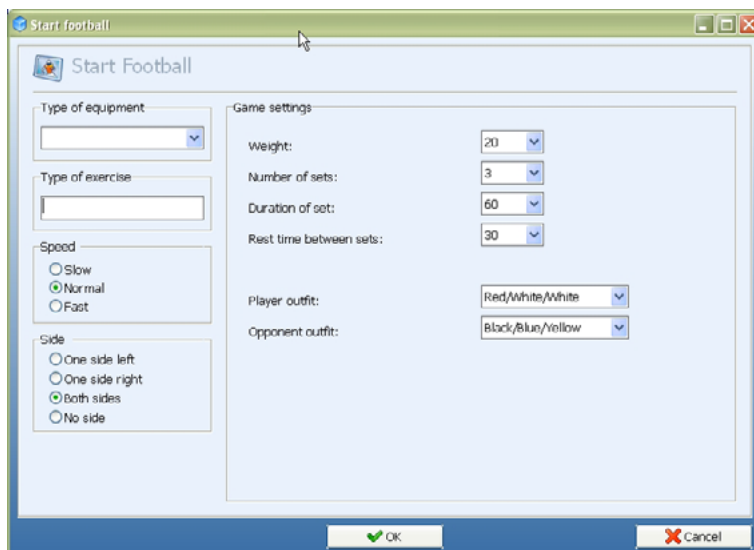

How to set up a football game step by step:

- 1) Select "type of equipment" where the equipment is connected to (optional)
- 2) Describe the type of exercise in "type of exercise" (optional)
- 3) Select the speed of the exercise
- 4) Select one side left/right, both sides or no side training
- 5) Select the exercise weight
- 6) Select number and duration of sets
- 7) Select rest time between sets
- 8) Choose the outfit of the player and the opponent
- 9) After starting, the range of motion has to be set by doing one repetition:



When confirmed, the game will start.




The aim of the game is to make a touchdown. You have to cross the field without being tackled. Every game you have to run 100 yards, the time to achieve this, can vary.

If you get too close to your opponent, you get tackled. You have to start over, at the last position.




The score will be shown and you can try again.
The amount of yards run will be added.
You start at the total amount of yards you ran.

When the duration time has been achieved, you get a time-out. This screen appears:

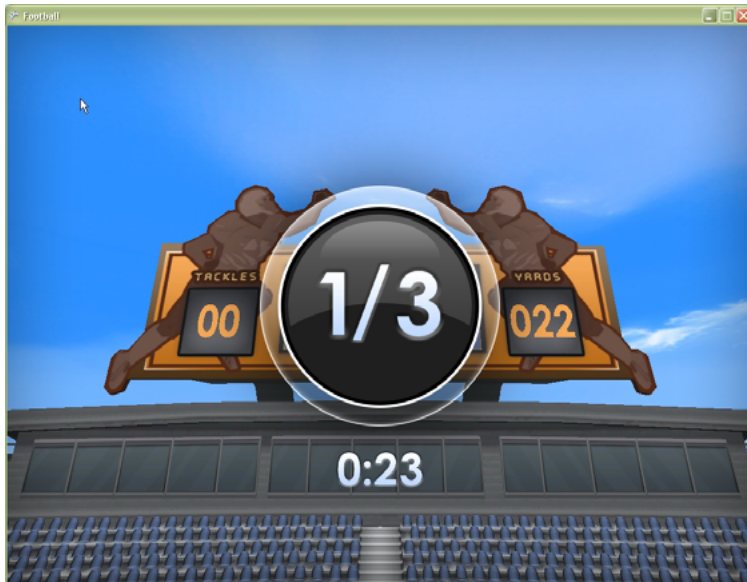


You can also see how many seconds of rest time you still have left.


This is the goal: achieving Touchdown:

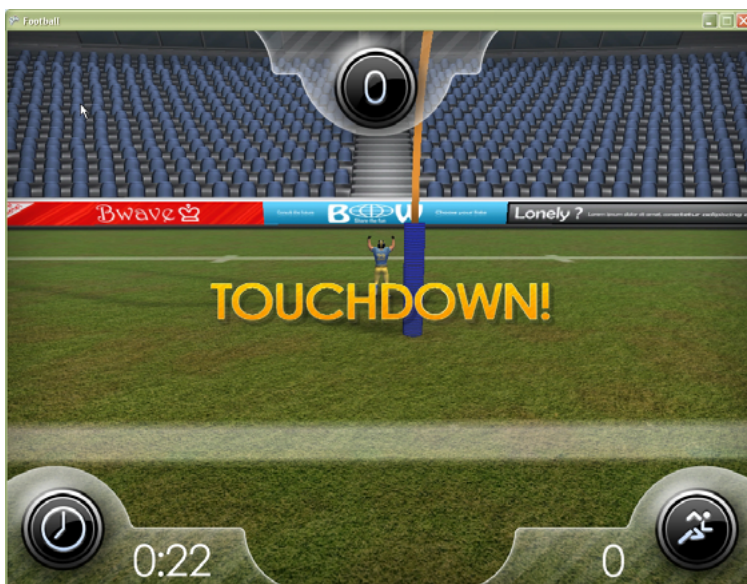

11. Testing with the MR Cube

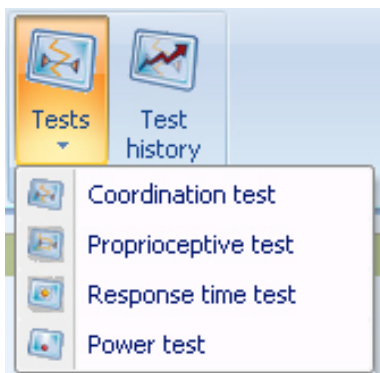


Choose 'Show tests'. Choose a test in the following screen:

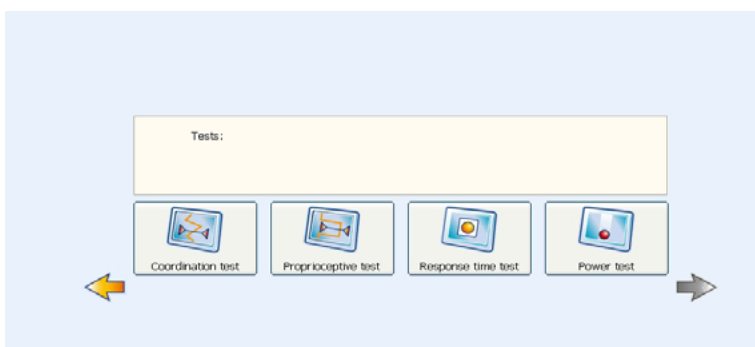

11.1 Coordination Test

The coordination test can be done safely in early rehabilitation since it is not related to strength. With the coordination we want to look at the quality of movement, we can see both concentric and eccentric and absolute strength is not important. The first screen of the coordination test:

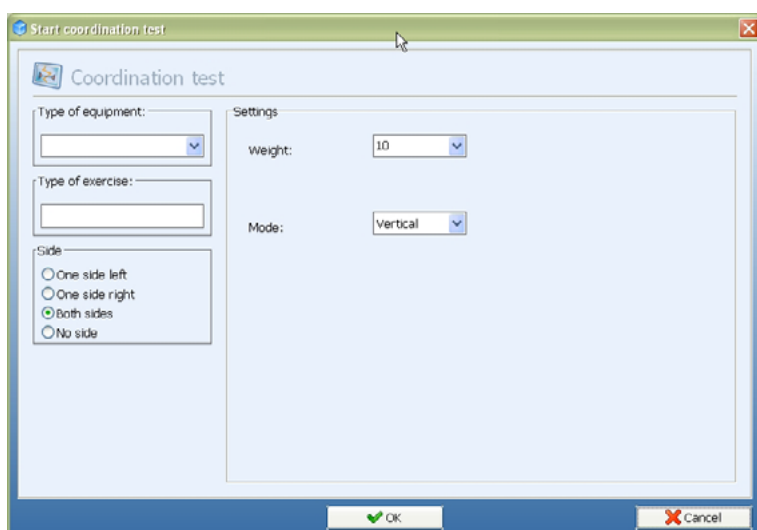

The amount of weight you select can be minimal. The test should always be easily achievable; you are not looking for strength but for coordination. 5 or 10 kg is enough weight in most of the tests you will do.

Mode: Choose "Vertical" or "Horizontal" moving of the yellow line.

Set min/max: Where in the previous max. force test and endurance strength test the range of motion was not so important, in the Coordination test it is extremely important to select the proper range of motion. Make sure the foot does not come off of the foot plate when setting the Min/Max.

After starting, the range of motion has to be set by doing one repetition:



The Min value should be around "0" the max value is the end of the range. If the "min" value is not "0" please reset the electronics (Reset button). Then make the repetition (to set min/max) again, ending in the "min" position.


The test will start in the "min" position.

Press <Enter> to start the test.


Test instructions: keep the red crosshair on the yellow line as exact as possible. The test starts with the Non-Involved side and afterwards the Involved side.



After the test (one side) the following results will appear:

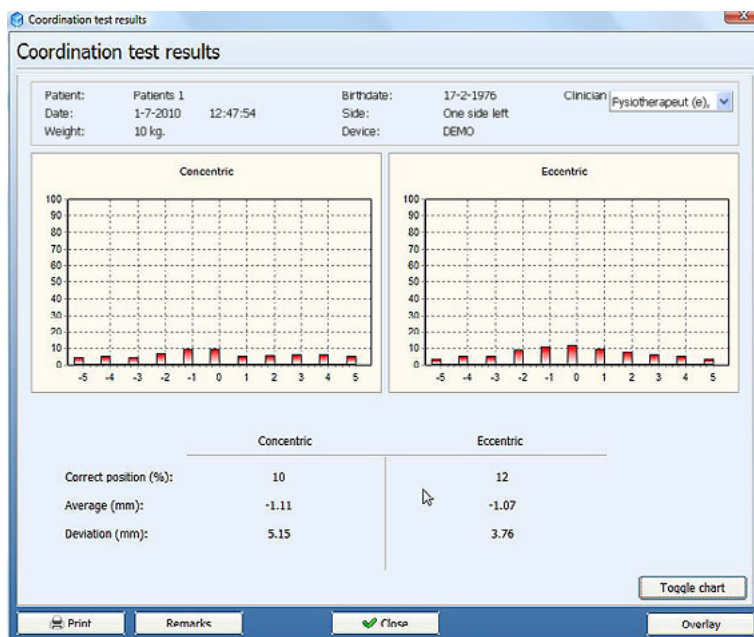

After the test (both sides) the following results will appear:

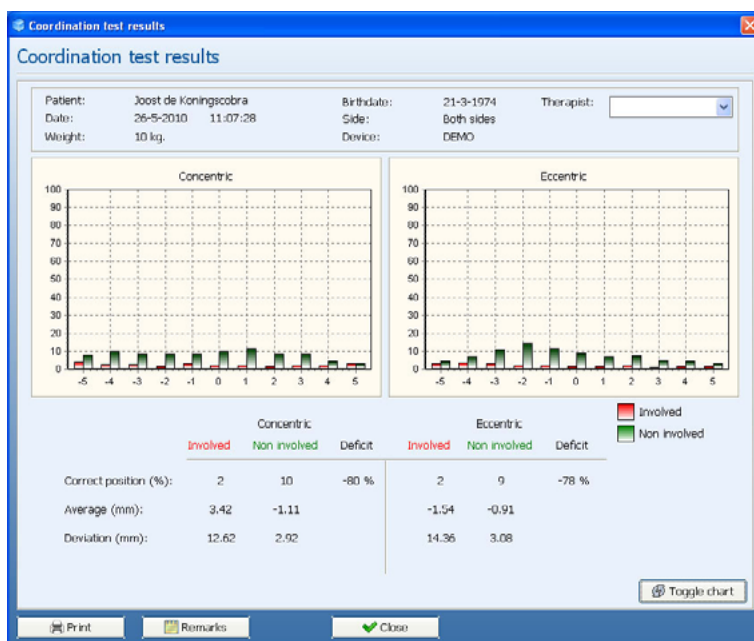

To print to paper use the <Print> button. If you want a print preview on the screen use the command; "Ctrl+p". To add a note use <Remarks>.

To view a line chart, use <Toggle chart>.

To compare results use <Overlay>. Choose the test results you want to compare and press <OK>. The results will be displayed.

Coordination test results



DEMO

Name: Joost de Koningscobra
Birthdate: 21-3-1974
Test date: 1-7-2010
Weight: 10 kg.

Clinician: Joost Heuperman
Referral: Akkermans
Involved side: Left
Side: Both sides

Concentric	Involved	Non involved	Deficit
Correct position (%):	2	10	-80 %
Average (mm):	3.42	-1.11	
Deviation (mm):	12.62	2.92	


Eccentric	Involved	Non involved	Deficit
Correct position (%):	2	9	-78 %
Average (mm):	-1.54	-0.91	
Deviation (mm):	14.36	3.08	

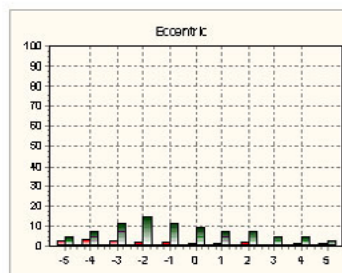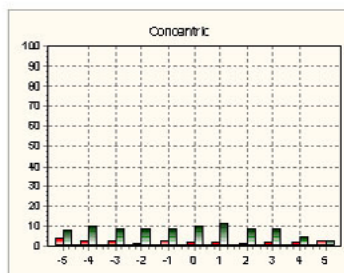

■ Involved
 ■ Non involved

11.2 Interpretation Coordination Test

Directly after the test the shape of the graphics will be shown;

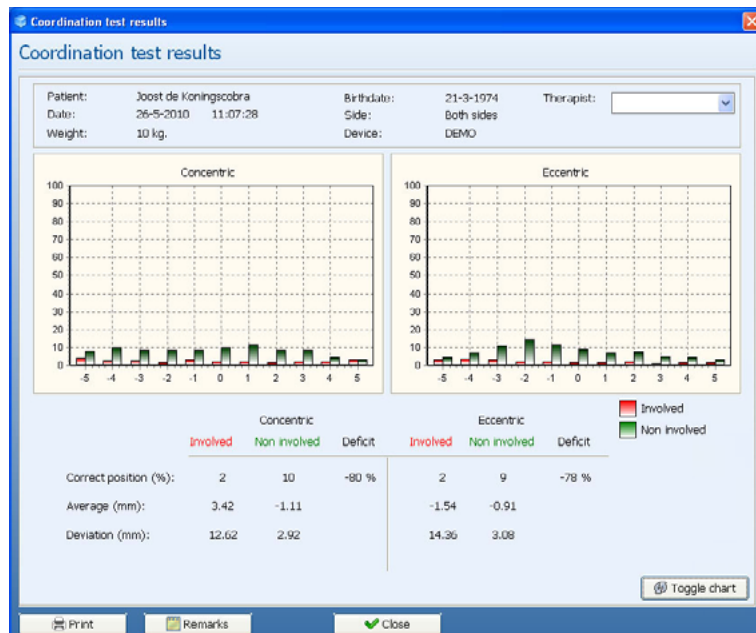

First we look at the bar graphs. What we want to see in both Concentric and Eccentric is that the highest bar is around zero. In the graphic the "o" point means that during the test the crosshair was exactly on the blue line. In the graph left from zero (minus region) means that during the test crosshair was at the left side of the blue line. In the graph right from zero (plus region) means that during the test the crosshair was at the right side of the blue line. The height of the bars in the graphs represents the amount of time the red crosshair was in that specific position during the total test time (% exercise time).

What we expect to see, in an ideal test, is one bar at the zero position, with a score of 100% of the test time. This ideal test result is not likely.

The above (involved) test is an example of a test result which would indicate poor coordination:


- a low bar on the "0" position means the test person was not able to keep the red crosshair at the yellow line for a longer time
- many small bars spread left and right from the zero position means the test person was not even able to keep the red crosshair on the yellow line but was moving around the yellow line. The more bars we see at a bigger distance from the "0" point (left and/or right) the more moving around the yellow line was registered and therefore less control was demonstrated.

The numbers we see on screen for both concentric and eccentric:


- Weight: The weight selected for this test
- Duration: The time in seconds selected for this test
For a reproducible test it is important to have the same weight and duration in the next test of the same person.
- Average: We get a separate value for Average Concentric and Eccentric for both Involved and Non-Involved

The blue line is the zero point. The average is calculated by adding all the deviations and dividing them by the number of measurements (25 Hz). If the average is "0", or nearby, this normally means (if also the deviation is low) the patient demonstrated good coordination.


Some examples :

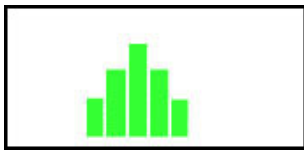

In this test the average will be around "0"

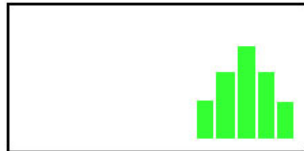

In this test the average will be above "0" i.e. 5 cm

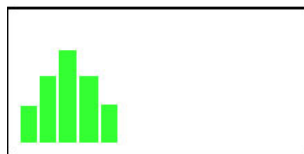


In this test the average will be minus "0" i.e. -5 cm

Deviation: The deviation is a value for the variation in the measurements. A higher value means there is a greater spread in the positions during the measurement. The test person with a high deviation has been moving far above or under the yellow line a lot and did not stay close enough (or on) to the yellow line.



Low Deviation

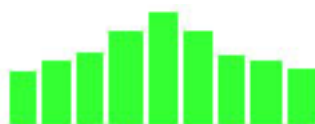

High Deviation

If you have an average around "0" you could think this is a good test but when the Deviation is high, this "good" average is a lucky shot and the test person did not perform well.

Some examples:

Low Deviation

High Deviation

Small Average

Very good

Can not stay stable on track

Big Average


Steady, but not at track

Unstable and can not stay on track

In the perfect test we see that both the Average and the Deviation are close to "0"

The Correct position: is the amount of time the crosshair is on the yellow line expressed as an percentage of the overall test time. The larger this percentage, the better the test person performed.


11.3 Power Test

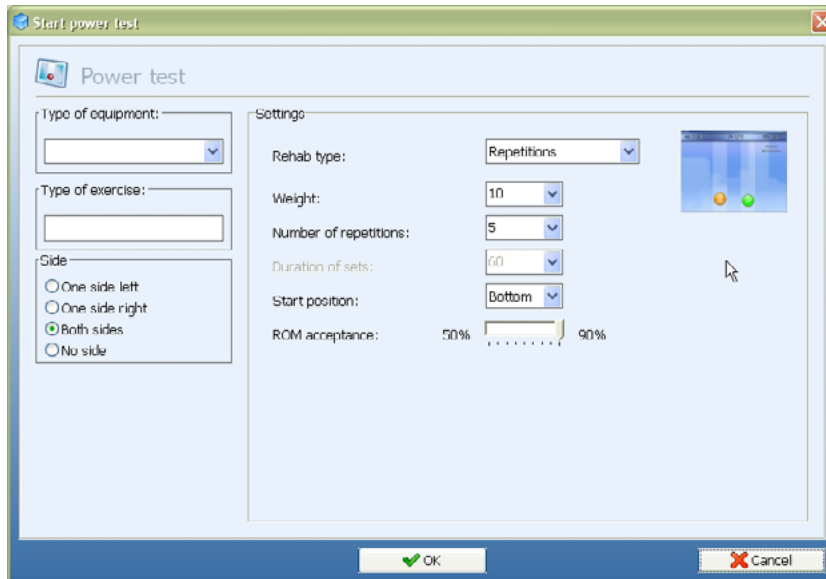

- 1) Select Type of equipment (optional).
- 2) Choose Number of repetitions or Repetitions per time. After choosing Repetitions per time fill in the Number of sets and Rest time between sets.
- 4) Select weight.
- 4) Select number of repetitions or duration of sets.
- 5) Select start position.
- 6) Set Range of Motion acceptance, this is the percentage of deviation when the software registers the repetition.

Choose <OK> to begin the test.


After starting, the range of motion has to be set by doing one repetition:

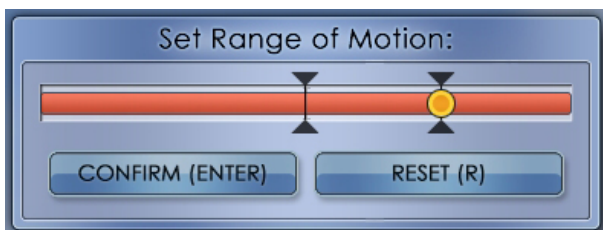

After pressing <Enter> the test will start.




After finishing the test, the following screen will appear:

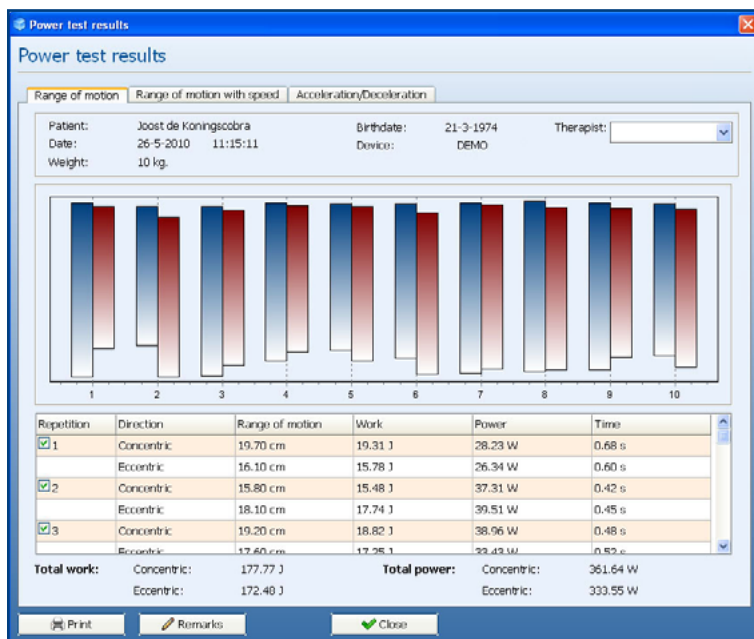

If the repetition of the test is activated, these results will be counted into the Total work and Total power. Press the next tab for going to the next page.



The average speed is shown per repetition. The second repetition of the fatigue section is always set at 100%. The other repetitions are calculated on this percentage. Change legend speed can be adjusted in steps of 15 cm/s. Press the next tab for going to the next page.

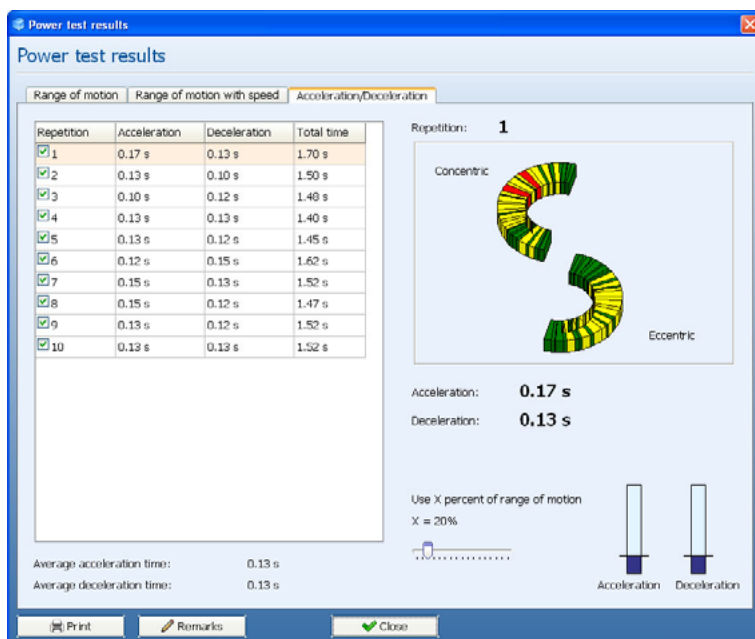

This screen shows per repetition the deceleration and acceleration of the test. "Use x% of range of motion" shows the area of the motion in which the deceleration and acceleration is calculated per second. To view the visual image of the motion, click on the specific repetition in the grid.

Press <Print> or "Ctrl+p" to print the report. To add a note use <Remarks>.

Power test results

DEMO

Name: Joost de Koningscobra
Birthdate: 21-3-1974
Test date: 26-5-2010
Weight: 10 kg.
Diagnosis: ACL

Clinician: Joost Heuperman
Referral: Akkermans
Involved side: Left
Side: Both sides

Concentric


Involved

Total work: 177.77 J
Total power: 361.61 W
Average speed: 36.90 cm/s
Average acceleration time: 0.13 s

Eccentric

Involved

Total work: 172.48 J
Total power: 333.55 W
Average speed: 36.90 cm/s
Average deceleration time: 0.13 s

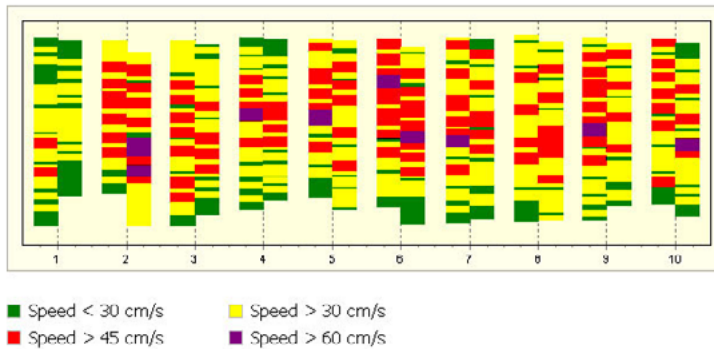


11.4 Proprioceptive Test



The Proprioceptive Test can be done safely in early rehabilitation since it is not related to strength. With the Proprioceptive Test we want to look at the quality of reproducing a certain joint position.

The first screen of the Proprioceptive Test:

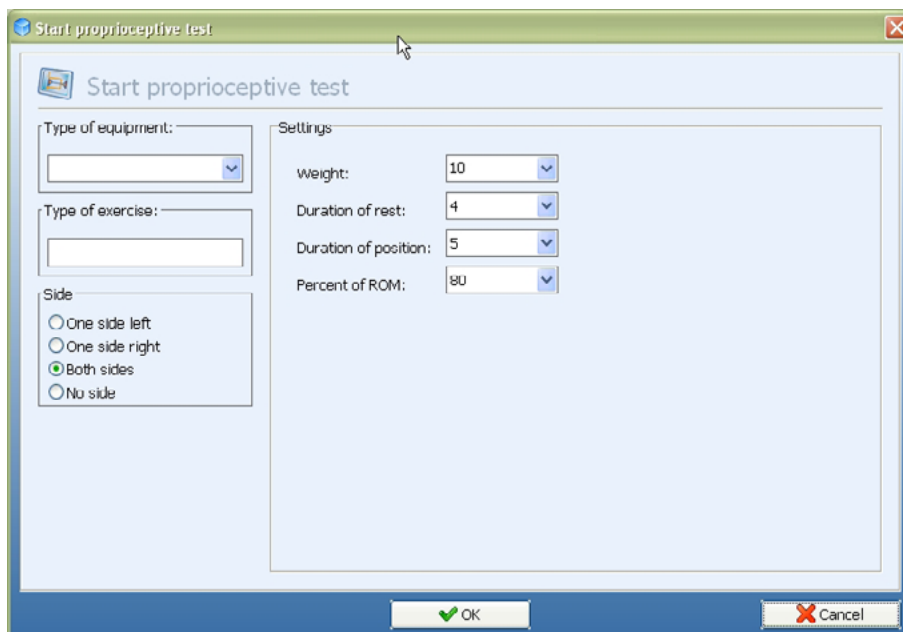

Start proprioceptive test

Type of equipment:

Type of exercise:

Side:

- ☐ One side left
- ☐ One side right
- ☒ Both sides
- ☐ No side

Settings:

Weight:

Duration of rest:

Duration of position:

Percent of ROM:

OK Cancel

- 1) Select Side: One side left/right, Both sides leg or No side (no use of legs) training.
- 2) The amount of weight you select can be minimal. The test should always be easily achievable; you are not looking for strength but for proprioception. 5 or 10 kg is enough weight in most of the tests you will do.
- 3) Duration rest: the amount of time the test person has to be in the rest position. The longer this time the more difficult to re-acquire the "correct position"
- 4) Duration position: the amount of time the test person has to be in the "correct" position.
- 5) Percent of ROM: The joint position where we test proprioception, expressed in terms of the % of the Range of Motion. We suggest a position between 30 % and 70%
- 6) After starting, the range of motion has to be set by doing one repetition:




Set Range of Motion:


After pressing <Enter> the exercise will start.

Test instructions:

- The test person is asked to replicate joint position (fixed % of the ROM) four times.
- The first and second repetition are with feedback
- The third and last repetition without feedback on the screen. We ask the test person to close their eyes in order to prevent the use of extra visual feedback (e.g. height of weight stack, joint angles) to replicate the joint position. The tester should give verbal cues when the test person should move from rest position to the desired joint position and back again.

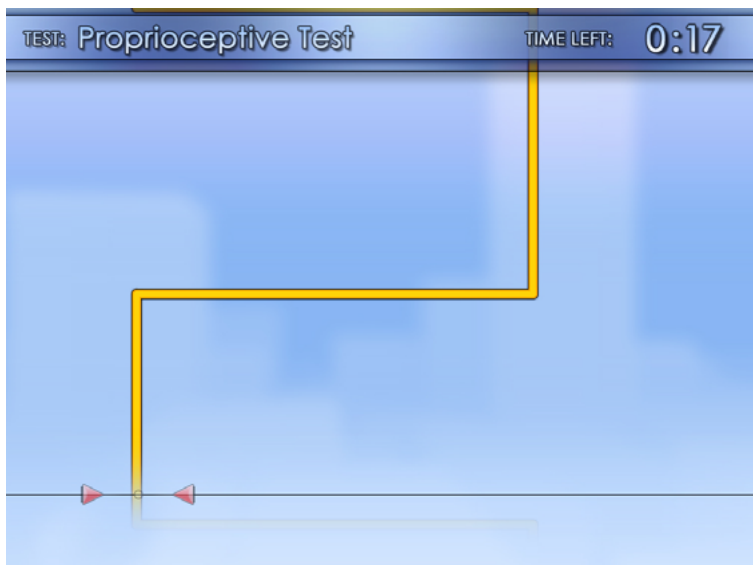


The test person should move to the right vertical line before the horizontal line disappears and maintain the crosshair on that line until the next horizontal line disappears. Repeat the same process for all four positions. Remember the last two positions have no crosshairs.

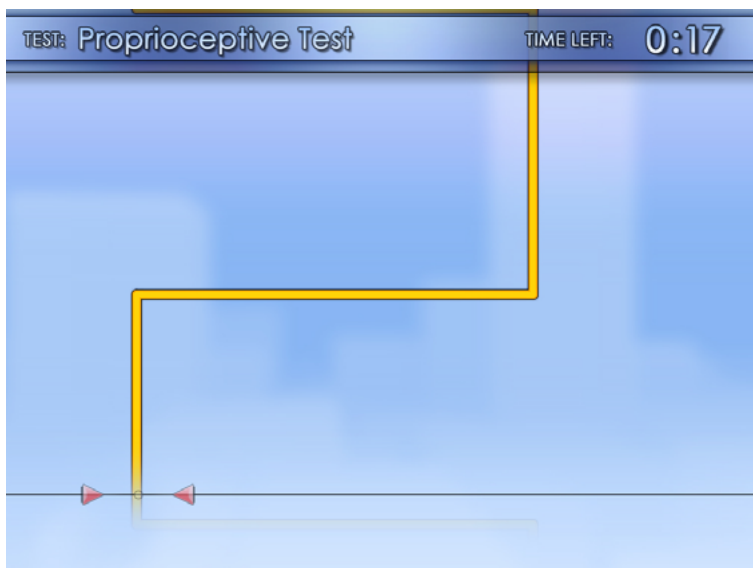


The test person should follow the hypothetical red line and be in time at the “correct” position (black circle), at this point the measurement starts and the horizontal line disappears. After two times in position the red crosshair disappears and we ask the test person to close his/her eyes so no visual input at all is possible.

Test overview:

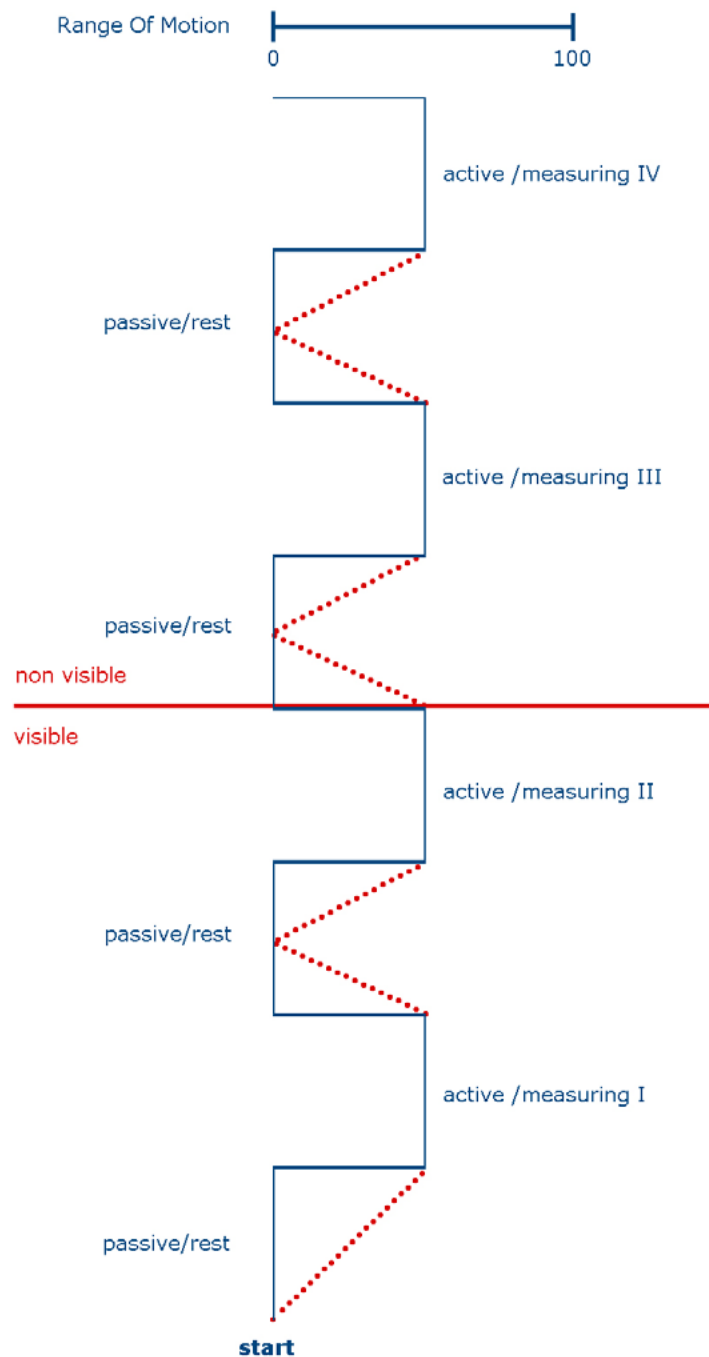


After the test the following screen appears:

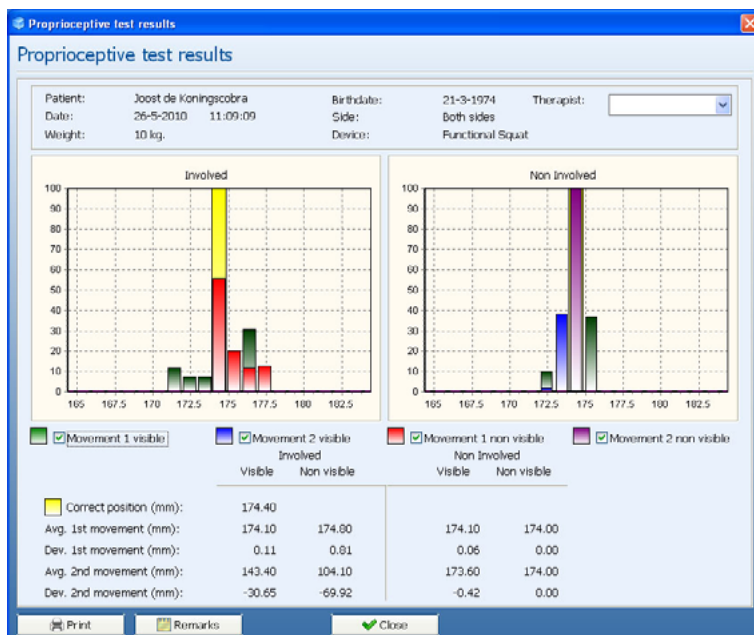

To print to paper use the <Print> button. If you want a print preview on the screen use the command; "Ctrl+p"


To add a note use <Remarks>.

To compare results use <Overlay>. Choose the test results you want to compare and press <OK>. The results will be displayed.

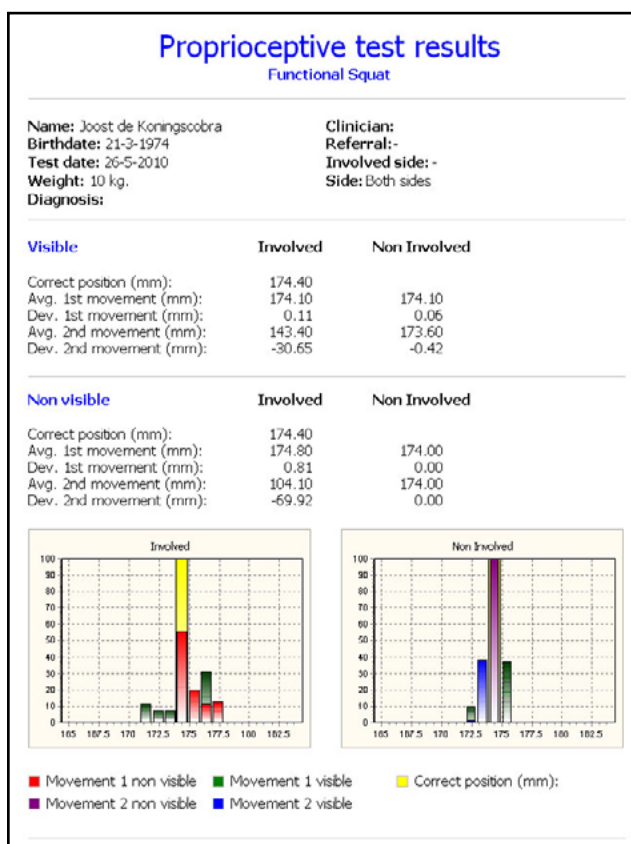

11.5 Interpretation Proprioceptive Test

Directly after the test we check the distribution of the bar graphs.

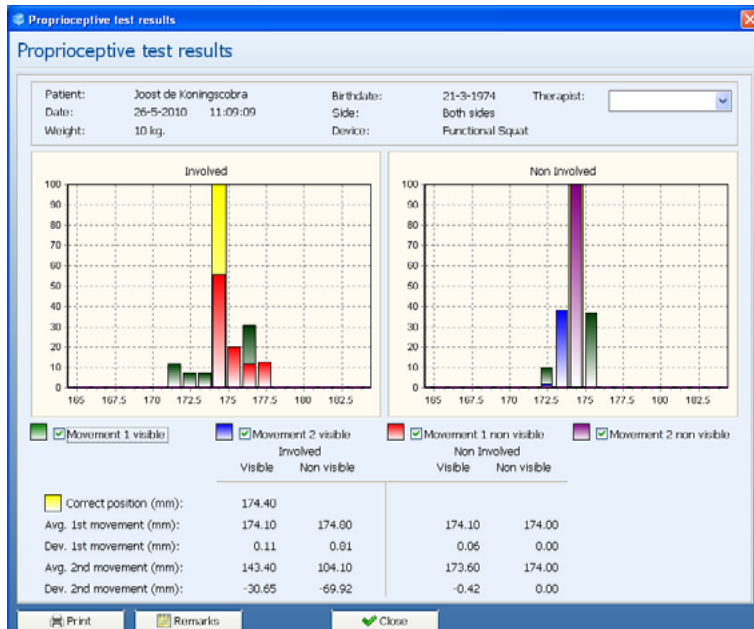

The green and blue marker (first and second) show us the actual position of the patient relative to the correct position when the crosshair was present (reps 1&2).

The red and purple marker (first and second) show us the actual position of the patient relative the correct position after the crosshair disappeared (reps 3&4).

Green and blue bar:


The green and blue bar do NOT tell us anything about the quality of Proprioception!!

If the patient is not capable of getting the green and blue marker in the correct position, this means the coordination is not OK. This information tells us that we are not ready to do a Proprioceptive test; but rather we should train to achieve better coordination ability first.

If the patient has both green and blue bar on or close to the correct position, we can interpret proprioceptive ability by reviewing the size and location of the red and purple bar:

Red and purple bar:

The red and purple bar tell us how well the patient can re-acquire the correct joint position (learned with the green and blue bar). The bars should be as high as possible, meaning a long time at a specific position, ideally in the neighborhood of the correct position. If you observe many small bars, the patient was moving around during the test time and uncertain about the correct position. In the graphical view, it may not always be possible to see all the bars, so we refer to the data including average and deviation.

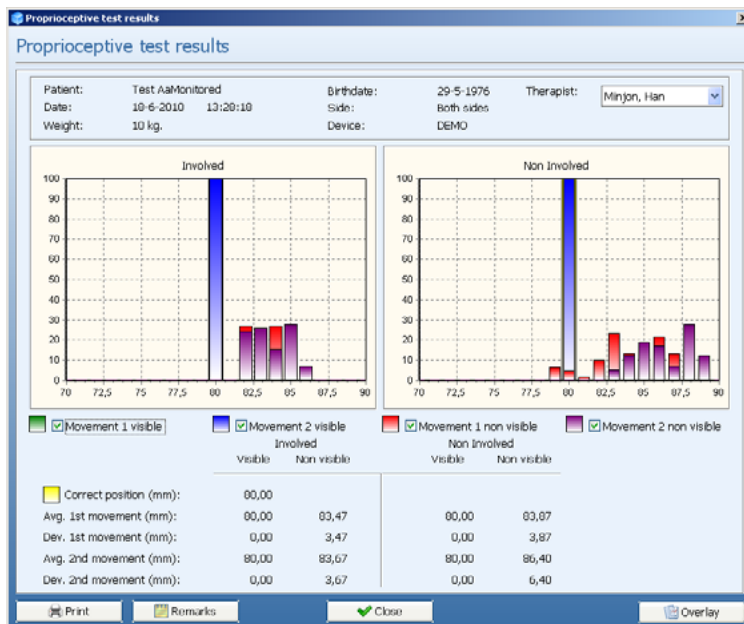

In the above picture you see that the green and blue bar are on the correct position. Both the red and purple bar are moving during the test through a wider range of motion as expected.

To print to paper use the <Print> button. If you want a print preview on the screen use the command; "Ctrl+p". Press <OK> to get a preview.


Proprioceptive test results			
DEMO			
Name: Test AaMonitored		Clinician: Han Minjon	
Birthdate: 29-5-1976		Referral: Lindenhovius	
Test date: 18 6 2010		Involved side: Both	
Weight: 10 kg.		Side: Both sides	
Diagnosis: asdfasdf			
Visible	Involved	Non Involved	
Correct position (mm):	80,00		
Avg. 1st movement (mm):	80,00	80,00	
Dev. 1st movement (mm):	0,00	0,00	
Avg. 2nd movement (mm):	80,00	80,00	
Dev. 2nd movement (mm):	0,00	0,00	
Non visible	Involved	Non Involved	
Correct position (mm):	80,00		
Avg. 1st movement (mm):	83,47	83,87	
Dev. 1st movement (mm):	3,47	3,87	
Avg. 2nd movement (mm):	83,67	86,40	
Dev. 2nd movement (mm):	3,67	6,40	

In the above printed numbers the correct position was 80, both visible markers have an average of 80 and essentially no (or 0,1) deviation which is what we would expect of a person with good coordination.

The non visible first movement average was 83,47 which tells us that the test person overshot the correct position 3,47 cm to the right.


The non visible second movement average of 83,67 is 3,67 cm less than the expected position, but the deviation of 3,67 indicates the test person had uncertainty about their joint positioning.

The numbers in a good test :

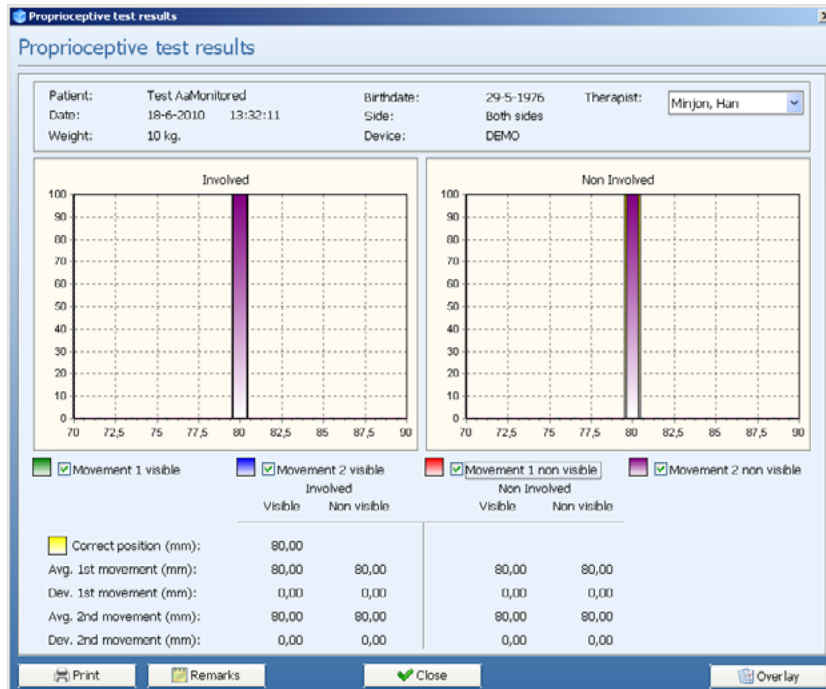


In this case the correct position was 80 and both the green bars and the red bars are in the correct position (average 80) and without any deviation (stable during the test). This is the best score possible.

11.6 Response time test

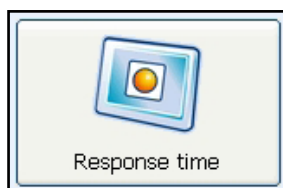

The Response Time test is designed to measure the test person's reaction time using an unexpected movement and their ability to finely control the weight after moving it.

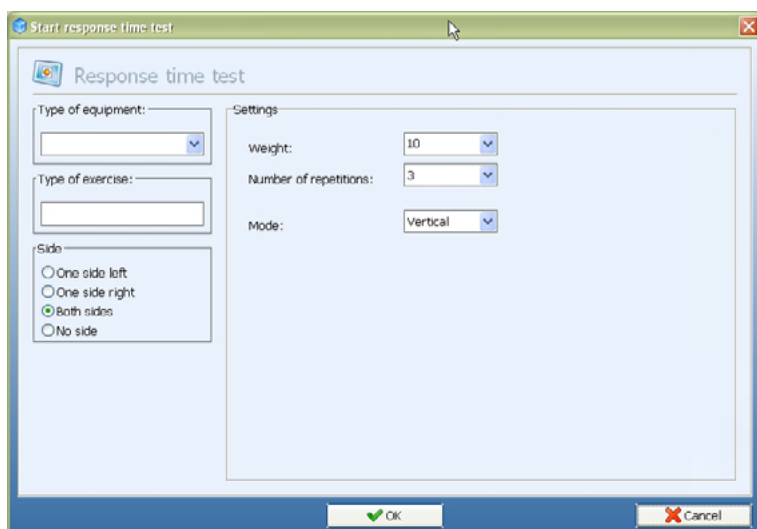

The Start Position corresponds to the specific location (expressed as a percentage) within the full range of motion where the test will be performed.

Weight: Use 5 to 10 kg's for the performance of this test
Choose <OK> to begin the test.

After starting, the range of motion has to be set by doing one repetition:



After pressing <Enter> the test will start.




Instruct the patient to move the orange circle into the empty box. When the circle is relocated to the center of the box, the box will turn green. After a couple of seconds (the time varies), the box will randomly move to either the left or right of its initial position. The patient should be instructed to relocate the circle into the box at the new position as fast as possible. The time that it takes for the test person to react and initiate movement of the circle is the "Response Time".



When the circle is properly located in the box, the box turns green. It is essential to control and hold that new position so that the box remains green. The time for this to occur is designated the "Time to Finish". The test administrator must then click the <Next Measurement> button or the "Enter" key on the keyboard. Each side is tested three times. (unless we are using the Back extension or Abdominal Crunch machines).




After three measurements are collected for the non-involved side, the following screen appears:

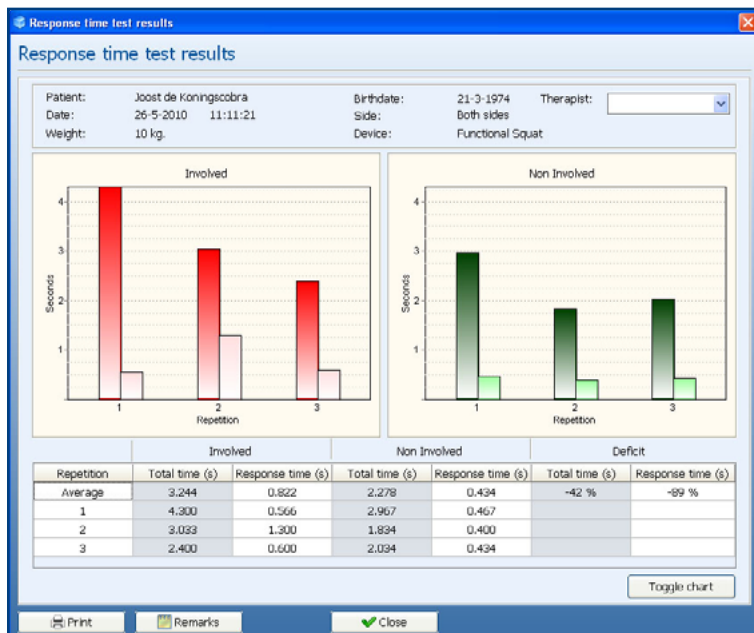

To print to paper use the <Print> button. If you want a print preview on the screen use the command; "Ctrl+p"

To add a note use <Remarks>.

To view a line chart, use <Toggle chart>.

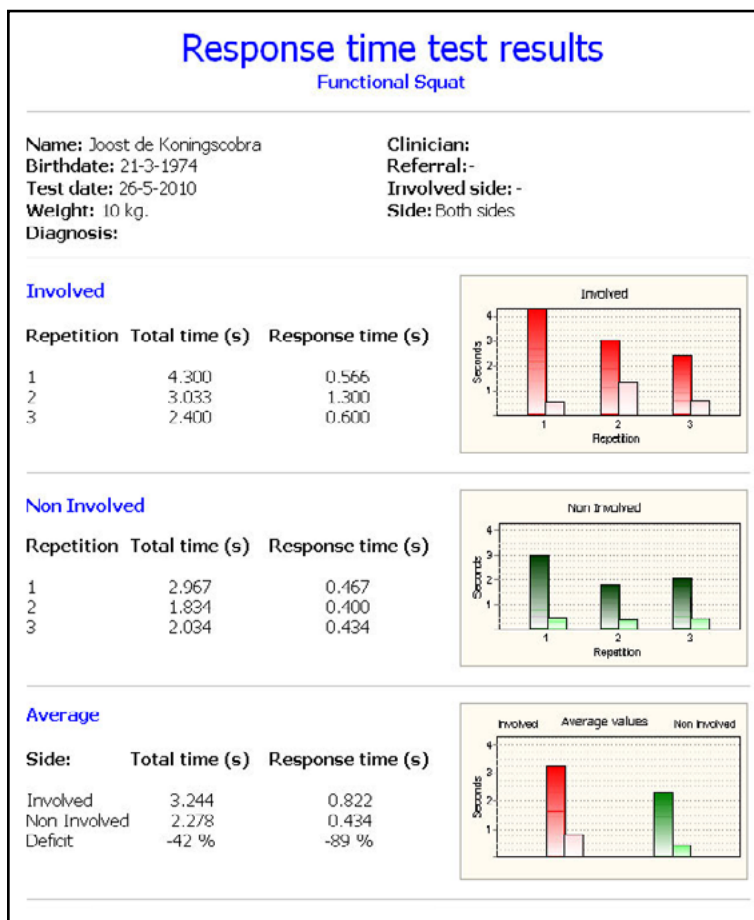

11.7 Interpretation response time test

Response Time: is defined as the time that it takes for the test person to react and initiate movement of the stick figure in response to the moved box. The "Response Time" is sometimes represented as a negative number. This means that the test person initiated movement in the direction opposite of the box movement before eventually relocating the circle to the box.

Total time: is defined as the total time elapsed from the initial movement of the box to the point where the box has turned to red and remained red.

Three repetitions are performed for each side. The results are then averaged to facilitate the calculation of a deficit.


Deficit: Defined as the percent difference relative to the Non-Involved side, e.g. 5 sec

	Involved		Non involved
If there is no deficit the value is 0%	5	-	5
If the Involved side is 20% slower it will say "20%"	6	-	5
If the involved side is 20% faster (not expected) it say's "-20%	4	-	5


Clinical Relevance of the Data: The smaller the "Response Time", the faster the test person's ability to react to unexpected demands. Longer "Time to Finish" durations may mean less fine motor control abilities necessary for holding the moved weight.

12. Test history

In the main menu you find the option "Test history":

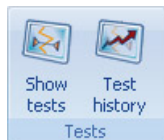


In this menu you can find all the tests the selected patient performed. You will either see all the tests on the machine selected or (use toggle) to see all tests the patient performed on all machines (if they are connected to a network).

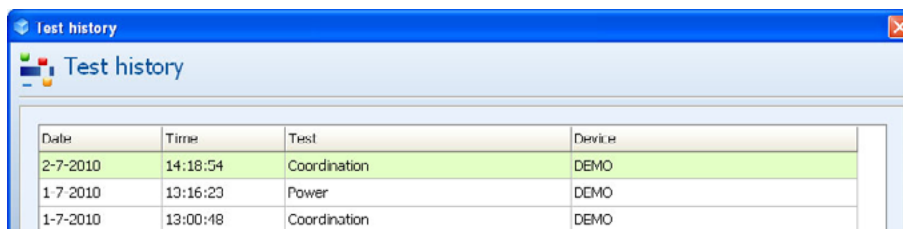


Date	Time	Test	Device
2-7-2010	14:18:54	Coordination	DEMO
1-7-2010	13:16:23	Power	DEMO
1-7-2010	13:00:18	Coordination	DEMO

Select the test and press <OK>, the test results will be displayed. If you want to compare tests of the same type, select the first test and use F12 (overlay) to select the other test.

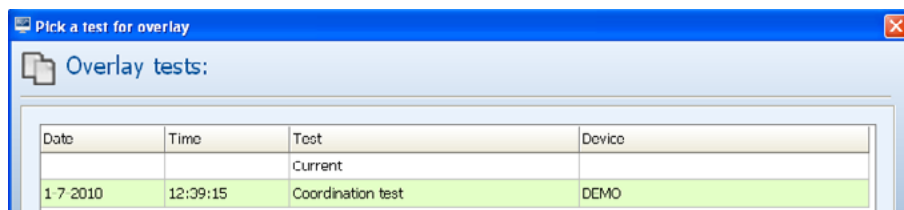

Press <OK>. The results will be displayed in the following screen.

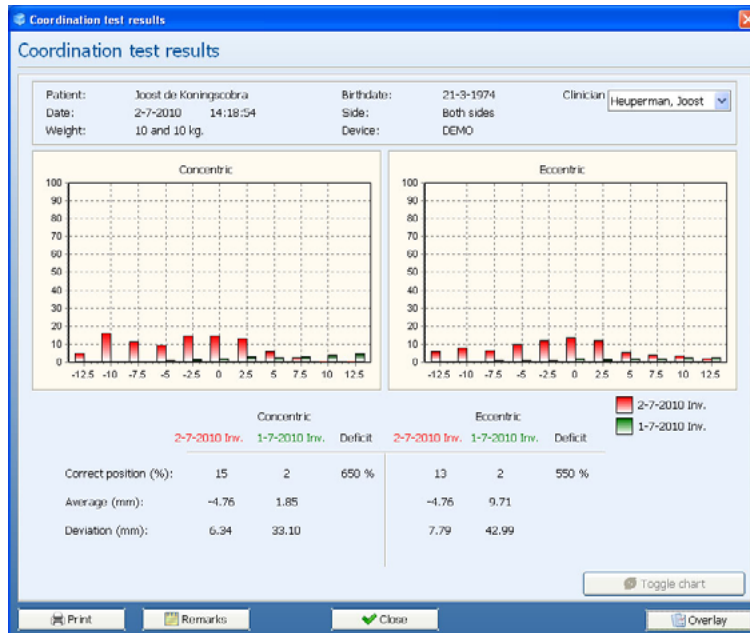


13. MR Cube Settings

The following settings can be found under the 'Settings' tab.

13.1 Show serial



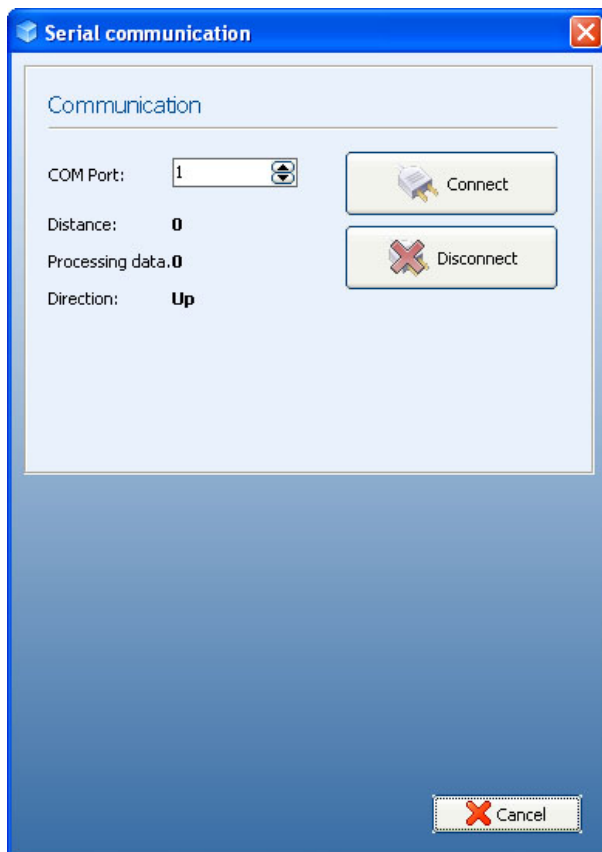

COM Port: select the comport where the electronics of the machine are connected to the computer.

Force: value force should be moving if the machine is being used.

Distance: should be moving if the machine is being used.

Direction: the direction of the cable, "up" or "down".

Press <Connect> to start communication with the electronics.

Press <Disconnect> to stop communication with the electronics.

13.2 Settings



Enable Synchronization:

This enables or disables synchronization between the monitor refresh rate and the drawing of the graphics. Enabling this will try to prevent screen tearing, but might decrease the frame rate.

Enable Anti Alias:


When enabled this improves image quality at the cost of a minor hit in the frame rate.

Enable Blur:


background with blur (not sharp).

Screen resolution:


choose the resolution of your screen to optimize your view.

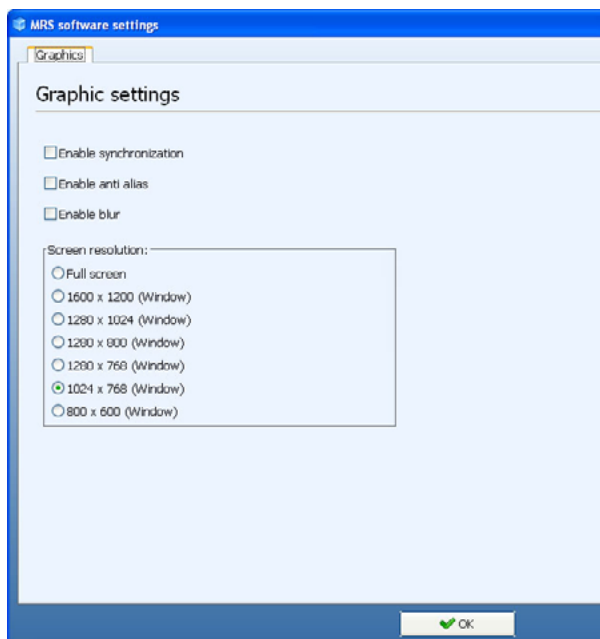

13.3 Administrator Settings

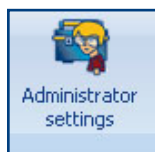


Database

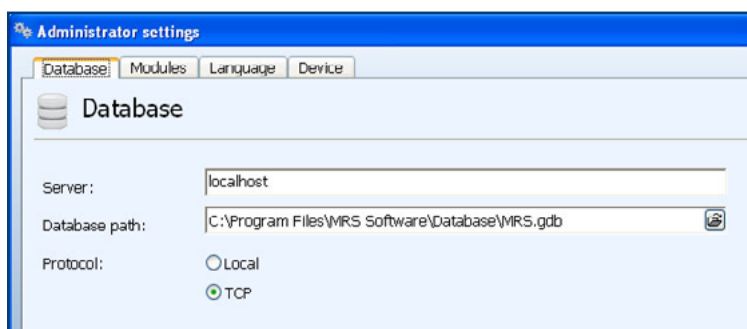

- Server: If you work with a central database on a server, the IP-address of the server should be entered in this field. If the computer is used as a server or stand alone, the setting is "localhost".
- Database path: displays the path to the database. (Both local or on a server).
- Protocol: TCP or local depending your network.

Modules:

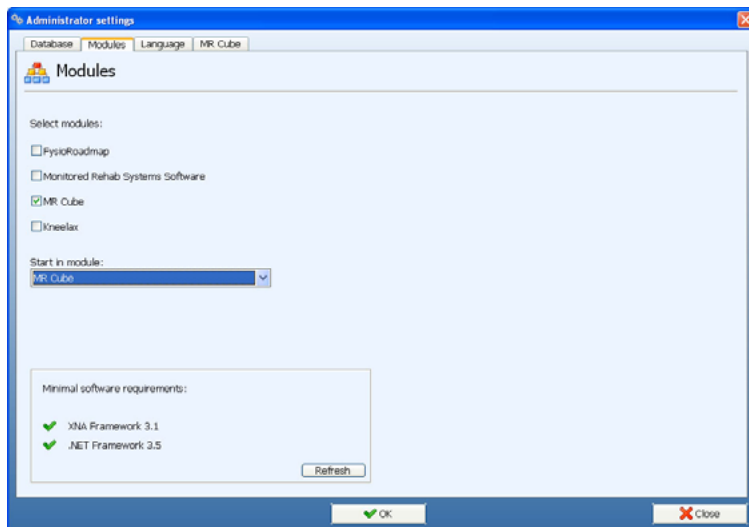

Select modules: If the software will be used also for other equipment or FysioRoadmap, select the modules you need.


Start in module: select the module you want the program to start.

Minimal software requirements: For the proper functioning of the software, additional software is required. Here is an overview of the specifications and whether the software is installed on your computer: (Green checkmark = succesfully installed, red cross = the software is not installed yet)


Refresh: after installing extra software while MRS Software is still in use, use the refresh button to update the overview.

Languages

Select your language and restart the software.

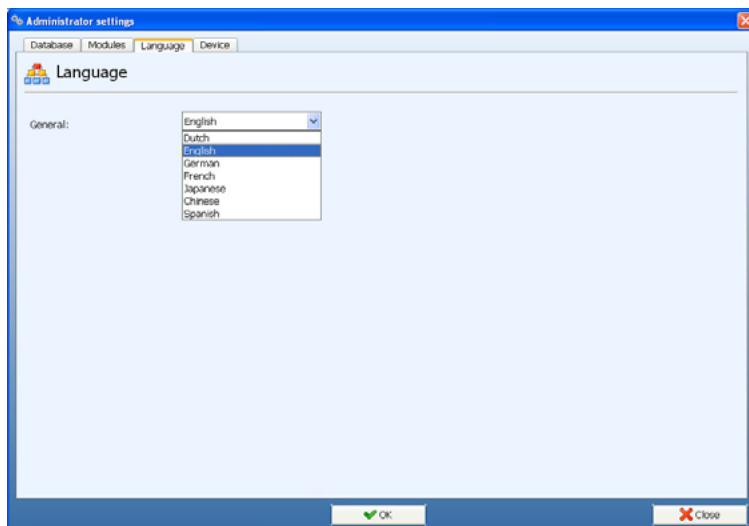

Device

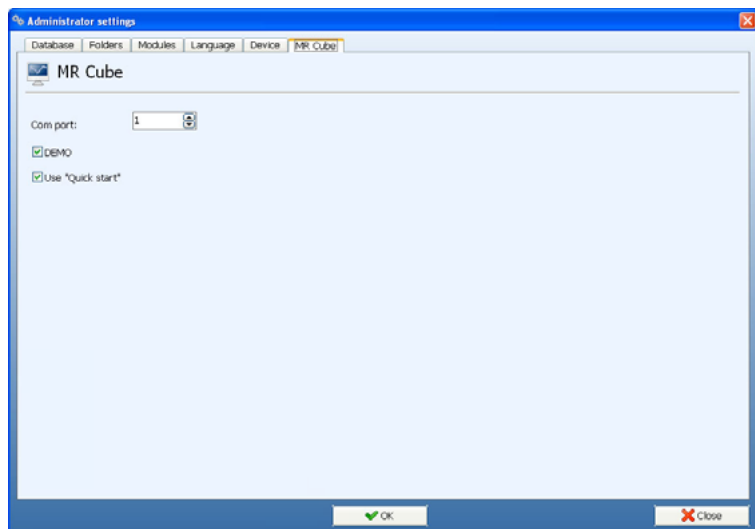

Device: select what kind of machine you are using

COM port: select the COM port where the electronics of the machine are connected to the computer.

DEMO: If there is no connection to electronics, but you want to see a demonstration of the software and use your mouse instead, use this option.

Use Quickstart: Select this option if you want to use the quickstart menu.

13.4 Back-up and restore

To make a back-up of the database, copy the file MRS.GDB when the software isn't running and put it someplace out of the installation folder.


You can find that file in the installation folder of the MRS Software.

Windows XP: C:\Program Files\MRS Software\Database\MRS.GDB

Windows Vista/7: C:\Program Files (x86)\MRS Software\Database\MRS.GDB

To restore the back-up, find the original MRS.GDB from the database folder and replace it with the back-up.

After restarting the software, the database of the back-up should be visible.

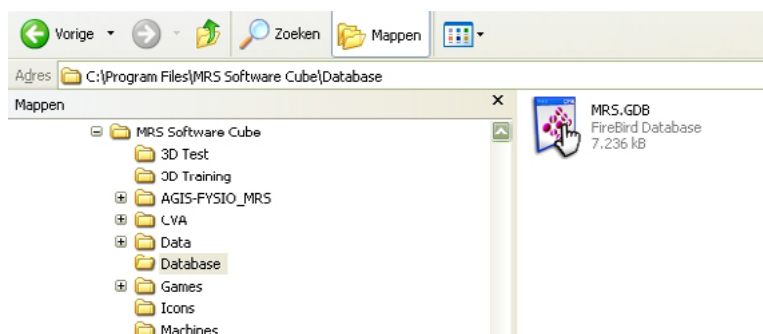

To back-up and restore from Firebird, open the Command Prompt, move into the Firebird folder (C:\Program Files\Firebird\Firebird_2_5\bin\)

```

C:\WINDOWS\system32\cmd.exe
C:\Program Files>cd Firebird
C:\Program Files\Firebird>dir/p
Het volume in station C heeft geen naam.
Het volumenummer is E4CE-FE28

Map van C:\Program Files\Firebird
21-10-2011 11:20 <DIR> .
21-10-2011 11:20 <DIR> ..
21-10-2011 11:19 <DIR> Firebird_2_1
21-10-2011 11:27 <DIR> Firebird_2_5
                0 bestand(en)      0 bytes
                4 map(en)      86.358.855.680 bytes beschikbaar
C:\Program Files\Firebird>cd Firebird_2_5
C:\Program Files\Firebird\Firebird_2_5>cd "C:\Program Files\Firebird\Firebird_2_5\bin"
C:\Program Files\Firebird\Firebird_2_5\bin>
  
```


A "normal" backup:

```
gbak -v -t -user SYSDBA -password "masterkey" "C:\Program Files\MRS Software\Database\MRS.GDB" c:\backups\MRS.GBK
```


A "normal" restore:

```
gbak -c -v -user SYSDBA -password masterkey c:\backups\MRS.GBK "C:\Program Files\MRS Software\Database\MRS.GDB"
```

13.5 Institute Management

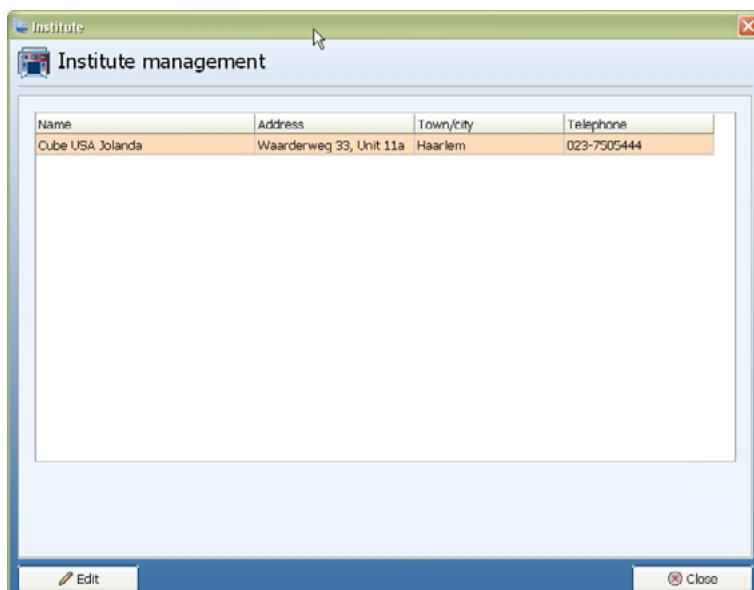

Select the line <Type here your institute name> and choose <Edit>. Fill in your details and save.

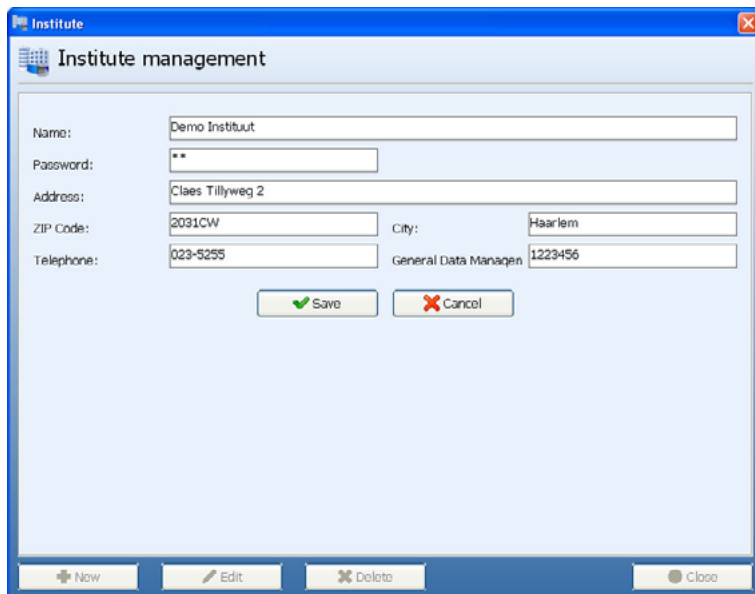

Institute management

Name: Demo Instituut

Password: **

Address: Claes Tillyweg 2

ZIP Code: 2031CW City: Haarlem

Telephone: 023-5255 General Data Manager: 1223456

14. Shortcuts

F2	New Patient
F3	Edit Patient
F4	Select Patient
F5	Rehab
F6	Functional exercises
F7	Coordination test
F8	Isometric training
F10	Administrator settings
F11	Test history
Ctrl H	Extended history




MR SYSTEMS

MONITORED REHAB SYSTEMS


OTHER PRODUCTS:



Kneelax

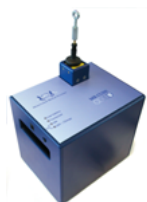

MR Cube

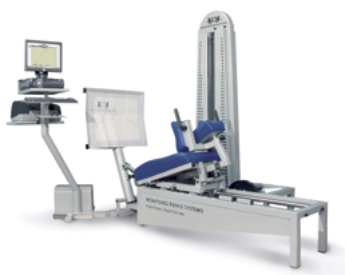

Functional Squat



Easy Slide



MR Wiggle

American customers only:

Monitored Rehab Systems B.V.

Waarderweg 33 - Unit 11A, 2031 BN Haarlem
The Netherlands

Phone: +31 (0)23 750 5444 | Fax: +31 (0)23 750 5445
info@mrsystems.nl | www.mrsystems.nl

MRS USA Inc.

1023 Red Dunes Run, Avon IN 46123

Phone 513-673-9371

j.vandever@mrsystems-usa.com

www.mrsystems-usa.com